

How to Update RA & RB

Internet Connection

Connect the manikin to your computer using the supplied USB cable. (Wireless connections will not work for manikin software updates).

[Click here to update your manikin](#)


No Internet Connection

If there is no internet connection where the manikins are located, [download the update-file](#) to your computer.

Connect the manikin to your computer using the supplied USB cable.

Run the update-file 'QCPRManikinUpdate_RA_1_3_1_17352_RB_1_3_1_17352'.

Select your manikin and hit 'Update'.

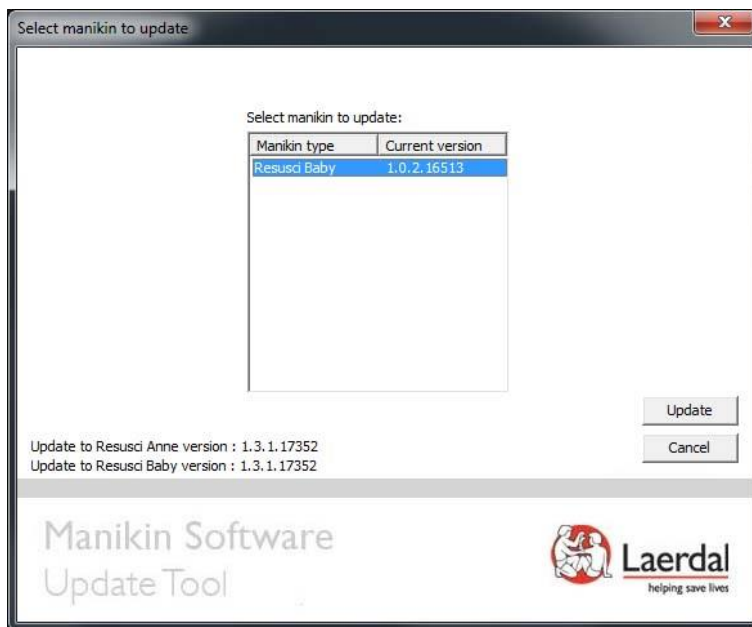

NB: The software update is embedded in the update tool. If there are new updates available, a new update-file must be downloaded from [here](#).