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### 1. CONTENU DE L'EMBALLAGE

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## ENGLISH Directions for Use

### 1. PACKAGE CONTENTS

- WALL BRACKET
- DIRECTIONS FOR USE
- DRILLING TEMPLATE

### 2. INTENDED USE

The LSU Wall Bracket is intended to hold and provide power connection to the Laerdal Suction Unit inside an ambulance, hospital or clinic cart. Cat. No 782640 is without power connection.

### 3. OVERVIEW

- A - Locking handle
- B - Wall Bracket upper plate
- C - Upper screw holes
- D - Power connector (not included on 782640)
- E - Steering pin
- F - Hooks
- G - Lower screw hole
- H - Wall Bracket lower plate
- I - Guide pins
- J - Cable (not on 782640)

### 4. CATALOG NUMBERS

782600	LSU Wall Bracket with DC power cord, 1.6 meter
782601	LSU Wall Bracket with DC power cord, 3 meter
782610	LSU Wall Bracket with AC power cord (US)
782620	LSU Wall Bracket with AC power cord (EU)
782630	LSU Wall Bracket with AC power cord (UK)
782640	LSU Wall Bracket without power cord

### 5. MOUNTING INSTRUCTIONS LSU WALL BRACKET

Please refer to the enclosed template while following the instructions below (The template is a 1:1 scale and can be used directly on the wall while mounting the Wall Bracket)

- Choose a suitable indoor location with a sufficiently clear area for the installation of the Wall Bracket.
- Select fastening hardware that is suitable to hold the Wall Bracket firmly in place on the chosen mounting surface. The screws must be of adequate size and strength to hold the Wall Bracket to the wall with more than 20g force (500N/screw)
- The Wall Bracket must be mounted in a vertical position using suitable spacers if necessary.
- Tape the template to the chosen mounting surface and drill the holes according to the template.
- Partially install upper screws [C] (if the screws are not properly inserted into the wall the risk is present for loosening the screws into the Wall Bracket)
- Mount the Wall Bracket over the 2 screws and move it downward until it fastens securely into place.
- Install the screw in the lower screw hole [G], and tighten the upper screws.
- Do not bend the cable over sharp edges.

*Note: We recommend mounting the LSU Wall Bracket on fixed surfaces. Mounting on e.g. a swinging door may generate forces beyond 20 g.*

© 2011 Laerdal Medical AS. All rights reserved. Printed in Norway 7988 rev B

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### 6. INSTALLATION OF LSU

- Guide the bottom-grooves under the LSU onto the Wall Bracket guide pins [I].
- Lift the locking handle [A]
- Press the LSU firmly into place.
- Lower the locking handle while holding the LSU in place. (The movable parts might cause hands or fingers to be squeezed or pinched).

*Note: If excessive force is needed to lower the locking handle: a. the LSU is not placed correctly on the guide pins or b. a foreign object is between the LSU and the Wall Bracket. If either occurs the hooks might not secure the LSU. Lift the locking handle and relocate the LSU and/or remove foreign object.*

Make sure the LSU is safely installed. A correct placed LSU shall have the locking handle fully down over the LSU handle. The LSU External Power LED shall be lit (except 782640 LSU Wall Bracket without cable).

### 7. RELEASE OF LSU

- Lift the Locking handle [A]
- Remove the LSU

### 8. CLEANING

Use a cloth that is dampened with a mild detergent to clean the external surfaces.

### 9. TEMPERATURE

Use: from -5°C up to +40°C  
Storage: from -40°C up to +70 °C

### 10. WARRANTY

See Global Warranty for terms and conditions of warranty and it is also available on [www.laerdal.com](http://www.laerdal.com)

### 11. SERVICE

There are no serviceable parts.

### 12. COMPONENTS AND DISMANTLING

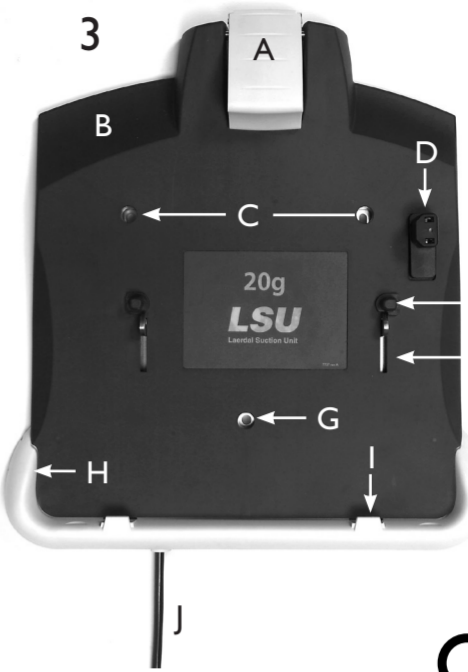
We recommend when discarding the Wall Bracket to dismantle and discard according to local protocol. The Wall Bracket is dismantled simply by unscrewing the screws on the back side of the wall bracket, remove the different components and sort according to the table below.

Blue plastic parts	PA with glass fibers
Labels	PC - Plastics
Back plate	Aluminium
Hook plate	Steel
Springs	Steel
Hinge pins	Steel
Screws	Steel
Cable	Cable (not on 782640)

The Wall Bracket can also be delivered to your local Laerdal Medical representative for dismantling/disposal. Laerdal Medical AS will not charge any fee for the dismantling; the cost of the shipment shall be carried by the sender.

# LSU Wall Bracket

Manufacturer:  
Laerdal Medical AS  
Stavanger, Norway

Distribution:  
For World Wide distribution, see Global Warranty or [www.laerdal.com](http://www.laerdal.com).

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