# Resusci Anne Advanced SkillTrainer

User Guide

English

[www.laerdal.com](http://www.laerdal.com)

**Laerdal**  
helping save lives

## Resusci Anne Advanced SkillTrainer

---

The Resusci Anne Advanced SkillTrainer is designed to meet the core learning objectives of intermediate to advanced healthcare. The manikin supports training for CPR, defibrillation, IV therapy and basic to intermediate (supraglottic) airway management procedures. Adding the optional first aid and rescue limbs also enables first aid and extrication practice.

SimPad and SkillReporting software enables documentation of key events and comprehensive instructor-to-student debriefing.

---

Note

Photos/illustrations may vary from product.

# Contents


---

1	Items included	4
2	Get started	6
3	Use	8
4	Maintenance	12
5	Cleaning	16

See Important Product Information pamphlet for more information or visit [www.laerdal.com](http://www.laerdal.com)

# 1

## Items Included

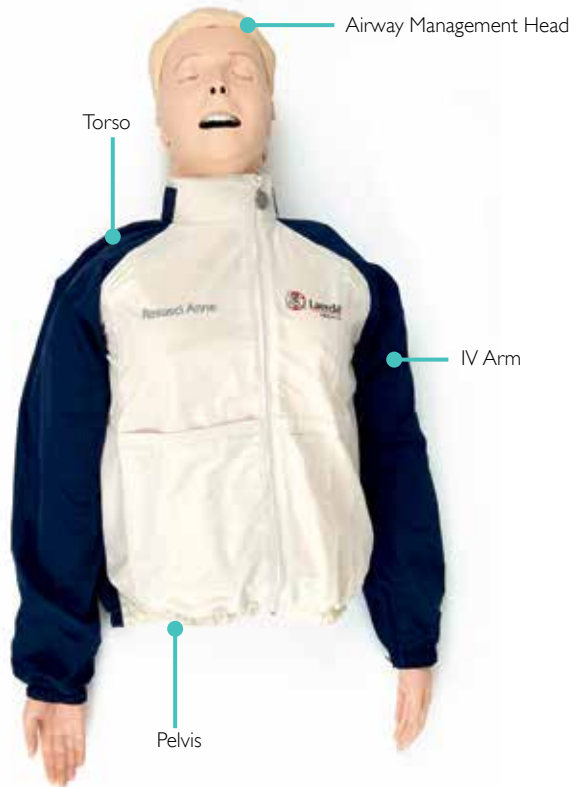SimPad (Link Box with lithium-ion battery installed in manikin).  
SimPad Wrist Strap and Sleeve

Manual  
Defibrillation  
Plates

3 Eye Sets (normal,  
constricted, dilated pupils)

# Items Included

Lower Body

Carry case

2 x AC Adapters

USB Cable

Simulated Blood

Lubricant

### Charge battery

Charge battery using AC adapter included.

### Connect external microphone (not included)

Plug microphone into jack on side of Link Box. Pink extension cable provided.

### Prepare IV Arm

Select a vein and attach IV tubing to IV bag.

Allow fluid to flow through arm and out of other vein. Clamp off flow (with a knot or clamp) of fluid from open vein.

## Get Started

---

### Connect SimPad to manikin

Refer to SimPad User Guide for instructions.

### Connect PC to manikin (optional for SimPad Patient Monitor)

To use PC software, connect Ethernet cable between manikin and a computer. (Cable not included).

On/Off

On/Off button

Head tilt/Chin lift

Jaw thrust

# Use

## Ventilation

Airway management tools can be used.

## Chest compressions

Manikin will detect correct hand position.

## Defibrillation

Defibrillation connectors and ECG connectors can be used with AEDs or manual defibrillators.

### Note

Defibrillation must be performed on defibrillation connectors only. Observe safety precautions for use of defibrillators.

### Caution

- To prevent overheating, provide maximum 2 x 360J discharges per minute.
- Keep manikin chest dry. Ensure manikin remains dry when using IV arm.
- To prevent chest pitting, do not apply conductive gel or conductive defibrillation pads intended for patient use.

## Lung

The manikin is equipped with one disposable unilateral lung.

## IV arm

Use left arm for IV training.

### Note

If training session involves administration of fluids and/or drugs, empty arm immediately following session.

## Pupils

Use different pupils to simulate various patient conditions.

# Use

## Carotid pulses

Carotid pulses  
synchronize to ECG.

### Note

Do not use excessive force when palpating the carotid pulse as this will result in no pulse felt.

## To run simulations

Refer to SimPad User Guide for information on how to control the manikin during sessions.

### Pupil replacement

Pull back hair.  
Detach the top of face  
skin and ears and fold  
back until eyes are  
exposed.

Detach eye from head.  
To insert new eye,  
place in hole and press  
into place.

Reattach face and hair  
by reversing procedure.

### Change battery

Only use Laerdal  
lithium-ion battery.

Release strap securing  
battery to Link Box.

## Maintenance

Remove battery.

Detach battery connector from Link Box.

Replace battery. Plug in battery connector.

Secure battery with strap.

## Ventilation

- Lubricate airway with two or three sprays prior to starting your training session.
- Lubricate airway management tools liberally prior to starting intubations. Consider whether additional lubrication is needed later in the session.
- Use only lubrication provided. Do not use silicon or other lubricant, as this may damage manikin.

Note

Head contains electrical components. Due to inability to sanitize airway, mouth-to-mouth/ mouth-to-mask ventilation should not be performed.

## Lung replacement

Detach chest skin to expose lung.

Detach tubing from blue airway connector located under lung plate.

Reattach tubing to blue airway connector and attach new lung.


Press gently to secure lung plate.

## Maintenance

### Stomach replacement

Detach chest skin and lift lung plate to locate and easily access stomach tubing.

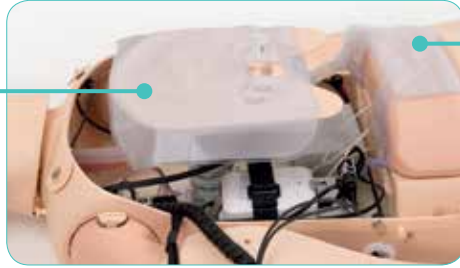Lift stomach piece.

Detach stomach bladder and tubing from stomach valve. Attach new stomach bladder.

Replace stomach piece. Reattach chest skin.

### Leg replacement

Detach lower chest skin and fold back chest skin.  
Remove stomach foam.

Remove bolt connector.

Disengage pelvis from torso.

To replace legs, reverse procedure.

Preventive maintenance is the best method to ensure optimal operation. General inspection should be conducted at regular intervals.

Periodically wash all skin parts that are not regularly sanitized using warm water and soap or manikin wipes.

Most stains can be removed with warm water and soap or Laerdal manikin wipes. Before use, test cleaning agents on a non-critical area (e.g. under the chest cover).

Note

Pigments from lipstick and pens may be impossible to remove. Avoid using coloured plastic gloves when handling the manikin, as they may cause discolouration.

# Resusci Anne Advanced SkillTrainer

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© 2013 Laerdal Medical AS. All rights reserved.