One disposable airway for

Resusci Anne

24 pack Cat. No. 152250



Airway is designed for disposal after each class.



Having removed face, remove neck skin from chin.



Pull to remove jaw.



Detach airway connector and pull inspiration tube through the neck.



Pull expiration tube from its retainer in the manikin's side.



Connect expiration and inspiration tubes.



Slide tubing through slot in lung plate, to detach airway assembly. Discard used airway.



Slide tubing through slot in lung plate, with lung on top.




Push on lung connector until fully seated in lung plate. Make sure the ventilation slider can move freely.



Push expiration tube into retainer in manikin's side until end protrudes about 2-3 cm (1").



Push inspiration tube through neck and click airway connector into its holder.

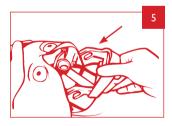

Install jaw by sliding its forked portions over steering pins in the inner head.



Fold neck skin over jaw, and install face.