



SimBaby

Quick Function Check Pre- and Post LLEAP Implementation

SimBaby S/N: _____

Equipment: Heartstart defibrillator (optional), stethoscope, BVM

		PRE	POST
1	Confirm chest rise for variable respiration rates	<input type="checkbox"/>	<input type="checkbox"/>
2	Check for Lung Compliance	<input type="checkbox"/>	<input type="checkbox"/>
3	Confirm activation and deactivation of "Left Pneumothorax"	<input type="checkbox"/>	<input type="checkbox"/>
4	Confirm "Head Tilt"	<input type="checkbox"/>	<input type="checkbox"/>
5	Confirm "Jaw thrust"	<input type="checkbox"/>	<input type="checkbox"/>
6	Confirm "Stomach Distension"	<input type="checkbox"/>	<input type="checkbox"/>
7	Check for "airway resistance"	<input type="checkbox"/>	<input type="checkbox"/>
8	Confirm "Tounge Edema"	<input type="checkbox"/>	<input type="checkbox"/>
9	Check for "Fontanelle" function	<input type="checkbox"/>	<input type="checkbox"/>
10	Confirm Cyanosis	<input type="checkbox"/>	<input type="checkbox"/>
11	Activate the function "Baby cry" and listen for a clear sound form the head loudspeaker.	<input type="checkbox"/>	<input type="checkbox"/>
12	Check for pulse at the left arm (radial, brachial) and both femoral sites	<input type="checkbox"/>	<input type="checkbox"/>
13	Activate the "Mute" function and listen for the breathing sound of the 2 loudspeakers on the left side and the 2 loudspeakers on the right side	<input type="checkbox"/>	<input type="checkbox"/>
14	Listen for the heart sound on the upper loudspeaker and the same for the lower loudspeaker. Deactivate the "Mute" function.	<input type="checkbox"/>	<input type="checkbox"/>
15	Confirm Korotkoff sound in the left arm while you release the inflated air	<input type="checkbox"/>	<input type="checkbox"/>
16	Confirm compressions, check log	<input type="checkbox"/>	<input type="checkbox"/>
17	Optional: Attach a defibrillator to the monitoring contacts. Select leads and observe that you have ECG reading.	<input type="checkbox"/>	<input type="checkbox"/>
Defibrillation and ECG (Optional)			
18	Attach a defibrillator to the monitoring contacts. Select leads and observe that you have ECG reading.	<input type="checkbox"/>	<input type="checkbox"/>
19	Attach the defibrillator to the defib Connectors. Select for running rhythm V.Fibr and waiting rhythm To Sinus. Select 10J (optional 150 J), analyze and press to shock. Observe that the rhythm changes from V.Fibr. To Sinus rhythm.	<input type="checkbox"/>	<input type="checkbox"/>
Network testing of the system			
20	Test the network setup of the system (manikin, Instructor PC, PM, SimPad) E.g. PC – Manikin – SimPad PC – ELO - Manikin NOTE: In the SR for this setup, please file the network diagram for future need and troubleshooting	<input type="checkbox"/>	<input type="checkbox"/>
Debrief System (if applicable)			
21	If the manikin is connected to a debrief system (SimView) run one minute of recording and confirm playback is ok NOTE: 3rd party debriefing system is not included in the Legacy to LLEAP Implementation Service	<input type="checkbox"/>	<input type="checkbox"/>



Comments:

Test performed by: _____

Customer Signature: _____

Date: _____

sheet to be attached to Implementation SR