

SimMan 3G Simulator

Quick Function Check Pre- and Post LLEAP Implementation

****MAKE SURE POST UPGRADE CHECK IS DONE WIRELESSLY****

SimMan 3G S/N: _____

Equipment: Heartstart defibrillator (optional), stethoscope, intubations and ventilation equipment, distilled water, BVM, flashlight

		PRE	POST
1	Perform direct laryngoscope, inspect: epiglottis, vocal cords, upper trachea, esophageal area for damage, rupture or leakage	<input type="checkbox"/>	<input type="checkbox"/>
3	Confirm RFID tag is registered both in the arm and in the jaw	<input type="checkbox"/>	<input type="checkbox"/>
4	Inject distilled water to the IV arm and confirm and the value registered by SW is ok	<input type="checkbox"/>	<input type="checkbox"/>
5	Confirm head tilt/chin lift	<input type="checkbox"/>	<input type="checkbox"/>
6	Confirm DCRM	<input type="checkbox"/>	<input type="checkbox"/>
7	Check for cyanosis	<input type="checkbox"/>	<input type="checkbox"/>
8	Confirm eyes blinking	<input type="checkbox"/>	<input type="checkbox"/>
9	Confirm pupils are reacting to light	<input type="checkbox"/>	<input type="checkbox"/>
10	Confirm presence of pulses	<input type="checkbox"/>	<input type="checkbox"/>
11	Confirm chest rise for variable respiration rates	<input type="checkbox"/>	<input type="checkbox"/>
12	Confirm BVM gives easily visible chest rise	<input type="checkbox"/>	<input type="checkbox"/>
13	Confirm convulsions	<input type="checkbox"/>	<input type="checkbox"/>
14	Confirm presence of heart sounds	<input type="checkbox"/>	<input type="checkbox"/>
15	Confirm presence of lung sounds	<input type="checkbox"/>	<input type="checkbox"/>
16	Confirm presence of vocal sounds	<input type="checkbox"/>	<input type="checkbox"/>
17	Confirm presence of bowel sounds	<input type="checkbox"/>	<input type="checkbox"/>
18	Confirm Korotkoff sounds	<input type="checkbox"/>	<input type="checkbox"/>
19	ECG works (select different ECG rhythms)	<input type="checkbox"/>	<input type="checkbox"/>
20	Confirm compressions, check log	<input type="checkbox"/>	<input type="checkbox"/>
Network testing of the system			
21	Test the network setup of the system (manikin, Instructor PC, PM, SimPad) E.g. PC – Manikin – SimPad PC – ELO - Manikin NOTE: In the SR for this setup, please file the network diagram for future need and troubleshooting	<input type="checkbox"/>	<input type="checkbox"/>
Debrief System (if applicable)			
22	If the manikin is connected to a debrief system (SimView) run one minute of recording and confirm playback is ok NOTE: 3rd party debriefing system is not included in the Legacy to LLEAP Implementation Service	<input type="checkbox"/>	<input type="checkbox"/>



Comments:

Test performed by: _____

Customer Signature: _____

Date: _____

sheet to be attached to Implementation SR