



SimMom

Quick Function Check Pre- and Post LLEAP Implementation

SimMom S/N: _____

Equipment: Heartstart defibrillator (optional), stethoscope, BVM

		PRE	POST
1	Confirm spontaneous breathing	<input type="checkbox"/>	<input type="checkbox"/>
2	Confirm lung blockage (shut off L/R or Both)	<input type="checkbox"/>	<input type="checkbox"/>
3	Confirm Tongue swelling	<input type="checkbox"/>	<input type="checkbox"/>
4	Confirm Head tilt/chin lift	<input type="checkbox"/>	<input type="checkbox"/>
5	Confirm Jaw trust	<input type="checkbox"/>	<input type="checkbox"/>
6	Confirm Seizure (tonic – clonic)	<input type="checkbox"/>	<input type="checkbox"/>
7	Attach “ boggy uterus ” to green labeled tube and inflate it, hold for 5-7 seconds, and deflate uterus – check log	<input type="checkbox"/>	<input type="checkbox"/>
8	Check bilateral carotid pulses	<input type="checkbox"/>	<input type="checkbox"/>
9	Confirm presence of brachial pulse (<i>left arm</i>)	<input type="checkbox"/>	<input type="checkbox"/>
10	Confirm presence of brachial and radial pulses (<i>right arm</i>)	<input type="checkbox"/>	<input type="checkbox"/>
11	Set Blood Pressure to 120/80 and use stethoscope to confirm presence of BP sound in right arm	<input type="checkbox"/>	<input type="checkbox"/>
12	Confirm presence of heart sound of mother	<input type="checkbox"/>	<input type="checkbox"/>
13	Confirm presence of 3 x right lung sound	<input type="checkbox"/>	<input type="checkbox"/>
14	Confirm presence of 3 x left lung sound	<input type="checkbox"/>	<input type="checkbox"/>
15	Confirm presence of vocal sound from head, change sound and reconfirm	<input type="checkbox"/>	<input type="checkbox"/>
16	Confirm presence of bowel sound, change sound and reconfirm	<input type="checkbox"/>	<input type="checkbox"/>
17	Perform 3 chest compressions – check icon appears on screen	<input type="checkbox"/>	<input type="checkbox"/>
	ECG and Electrical Therapy (Optional) Warning: All standard safety precautions when using the defibrillator on the simulator must be taken and Cautions & Warnings in the SimMom Direction for Use (DFU) must be followed.		
18	Confirm presence of ECG at ¼ lead and at hands free (sternum and apex)	<input type="checkbox"/>	<input type="checkbox"/>
19	With running rhythm set at VF, waiting rhythm at NSR, confirm that defibrillation at 150J will move waiting to running rhythm	<input type="checkbox"/>	<input type="checkbox"/>
	Network testing of the system		



20	Test the network setup of the system (manikin, Instructor PC, PM, SimPad) E.g. PC – Manikin – SimPad PC – ELO - Manikin NOTE: In the SR for this setup, please file the network diagram for future need and troubleshooting	<input type="checkbox"/>	<input type="checkbox"/>
Debrief System (if applicable)			
21	If the manikin is connected to a debrief system (SimView) run one minute of recording and confirm playback is ok NOTE: 3rd party debriefing system is not included in the Legacy to LLEAP Implementation Service	<input type="checkbox"/>	<input type="checkbox"/>

Comments:

Test performed by: _____

Customer Signature: _____

Date: _____

sheet to be attached to Implementation SR