

SimNewB

Quick Function Check Pre- and Post LLEAP Implementation

SimNewB S/N: _____

Equipment: Heartstart defibrillator (optional), stethoscope, BVM

		PRE	POST
1	Confirm cyanosis is easily visible under light	<input type="checkbox"/>	<input type="checkbox"/>
2	Confirm chest rise is easily visible	<input type="checkbox"/>	<input type="checkbox"/>
3	Confirm vocal sounds synchronous with chest rise and breathing	<input type="checkbox"/>	<input type="checkbox"/>
4	Confirm heart sounds are audible	<input type="checkbox"/>	<input type="checkbox"/>
5	Confirm Korotkoff sounds on between systolic and diastolic	<input type="checkbox"/>	<input type="checkbox"/>
6	ECG works (select different ECG rhythms)	<input type="checkbox"/>	<input type="checkbox"/>
7	Brachial pulse is easily palpable and turns on pulses and pulses off	<input type="checkbox"/>	<input type="checkbox"/>
8	Umbilical pulse is easily palpable when ON	<input type="checkbox"/>	<input type="checkbox"/>
9	BVM gives easily visible chest rise	<input type="checkbox"/>	<input type="checkbox"/>
10	Partial closure of both lungs pops valve on bag and minimal chest rise	<input type="checkbox"/>	<input type="checkbox"/>
11	Partial closure of both lungs gives full rise when valve on bag held closed	<input type="checkbox"/>	<input type="checkbox"/>
12	Closure of right lung gives chest rise of left side only	<input type="checkbox"/>	<input type="checkbox"/>
13	Closure of left lung give chest of right side only	<input type="checkbox"/>	<input type="checkbox"/>
14	CPR Test	<input type="checkbox"/>	<input type="checkbox"/>
(Select level off)			
15	Selecting tonus lifts arms and legs off table	<input type="checkbox"/>	<input type="checkbox"/>
16	Selecting motion moves arms and legs	<input type="checkbox"/>	<input type="checkbox"/>
17	Selecting limp drops arms and legs to table	<input type="checkbox"/>	<input type="checkbox"/>
Network testing of the system			
18	Test the network setup of the system (manikin, Instructor PC, PM, SimPad) E.g. PC – Manikin – SimPad PC – ELO - Manikin NOTE: In the SR for this setup, please file the network diagram for future need and troubleshooting	<input type="checkbox"/>	<input type="checkbox"/>
Debrief System (if applicable)			
19	If the manikin is connected to a debrief system (SimView) run one minute of recording and confirm playback is ok NOTE: 3rd party debriefing system is not included in the Legacy to LLEAP Implementation Service	<input type="checkbox"/>	<input type="checkbox"/>



Comments:

Test performed by: _____

Customer Signature: _____

Date: _____

sheet to be attached to Implementation SR