HeartStart FR3 Training Pack

Pack de formation au HeartStart FR3

HeartStart FR3 Schulungspaket

HeartStart FR3-trainingspakket

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ENGLISH: About This Edition

The information in this guide applies to the FR3 Training Pack 989803150191 and HeartStart Training Pads III 989803150181. This information is subject to change. Please contact Philips at www.philips.com/productdocs or your local distributor for information on revisions.

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No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Manufacturer

Philips Medical Systems
Seattle, WA, USA 98121-1825

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Philips Medical Systems, Seattle, WA, USA

98121-1825

INTENTIONALLY BLANK
INTENTIONNELLEMENT LAISSÉ EN BLANC
ABSICHTLICH LEER GELASSEN
OPZETTELIJK ONBEDRUKT GELATEN
ESTE ESPACIO SE HA DEJADO EN BLANCO DELIBERADAMENTE
PAGINA LASCIATA INTENZIONALMENTE VUOTA
PÁGINA DEIXADA EM BRANCO INTENCIONALMENTE
SIDAN ÄR AVSIKTILIGEN TOM
SKAL VÆRE TOM
JÄTETTY TARKOITUKSELLA TYHJÄKSI
刻意留白
刻意留空
このページは空白です
이 페이지는 공백 페이지입니다

HEARTSTART FR3 TRAINING PACK

DESCRIPTION AND USE

The 989803150191 FR3 Training Pack includes the FR3 training battery, a set of HeartStart Training Pads III, a HeartStart Training Pads III interconnect cable, and an external manikin adapter. These reusable, optional accessories are designed for use with the HeartStart FR3 Defibrillator* (FR3) for demonstration and training applications. The lithium ion FR3 training battery is rechargeable using the 861394 battery charger only.† Replacement training pads [REF: 989803150181] can be ordered separately. The interconnect cable is intended for indefinite reuse; a spare interconnect cable can be ordered separately [REF: 989803150201].

Using the training battery and training pads with the FR3, you can choose any of three pre-set emergency scenarios. The training scenarios are available to trainers to supplement physician-authorized training programs. The training scenarios are not intended to teach basic lifesaving skills. The scenarios are described in the table on page 6.

For indications and instructions for using the FR3, refer to the *Guide to Setup, Operation, and Maintenance* or the *Instructions for Administrators* provided with the FR3.‡

NOTE: *The training battery and pads cannot be used to deliver defibrillation treatment. If the FR3 is needed for patient treatment during a training session, disconnect the training pads, remove the training battery, install an FR3 clinical use battery, and connect a set of HeartStart therapy pads.*

* These instructions are specific to use of the FR3 training battery and training pads with an FR3.

† The FR3 training battery must be fully charged before initial use. See the *Instructions for Use* provided with the charger for directions.

‡ See the *Guide to Setup, Operation, and Maintenance* provided with the FR3 for an explanation of symbols on the Training Pack and its packaging.

The FR3 training battery is yellow and has a long yellow label, or “tail,” to help avoid confusion with the FR3 clinical use batteries. Similarly, the HeartStart Training Pads III connector plug is yellow, to help prevent confusion with the blue connector of the HeartStart therapy pads used for patient treatment. In addition, the training pads cable has a red label that identifies its purpose as training.

TRAINING SETUP

REQUIRED EQUIPMENT

The FR3 Training Pack can be used with the FR3 to train in adult or pediatric defibrillation. The following equipment is required:

- HeartStart FR3 Defibrillator
- FR3 training battery, fully charged using the 861394 charger
- HeartStart Training Pads III* with an interconnect cable
- CPR training adult manikin with external manikin adapter [REF: M5089A] OR
- Fold-out Adult Pads Placement Guide† [REF: M5090A]

Other FR3 accessories, such as the FR3 system case, can also be used for training.

For training in infant/child defibrillation, you will also need:

- FR3 Infant/Child Key [REF: 989803150031] OR
- CPR training child manikin with external manikin adapter [REF: M5089A] AND a Fold-out Infant/Child Pads Placement Guide‡ [REF: 989803139281]

* If desired, the FR3 can be used with HeartStart Adult Training Pads [REF: 07-10900]. See “Connecting the Training Pads.”


† Metal areas on the pads placement guides allow the defibrillator to sense when the pads are properly applied.

‡ Metal areas on the pads placement guides allow the defibrillator to sense when the pads are properly applied.

INSTALLING THE FR3 TRAINING BATTERY

To install the training battery, follow the steps below:

1. Remove the clinical-use battery from the battery compartment on the back of the FR3.
2. Install the FR3 training battery into the battery compartment as shown. Be sure it clicks into place.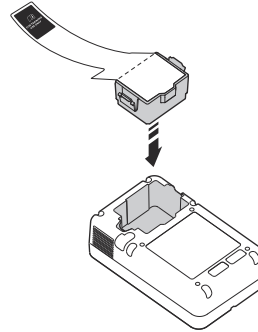

Install the training battery.

As soon as the training battery is installed, and throughout the training session, the FR3 display screen says "TRAINING." The green Ready light is on steady throughout the training session.


NOTE: The yellow tail of the training battery is designed to hang outside a closed FR3 system case as a reminder that the unit is not equipped for clinical use.

CONNECTING THE TRAINING PADS

If you want to simulate use of the FR3 equipped with pre-connected pads, press the On/Off button to put the FR3 into training standby mode, then connect the Training Pads III as described below.

1. Be sure that the FR3 training battery is installed in the FR3 and that the FR3 green Ready light is on solid (not flashing).
2. Remove the Training Pads III from their storage bag, leaving them on their liner. Retain the pads storage bag.


3. Remove the interconnect cable from its storage bag. Connect the pads cable to the interconnect cable.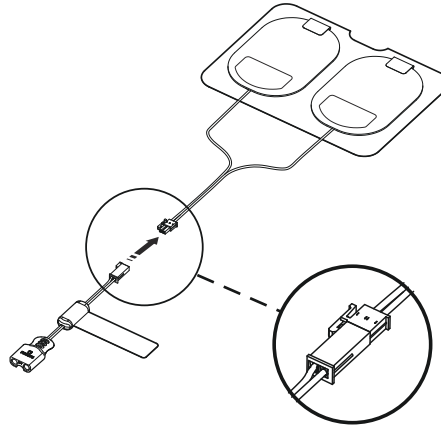

Connect the pads cable to the interconnect cable.

4. Plug the interconnect cable's yellow connector into the connector socket on the front of the FR3.

NOTE: If you do not pre-connect the Training Pads III, the FR3 prompts you to connect the pads when you run a training scenario.*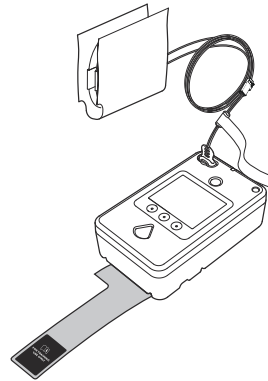


Connect the training pads.

5. If you are using an FR3 system case, **DO NOT USE THE PADS SENTRY with the training pads.** Remove the Pads Sentry from the case as follows:


* If you are using HeartStart Adult Training Pads, do not connect the pads to the FR3 until *after* they have been placed on the training manikin.

- Gently pull the Pads Sentry back to disengage the snap-tab at its open end (1).
- Lift the Pads Sentry out of its compartment (2).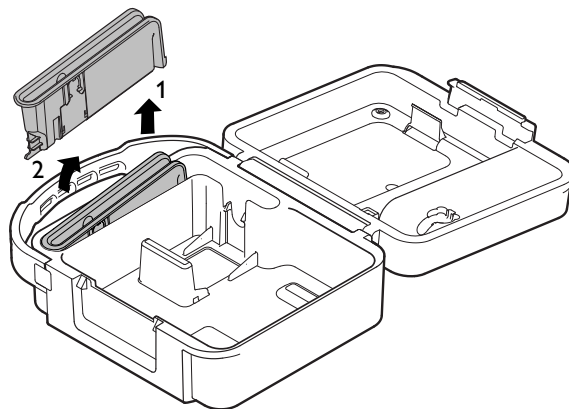

Remove the Pads Sentry from the case.

6. Place the pads, on their liner, in the empty compartment in the bottom of the carry case.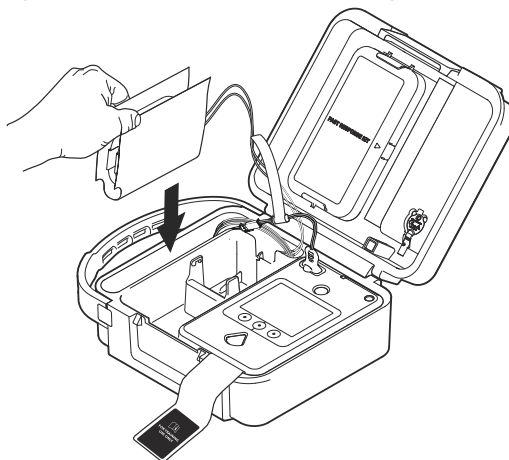

Insert pads on liner into case.

NOTE: If you are using an FR3 system case, be sure that the “tail” of the battery label remains outside the case, even when the case is closed, to show that the FR3 is not equipped for clinical use.


TRAINING IN INFANT/CHILD MODE

To train in infant/child defibrillation, you will need the FR3 Infant/Child Key [REF: 989803150031].

Typically, the Infant/Child Key is inserted after selecting and beginning a training scenario. Insert the Infant/Child Key into the port at the upper right of the front panel of the FR3.

With the Infant/Child Key inserted, the FR3 automatically operates in infant/child training mode. It provides voice instructions, including coaching for CPR protocols, appropriate for an infant or child.

If the Infant/Child Key is removed during a training scenario, the FR3 returns to adult training mode and restarts the scenario. It provides voice instructions, including coaching for CPR protocols, appropriate for an adult.

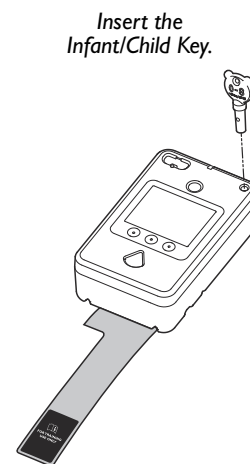

PHILIPS MEDICAL SYSTEMS

TRAINING SCENARIOS

DESCRIPTION

There are three training scenarios available when using the FR3 Training Pack with the FR3.

The training scenario descriptions in the following table are based on the default setup of the FR3. See the *FR3 Instructions for Administrators* for detailed information about the default configuration of the device. If its setup has been modified, FR3 behavior during training reflects its current configuration.

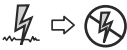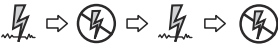SCENARIO NUMBER AND OVERVIEW	SCENARIO DETAILS
<p>Scenario 1: Shockable rhythm, one shock needed for conversion</p> 	<ul style="list-style-type: none"> • The FR3 detects a shockable rhythm, tells user to deliver a shock. • The FR3 provides a Basic CPR protocol. • The FR3 detects a non-shockable rhythm (no shock advised; NSA). • The FR3 provides an NSA CPR protocol.
<p>Scenario 2: Shockable rhythm, conversion, return to shockable rhythm, conversion*</p> 	<ul style="list-style-type: none"> • The FR3 detects a shockable rhythm, tells user to deliver a shock. • The FR3 provides a Basic CPR protocol. • The FR3 detects a non-shockable rhythm (no shock advised; NSA). • The FR3 provides an NSA CPR protocol. • The FR3 detects a shockable rhythm, tells user to deliver a shock. • The FR3 provides a Basic CPR protocol. • The FR3 detects a non-shockable rhythm.
<p>Scenario 3: Non-shockable rhythm</p> 	<ul style="list-style-type: none"> • The FR3 detects a non-shockable rhythm throughout; alternates between rhythm analysis and providing an NSA CPR protocol.


* Conversion from simulated ventricular fibrillation (VF). VF is the most common cause of sudden cardiac arrest.

SELECTING A TRAINING SCENARIO

When the training battery is installed, the display screen lists the training scenarios with icons. Scenario number 1, the default training scenario, is highlighted. Press the ✓ option button to select this scenario.

To select a different scenario, press the ▼ option button to scroll through the three scenarios in sequence. When you reach the scenario number you want, press the ✓ option button to select it.

The FR3 goes into training standby until you press the On/Off button or, if you are using an FR3 system case, you open the case to automatically turn on the FR3 and begin the scenario.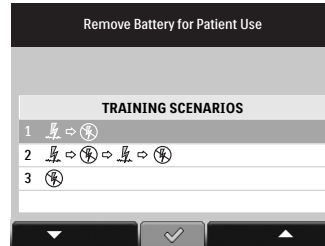

NOTE: As long as the training battery is not removed, you can delay the training session using the selected scenario for up to 30 minutes. After 30 minutes in training standby mode without user interaction, the defibrillator will start chirping to alert the user to either continue the training use or replace the training pads and training battery with HeartStart therapy pads and an FR3 clinical use battery.


RUNNING A TRAINING SCENARIO

To run the selected training scenario, treat the training pads exactly as you would the SMART Pads III. Use an Adult or Infant/Child Pads Placement Guide or a CPR training manikin with an external manikin adapter to simulate the patient. (Illustrated instructions for applying the manikin adapter to a training manikin are provided with the adapter.) Connect the pads to the FR3 as previously described.*


1. Turn on the FR3 by pressing the green On/Off button or, if you are using an FR3 system case, by opening the system case.
2. If you are training in infant/child defibrillation, insert the Infant/Child Key into the FR3.

* If you are using HeartStart Adult Training Pads, do not connect the pads to the FR3 until *after* they have been placed on the training manikin.

3. Follow the defibrillator prompts.
4. If you are using an FR3 system case, remove the training pads on their liner from the case.
5. Peel one training pad from the liner.
6. Place the pad on the Pads Placement Guide or the manikin. **Do not place the training pads on a person.** The FR3 screen displays an illustration of proper pads placement to help guide you. (Pads placement on infants and young children differs from placement on adults.) Press the pad down firmly.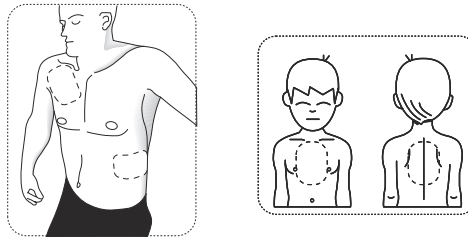

Placement of training pads on a Pads Placement Guide.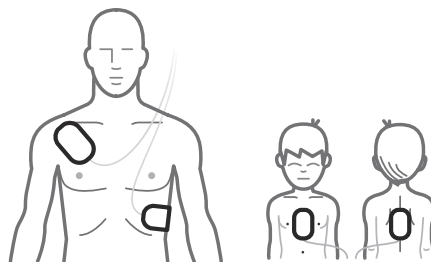

Placement of training pads on a training manikin.

7. Remove the other pad from the liner and place it on the Pads Placement Guide or the manikin. (It does not matter which pad is placed first or which pad is placed in which position.)
8. Continue to follow the FR3 instructions for the remainder of the scenario.
9. When the scenario is finished, or if you want to stop a scenario in process, press the On/Off button. The FR3 display returns to the main Training screen.

10. To repeat the same scenario, begin again by turning on the FR3. As long as the battery has not been removed, the FR3 automatically uses the last selected scenario. To select a different scenario, press the ▼ option button to scroll through the scenarios. When you reach the scenario number you want, press the ✓ option button to select it. Each time the training battery is inserted, the FR3 highlights the default training scenario number 1.

NOTE: To exit a training scenario at any time, press and hold the On/Off button. The FR3 then goes into training standby mode.

FR3 PERFORMANCE DURING TRAINING

When a training scenario is running, the FR3 behaves exactly as it would during emergency use, providing voice instructions, one or more simulated shocks, CPR protocol pauses, optional CPR guidance, etc., according to its setup configuration.*

For safety reasons, certain defibrillator features are not functional while the training pads are installed:

- The FR3 cannot be used for actual patient care; it cannot analyze patient heart rhythm or deliver a shock.
- The FR3 will not perform the power on self-test and periodic self-tests.
- If the FR3 is not used (no buttons are pressed) for 30 minutes with the training battery inserted and training pads connected, the defibrillator will start “chirping” to alert the user to either continue the training use or remove the training battery.


* In the unlikely event that the FR3 displays an error screen during training, remove the training battery and training pads from the FR3. Install the clinical battery and run a user-initiated test to confirm that the FR3 is ready for use in an emergency.

NOTE: If the battery power becomes low during training, the defibrillator will tell you to replace the battery. Recharge the training battery using the 861394 charger according to the directions provided with the charger.

AFTER USING THE HEARTSTART TRAINING PADS III

After you have finished using the FR3 for training, press the On/Off button and allow the defibrillator to shut down.

1. Remove each pad from the Pads Placement Guide or manikin and replace it on the liner.
2. Disconnect the training pads connector from the defibrillator.
3. Return the training pads, on their liner, to their storage bag and store it for future use.
4. Remove the training battery and install a clinical use battery.
5. Connect a set of SMART Pads III.
6. If you are using an FR3 system case, reinstall the Pads Sentry and store the SMART Pads III, on their liner, in the Pads Sentry.

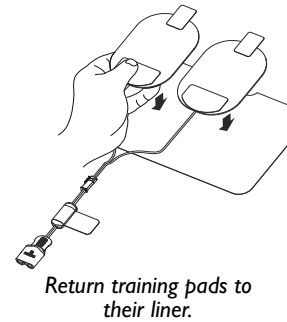IMPORTANT: To avoid confusion when using the FR3 in an emergency, be sure to store the training pads and training battery separately from the defibrillator.

MAXIMIZING SERVICE LIFE

The HeartStart Training Pads III are designed for up to 100 uses. To help ensure continued performance, be sure

to return the pads to their storage bag after each use.
Avoid leaving the pads in heat or direct sunlight.

NOTE: Please contact your local Philips representative if you have any questions about using the FR3 Training Pack or need technical support. In the United States, please call 1-800-263-3342.

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Niets uit deze uitgave mag in enige vorm of op enige manier worden veelevoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Medizin Systeme Böblingen GmbH
Philips Electronics Hong Kong Ltd.