
## Laerdal Silicone Masks

ENGLISH	Directions for Use	NORSK	Bruksanvisning
ESPAÑOL	Instrucciones de utilización	SVENSKA	Bruksanvisning
ITALIANO	Instruzioni per l'uso	SUOMI	Käyttöohje
FRANÇAIS	Mode d'emploi	DANSK	Brugsanvisning
DEUTSCH	Gebrauchsanweisung	日本語	取扱説明書
NEDERLANDS	Handleiding		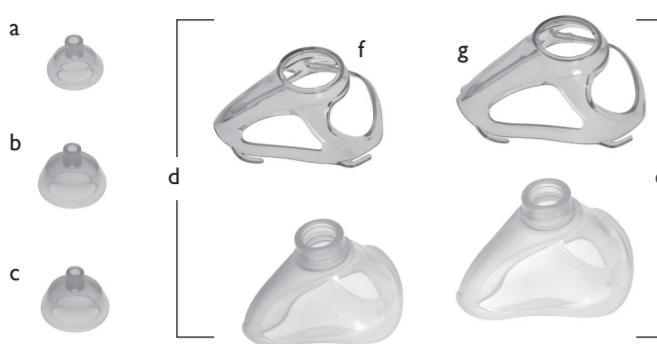

www.laerdal.com

### ENGLISH Directions for Use

#### Intended Use

Laerdal Silicone Masks are intended to be used for ventilation of human patients.

The Multi-Function Mask Cover is intended to be used to add additional support, when needed, to the Laerdal Silicone Masks (size 3-4, 4-5+) and to allow for strapping of the Laerdal Silicone Masks to the patient.

#### Decontamination

Thorough decontamination of the Laerdal Silicone Masks and Multi-Function Mask Covers is necessary after each use. It is also recommended to decontaminate the mask and mask cover prior to first time use.

#### Activity

- 1. Pre-rinse Disassemble the mask and cover (if applicable) and rinse in lukewarm (<43°C) water before exposure to detergent.
- 2. Washing Wash mask and cover in hot water (60°-70°C) with a detergent compatible with the mask and cover material (see materials list).
- 3. Rinse Rinse the mask and cover in clean water until all traces of detergent are removed. Allow to air dry.
- 4. Disinfection/Sterilization\* It is recommended to use one of the following methods:

Method	Process parameters	Post-treatment
	Parameters/Concentration	Exposure time

Sterilisation			
Steam Autoclaving	Autoclave at 132-137°C	15 minutes	Allow parts to cool
High-level disinfection			
Cidex OPA (orthophtaldehyde)	Conc.: 0.55% Ambient temperature	60 minutes	Rinse all parts thoroughly in sterile water. Allow to dry
Sodium Hypo-Chlorite (Not cleared for use in the US)	Conc.: 0.5% Ambient temperature	20 minutes	Rinse all parts thoroughly in sterile water. Allow to dry

\* Sterilisation Sterile/Sterility - State of being free from all living micro-organisms; in practice, usually described as a probability function, e.g. the probability of a surviving micro-organism being one in a million.

\*\* The sterilisation methods apply to all parts except head-straps.

High-level disinfection methods apply to all parts.

#### Technical Information

**Material:** Mask – Silicone (SI)  
Multi-Function Mask Cover – Polysulfone (PSU)  
HeadStrap - PolyVinylChloride (PVC)  
Attachment Ring - Polysulfone (PSU)

**Operating environmental Conditions:** -18°C to 60°C  
15% to 95% relative humidity

**Storage environmental Conditions:** -40°C to 70°C  
40% to 95% relative humidity

Patient port on masks fit standard 15/22 mm connectors

**CE 2460** Product is in compliance with essential requirements of council directive 93/42 EEC Medical Device Directive

#### Models

- a 85 15 00 Infant Silicone Mask 0/0
- b 85 16 00 Infant Silicone Mask 0/1
- c 85 17 00 Infant Silicone Mask 2
- d 86 02 20 Child Silicone Mask 3-4 w/Multi-Function Mask Cover
- e 87 02 20 Adult Silicone Mask 4-5+ w/Multi-Function Mask Cover
- 87 22 20 Adult and Child Silicone Mask Set

#### Accessories

- f 86 52 00 Multi-Function Mask Cover (only) for Child Mask 3-4
- g 87 52 00 Multi-Function Mask Cover (only) for Adult Mask 4-5+
- 87 13 00 Attachment ring for standard head strap (fits 0/0, 0/1, 2 masks)
- 87 04 00 Laerdal Head Strap with Laerdal Attachment Ring\*\*

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**Manufactured by**  
LAERDAL MEDICAL AS  
PO Box 377, N-4002 Stavanger  
Tel. +47 51 51 17 00, Fax +47 51 52 35 57  
E-mail: laerdal.norway@laerdal.no

**Distributed in USA by**  
LAERDAL MEDICAL CORPORATION  
167 Myers Corners Road, PO Box 1840  
Wappingers Falls, New York 12590-8840  
Tel. (800) 431-1055, +1 (845) 297-7770  
Fax (800) 227-1143, +1 (845) 298-4545  
E-mail: customerservice@laerdal.com

**Distributed in Canada by:**  
LAERDAL MEDICAL CANADA LTD.  
151 Nashdene Rd, Unit #45  
Toronto, ON, Canada M1V 4C3  
Phone: (905) 270-1111  
Toll free 888-222-AERDAL (523-7325)  
ou en français (800) 567-9987  
Fax +1 (416) 298-8016  
E-mail: svelvelsen@laerdal.ca

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