

# Resusci Anne QCPR & SimPad

Software Upgrade



## Items Needed

---



PC



SimPad



Resusci Anne QCPR



USB cable - SimPad to RA QCPR



USB cable - PC to SimPad

## Step 1

Go to [www.laerdal.com](http://www.laerdal.com) and download latest version of SimPad software to your PC.

### Note

These instructions are applicable for both SimPad and SimPad PLUS. Ensure you choose the appropriate product from the list on [www.laerdal.com](http://www.laerdal.com).

The screenshots illustrate the navigation path to download the latest SimPad software. The top-left screenshot shows the 'Nursing Anne Simulator' page. The top-right screenshot shows the 'Product and Software Downloads' page, which lists various software products. The bottom-right screenshot shows the 'Downloads for SimPad PLUS' page, which provides detailed information about the software updates, including the version number (6.1.6.3.31) and the release date (26/04/2022).



## Latest software available

XXX MB  
DD.MM.YYYY

### Note

Make sure you download the latest software available, as the name may vary depending on the date for the upgrade.

# Setup

## Step 2


Connect SimPad to PC with USB cable.

 Note

*Make sure that the unit has a battery capacity of at least 30% or is connected to the DC adapter.*




The SimPad will appear as a disk drive on the computer.

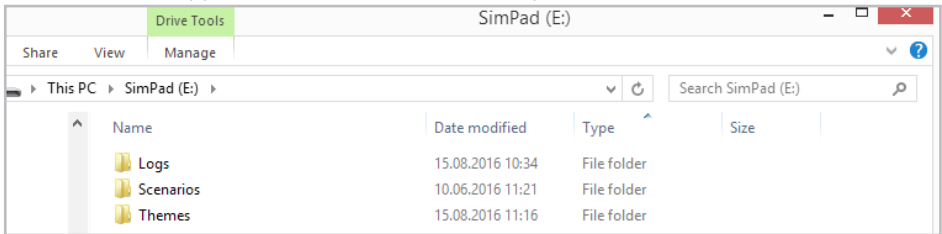

## Step 3

Copy the downloaded software file to SimPad.

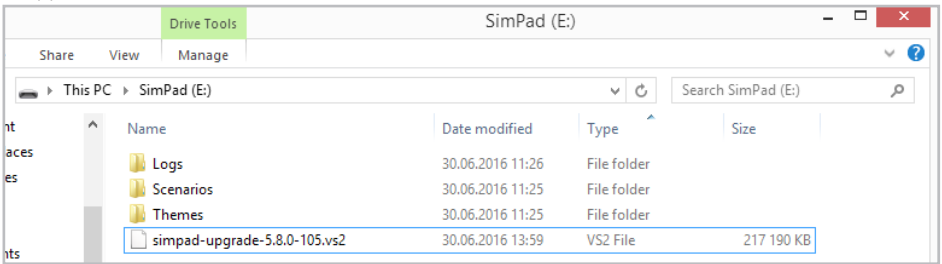

## Step 4

Safely remove SimPad from PC.


 Note

*File names may vary depending on the product e.g. vs2 (SimPad) and .spu (SimPad PLUS).*

## Step 5

### SimPad updating

The SimPad will now start updating and show "Updating" in the display and blink with the LED.

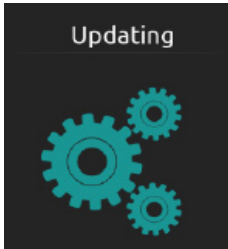


This update takes approx. 2 minutes.

When the SimPad Main screen re-appears, the upgrade is complete.

## Step 6


### Connect SimPad to manikin.

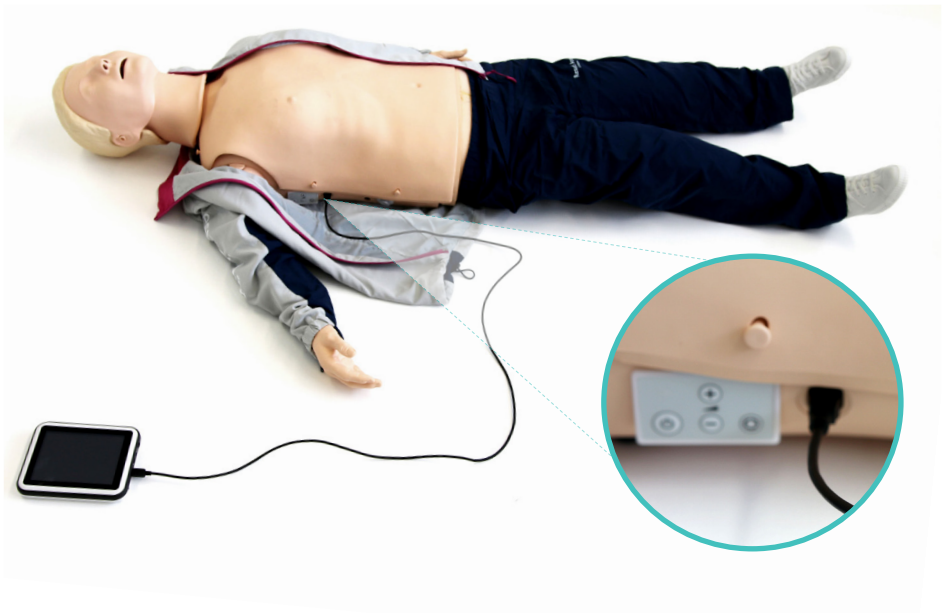


# Setup

## Step 7


Update manikin firmware.

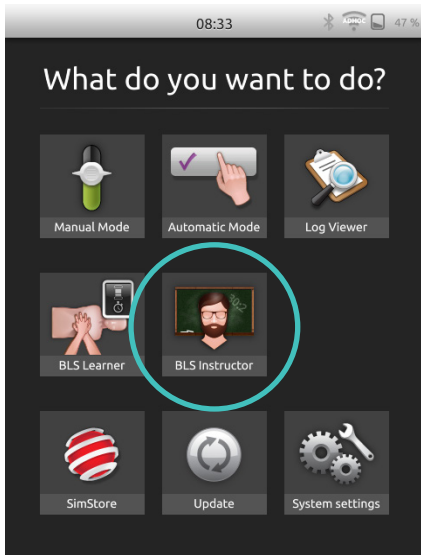


On SimPad, select BLS instructor app.

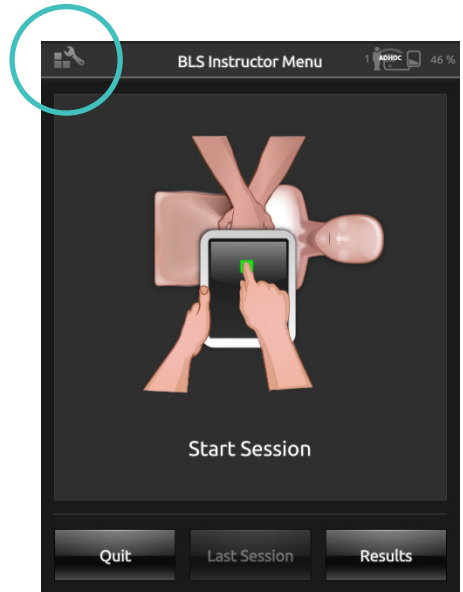

Open BLS settings in the top left corner.

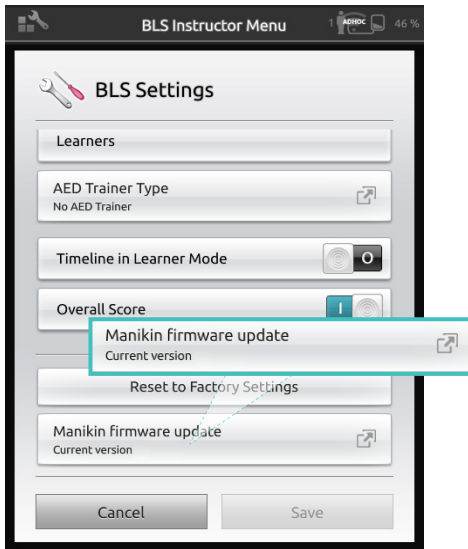


Scroll to the bottom and choose "manikin firmware update"

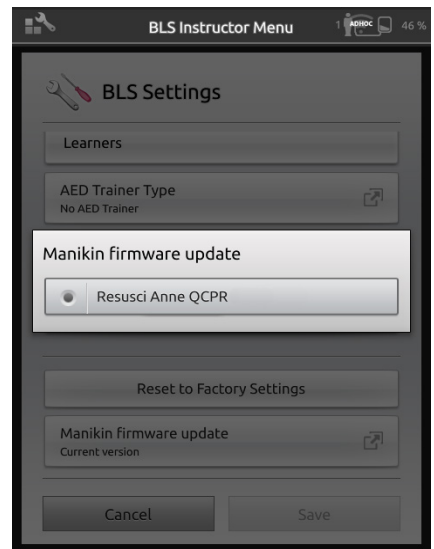

Choose the latest version and wait for the update to complete.

## Step 8

Disconnect SimPad from manikin.

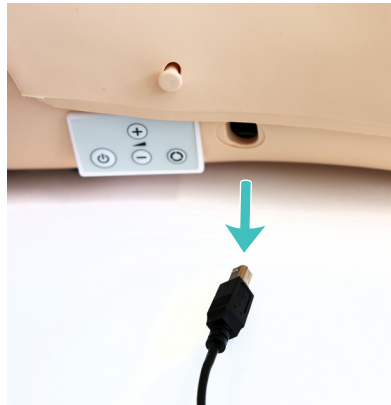

## Step 9

Power on manikin.



Power on the  
Resusci Anne QCPR.

## Step 10

Reconnect SimPad to manikin to  
validate the update.



SimPad and Resusci Anne QCPR are now both updated. Disconnect the cable and connect with Bluetooth to start training. Visit [www.laerdal.com](http://www.laerdal.com) to download SimPad User Guide for further information.

© 2017 Laerdal Medical AS.  
All rights reserved.

20-12758 Rev A

[www.laerdal.com](http://www.laerdal.com)



**Laerdal**  
helping save lives