EN

FR

DE

ES

IT

BR

NL

PL

JA

ZH

KO

# SimNewB

User Guide

<b>Introduction</b>	4
<b>System Overview &amp; Setup</b>	6
<b>Items Included &amp; Storage</b>	8
<b>Features</b>	9
<b>Power Panel</b>	16
<b>Charge</b>	20
<b>Connect</b>	22
<b>Overview Pulses and Sounds</b>	24
<b>Preparing for Simulation - Umbilical Venous Catheterization</b>	26
Prepare Reservoir for UVC	27
Large Volume Infusions	28
<b>Preparing for Simulation - IO Leg - Large Volume Infusions</b>	30
<b>Preparing for Simulation - Choose Pupils</b>	32
<b>Preparing for Simulation - Lubrication</b>	34
Lubricate Tools	34
Lubricate SimNewB	36
<b>Use - Intubation</b>	38
<b>Use - Ventilation</b>	40
<b>Use - CPR</b>	42
<b>Use - Unilateral Left Thoracentesis Midaxillary Access</b>	44
<b>Use - Umbilical Venous Catheterization</b>	46
<b>Use - IO Leg</b>	48
<b>Use - SimPad PLUS - Lung Recruitment Settings</b>	50
<b>Care and Cleaning</b>	52
General Care	52
Skin	52
Cleaning Umbilical Reservoir	58
<b>Maintenance</b>	62
Remove and Replace IO Leg Module	62

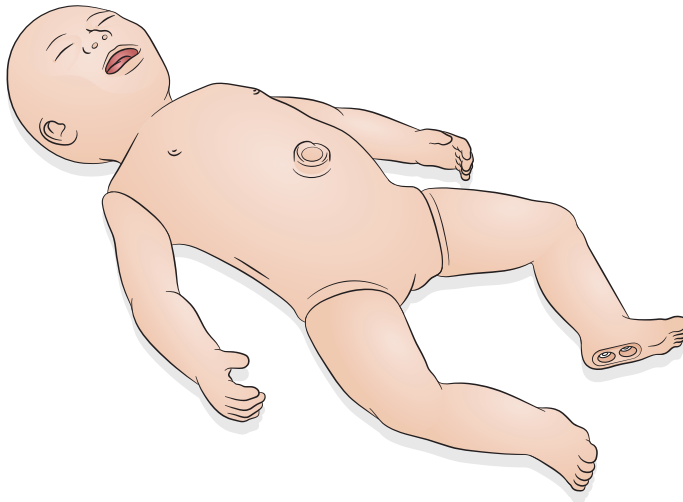
# Introduction

---

## EN

SimNewB is a tetherless newborn simulator designed to help improve neonatal resuscitation and to meet the specific learning objectives of neonatal resuscitation protocols. Focusing on the first 10 minutes of life, SimNewB provides realistic training for critical interventions.

Items included may vary in appearance and are subject to change.

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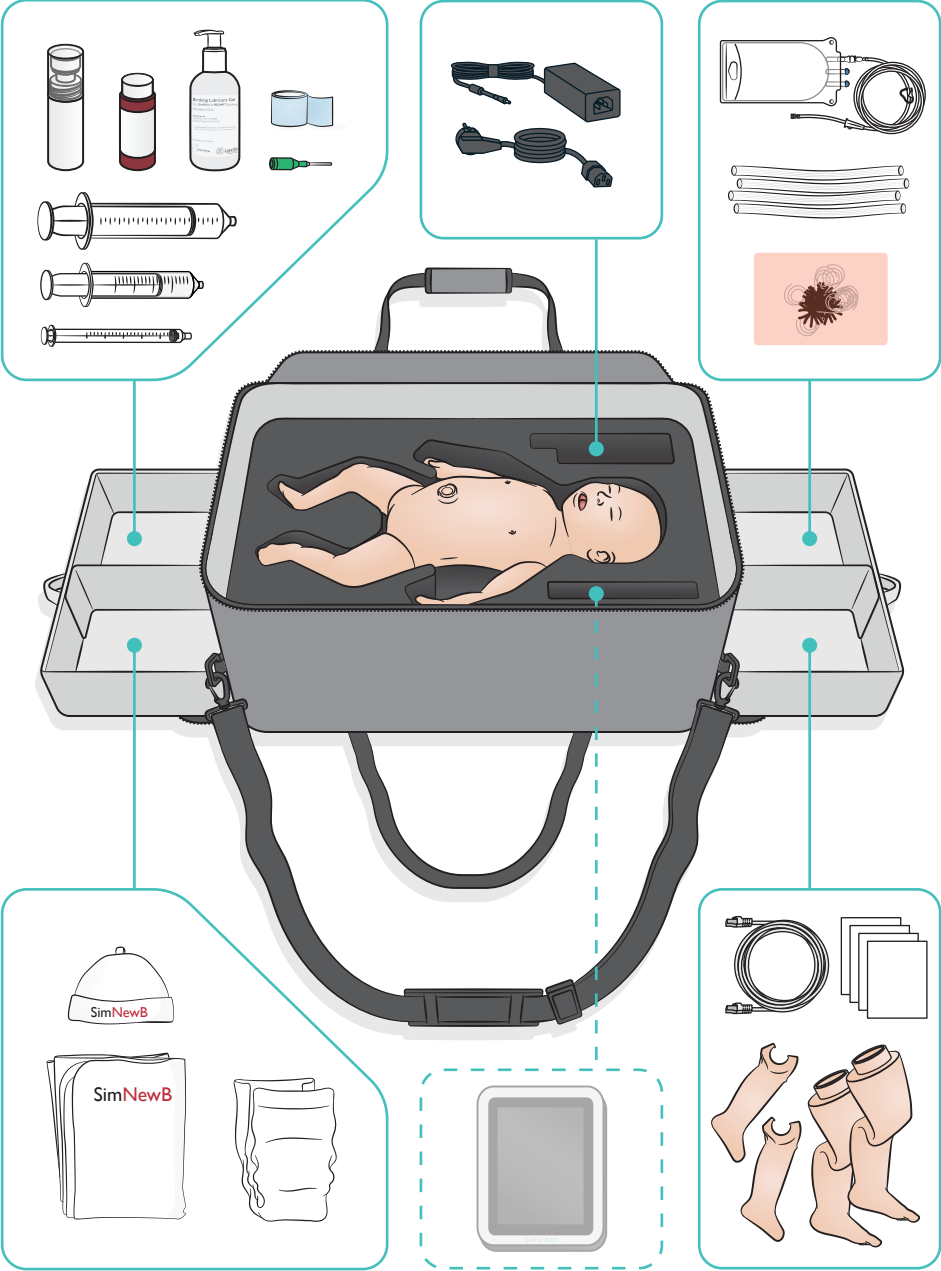
## EN

### Airway Features

- Anatomically realistic airway
- Nasal and oral ET tube insertion
- LMA and I-Gel insertion
- Sellick Maneuver
- Positive pressure ventilation
- Right mainstem intubation
- Suctioning
- Variable lung resistance
- Gastric tube insertion
- Stomach distention
- Lung Recruitment

### Breathing Features

- Spontaneous breathing
- Bilateral and unilateral chest rise and fall
- Normal and abnormal breath sounds
- Anterior lung sounds
- Unilateral breath sounds
- Oxygen saturation
- Unilateral thoracentesis

### Vascular Features

- Bilateral IO access
- Simulated umbilical cord
- Umbilical access
- Simulated blood flashback on cannulation

### Circulation Features

- Umbilical pulse
- Bilateral brachial pulse
- Central cyanosis

### Cardiac Features

- Basic CPR training

### Other Features

- Normal, blown and constricted pupils
- Vocal, lung and heart sounds
- Arms - movements and seizures

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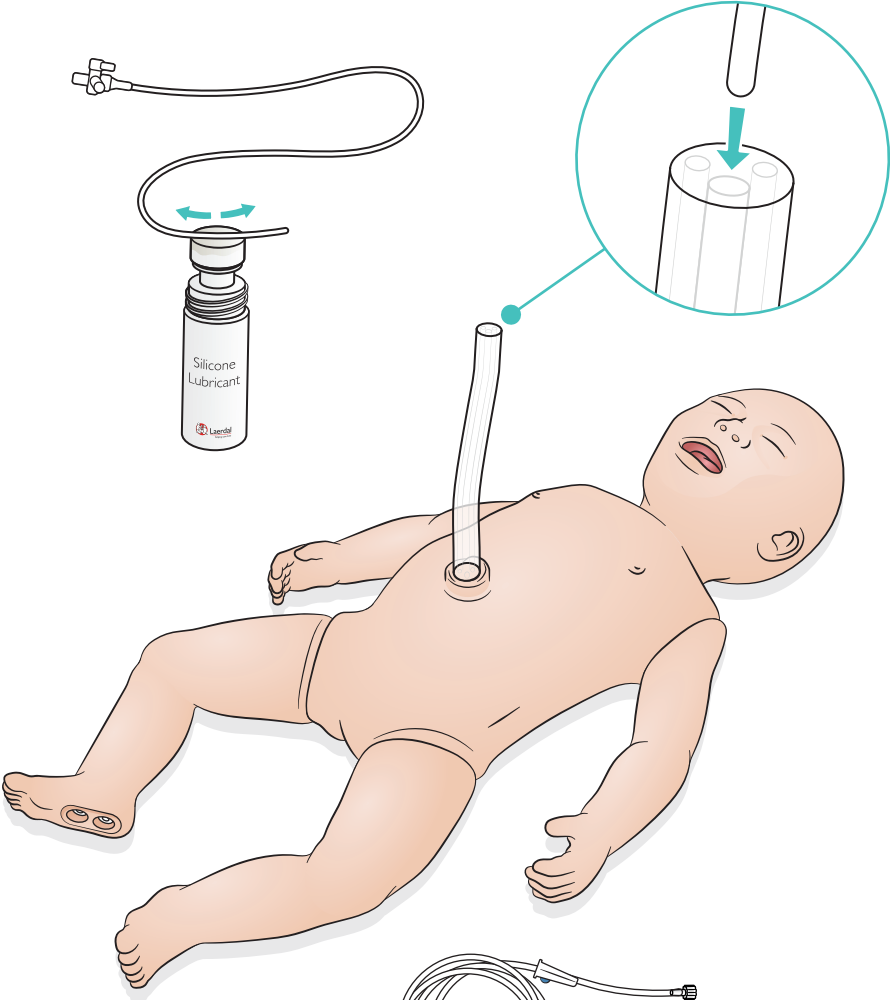
# Use - Umbilical Venous Catheterization

Simulated umbilical cord with venous and arterial access.  
Simulated blood flashback on cannulation.

For large volume infusions of fluid (exceeding 10 ml), connect empty drain bag to simulator.

# Use - Umbilical Venous Catheterization

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## FR

### Umbilical Venous Catheterization

- Simulated umbilical cord with venous and arterial access. Simulated blood flashback on cannulation.
- For large volume infusions of fluid (exceeding 10 ml), connect empty drain bag to simulator.

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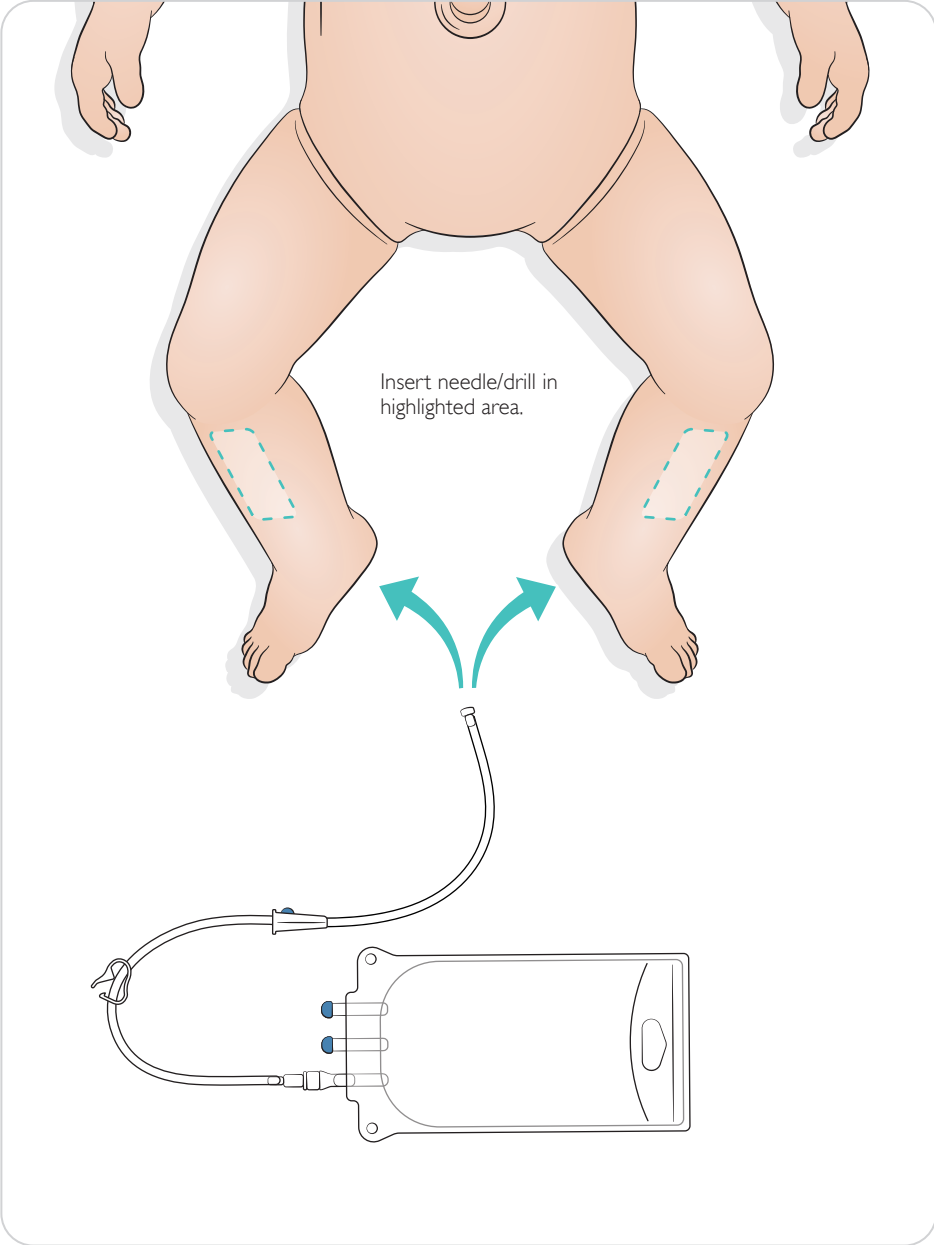
# Use - IO Leg

Note

The IO Leg should be replaced after each simulation when using fluids.

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# Use - SimPad PLUS - Lung Recruitment Settings

Activate Lung Recruitment in Manual Mode before starting scenario.

The image shows three sequential screenshots of the SimPad PLUS interface. The top screenshot shows the main dashboard with vital signs: Sinus (ECG), HR 140, SpO<sub>2</sub> 98, RR 40, etCO<sub>2</sub> 34, BP 60/40 (46), and Temp. 37.2 (36.1). The middle screenshot is the 'Lung Recruitment' settings screen, showing 'Number of Effective Ventilations' with options 'None', '4', and '7'. The '4' option is selected. At the bottom, there are 'CREATE New State' and 'ACTIVATE ✓' buttons. The bottom screenshot shows the 'Ongoing care' section with 'Lung Recruitment Ongoing' and '4 Ventilations' displayed. A 'START' button is visible at the bottom of this screen. Red arrows and numbers 1, 2, and 3 point to the '4' option, the 'ACTIVATE' button, and the 'START' button respectively.

1. Select ventilation number

2. Press

3. Start

# Use - SimPad PLUS - Lung Recruitment Settings

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EN

## General Care

- To maintain simulator skins, wash hands before use and place the simulator on a clean surface.
- Use gloves during simulation scenarios. Avoid using colored plastic gloves, as they may cause discoloration of the simulator skin.
- If a training session involves the use of fluids in the Umbilical Reservoir and IO Leg, drain the fluid immediately after the training session.

## Skin

- Use a lint free cloth to remove dirt and dust.
- Clean skin with mild soap and water. Do not submerge.
- To remove birthing lubricant, use lukewarm soapy water.

### Notes

*These can discolor the manikin:*

- *Pigments from lipstick and pens*
- *Latex gloves*
- *Using clothes other than what is originally provided with the simulator.*

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© 2018 Laerdal Medical AS. All rights reserved.