Practoplast

| | |
|----------|--------------------|
| ENGLISH | Directions for Use |
| FRANÇAIS | Mode d'emploi |
| DEUTSCH | Gebrauchsanweisung |
| NORSK | Bruksanvisning |

Directions for Use

Practoplast imitation wounds are fastened to the skin with the enclosed special Latex adhesive in the following manner:

1. Place a few drops of adhesive on the imitation wounds and spread it thinly and evenly over the surface with the plastic spatula.
2. In the same manner, apply adhesive to the skin where the imitation wound is to be attached.
The glued surface of the skin should be larger than the plastic wound.
3. Let adhesive dry until clear. Then place the Practoplast wound on the skin. Wound will now adhere to the skin.

The Practoplast wounds can easily be removed from the skin, and the adhesive can be easily pulled or wiped off. If adhesive is covering hairy portion of skin, moisten with alcohol, an scrub lightly with a brush.

During extensive training exercises, it is practical to apply adhesive to the imitation wounds in advance, thus saving time in drying.

To simulate bleeding:

1. Fasten plastic blood droplets to the imitation wound and skin, using the Latex adhesive.
2. Liquid blood: Concentrated artificial blood from tube is diluted in water. Use bottles with coneshaped spouts.

Mixture:

Light arterial blood – 1 part blood concentrate to 10 parts water
Dark venous blood – 1 part blood concentrate to 2 parts water

The coneshaped spout of the blood bottles makes it easy to apply blood for the wounded area.

Bleeding from artery or veins is simulated by using the tube and plastic bulb found with the Practoplast Kit. The bulb is filled as follows:

Squeeze the bulb firmly and insert it on the spout of the blood bottle. When the bottle is squeezed, the bulb will fill with blood.

Use of Practoplast wounds in training

In order that first aid instruction be fully effective, it is necessary that the students be able to practice their first aid knowledge in realistically simulated situations. By using Practoplast imitation wounds during training, a student will not only learn to identify various traumatic injuries, but will also become accustomed to treating bleeding wounds, open fractures, burns etc. The student will also receive the mental training which is necessary for him to act in a clear-thinking manner when an emergency arises. It is impossible to simulate a fully realistic situation by using the Practoplast imitation wounds and make up alone. The "injured person" must also be able to behave in the same manner as if actually wounded.

The instructor:

- must instruct the "injured person" on how he should act and react due to his injury. If the instructor is uncertain as to how the injured person should behave, he should consult a doctor, or some other experienced person.
- The "injured person" must be told what kind, and how extensive his injury is, and when, where, and how, the injury occurred, so that he can reply correctly when first aid personnel asks for details.

Example: Imitation No. 9 – "Compound fracture of humerus". The "injured person" informs the first aid personnel that he slipped and, while reaching out to support himself, fell on his arm. The accident occurred at 4 pm on the corner of Elm and Main Street.

If the training exercise is arranged so that the accident occurs in the presence of an audience, the "injured person" must be able to act and react in a realistic manner.

- Example: The "injured person" comes on the scene, the imitation wound fastened under his jacket sleeve, he slips, falls to the ground, groans and states that he has heard a bone break.

The "injured person" should know, and be able to simulate the extent of his pain. Are the pains weak, strong, burning, etc.?

With certain injuries the "injured person" is to be unconscious. He must be informed when and how he is to regain consciousness.

Some injuries, particularly those caused by serious accidents, may cause hysteria and other type of psychic trauma, in the "injured person". This should be simulated in a realistic manner, and care should be taken that it does not appear comical or unrealistic.

The "injured person" must be trained how to behave and react when first aid treatment is given. For instance, in case of careless treatment, he must express discomfort and great pain. In the same way, he must react psychologically (apprehensive and afraid) if the first aid personnel appear to be inexperienced or uncertain, as to what type of treatment should be given.

As previously mentioned, the dramatic ability of the "injured person" and his behaviour under a traumatic situation, will to a great extent decide whether or not the training session will be a success. The "injured person" should at all times, act and react in the same manner as he would in a true-to-life situation.

For people who have not had experience in the treating on-the-scene wounds and injuries, it is not easy to forecast how an injured person will behave in various circumstances. On the following pages, you will find a guide as to how an "injured person" should behave under various traumatic situations.

Training Examples

(Suggestions how to place the imitation wounds in parenthesis)

Tray No. 1**Fractures and special injuries**

1. **Closed fracture of mandible** (covering one jaw)
The "injured person" indicates pain by moaning. He has difficulty in talking clearly but is able to make it understood that the injury was caused by a blow to the jaw. Casually he supports the jaw with one hand. He indicates that pain has been alleviated, and he feels more comfortable after receiving a good supporting bandage.
2. **Closed fracture of orbit – with hemotoma** (over both eyes)
The "injured person" appears unconscious during whole procedure. He should be made up as a patient suffering from shock.
3. **Compound fracture of nasal bones** (on the nose)
The "injured person" expresses great pain by groaning, etc., but manages to indicate how the accident occurred, i.e. a heavy plank fell and hit him on the nose. He should be made up rather pale, and a great deal of blood should be flowing from the wound and the nostrils.
4. **Compound fracture of the ribs** (on either side of the chest at rib cage)
The "injured person" sits half upright, leaning on his elbow holding his hand over the wound. At the same time, his other hand is in his pocket operating the air pump. He indicates great pain by groaning and breathing quickly. (He must remember to use the air pump to simulate air coming from the wound)
He coughs up blood and speaks in whisper. He indicates how the accident occurred, i.e. from a gunshot accident and gives the approximate time and place. After correct first aid is given (the application of an air-tight dressing covering the wound) the symptoms cease.
5. **Incised wound of abdominal wall with protrusion of small intestine** (on side of the stomach)
The "injured person" indicates pain by groaning, and clutches at the wound, complains of thirst. He thinks the injury was caused by being hit by a piece of flying metal, i.e. from a workshop explosion, machinery malfunction etc. He also gives the time and place where the injury occurred. After correct treatment (application of sterilized bandage over the wound, the intestines supported in place before bandaging), he is placed with his knees well pulled up to his chest, and his lips are slightly moistened (no liquids should be given), the pain subsides. He should be made up rather pale, and a great deal of blood should surround the wound.
6. **Wound of orbit with evulsion of eye** (covering the left eye)
The "injured person" should appear apprehensive and frightened and indicate great pain by moaning and groaning. His hand should apply great pressure over the injured eye and he will speak incoherently of the accident. After treatment, he will quiet down and appear less frightened. He should be made up with pale make up and a great deal of blood should run from the injury.

Tray No. 2**Fractures, wounds and bites**

8. **Compound fracture of fibula** (on the shin bone)
The "injured person" twists his foot to indicate the shin bone is non aligned. He tells the first aider the time and place the injury occurred. He complains of great pain when the first aider moves the fractured leg. After correct treatment has been applied and the leg is well supported, he calms down and thanks the first aider for the good treatment.
9. **Compound fracture of humerus** (on upper arm)
The "injured person" holds the arm in area of fracture and indicates great pain when the first aider moves the arm. He indicates time and place and how injury occurred, i.e. he slipped and fell and reached out for support. After correct treatment is given and arm is bandaged and put into a splint, he indicates that pain has subsided.
10. **Dog bite** (upper side of upper arm)
The "injured person" indicates pain by moaning, and at the same time indicates his anger at the dog that bit him. He is also apprehensive and frightened about the possibilities of rabies. He calms down after being bandaged, and after being assured that rabies infection is very rare in this country.
Note:
The imitation wound can also be placed on the leg in the area of the calf or on the hand between the thumb and forefinger.
11. **Incised wound with glass fragment** (on the forehead, or the palm of the hand, or on side of neck)
The "injured person" indicates that a pane of glass fell near him and injury occurred when the splinters scattered around him.
12. **Incised wound with large metal fragment** (on outside of upper arm, or on the thigh, or on the scapula – back of shoulder)
The "injured person" indicates pain, particularly when being moved.
13. **Snake bite** (on the back of the ankle, or on back of hand, or on outside of foot)
The "injured person" appears frightened and apprehensive about being bit by a possibly poisonous snake. After the tourniquet is applied, immediately above the bite, two X-shaped incisions are made over each fang mark and venom is drained out of the wound by use of a suction cup. Afterwards, a dressing is applied and the wound is bandaged.
14. **Incised wound** (on the cheek, the stomach or bottom of foot)
The "injured person" puts his hand over the wounded area. The injury may have been received by either a knife wound, or in the case of a foot injury, stepping on a nail. If wound is placed on cheek, the "injured person" should be given a blood capsule so that he can spit blood.

Tray No. 3

Fractures and injuries

15. Closed fracture of fibula (on the shin bone)

The "injured person" should twist his foot to indicate dislocation or non-alignment of the leg bone. He indicates great pain and says that he felt the bone fracture. He also gives the time and place the injury occurred. He indicates great pain when the first aider moves the limb and when treatment is given. When the leg is well supported he indicates that pain has subsided.

16. Closed fracture of either radius or ulna (on the wrist)

The "injured person" will try to support the fractured wrist with one hand and insist that he has a broken bone and not simply a sprain. He also gives the time and place the injury occurred.

17. Compound fracture of fingers (on the middle finger)

The "injured person" supports the injured finger and states that a heavy object fell over his hand. He indicates pain when first aider carelessly moves the finger when being treated. A great deal of blood should surround the wound.

18. Contused wound of nail bed (on the index finger)

The "injured person" should indicate by moaning that he feels a great deal of pain. He tells the first aider how the injury occurred, i.e. the nail was torn off while he was working.

19. Compound fracture of toes with contusions (on the toes)

The "injured person" moans and indicates fear of what will happen to his foot. He explains how the accident occurred, i.e. an axe slipped during chopping wood and hit his foot. A great deal of blood should surround the wound and a partially severed sock and boot may also be used to give a realistic effect to the injury.

20. Frost bite of toes (on the toes)

The "injured person" should be found under specific circumstances where an injury such as this can occur. For example, well wrapped up underneath an inoperative truck or automobile. The "injured person" indicates concern as to what will happen to his foot, and a stretcher team will come to carry him away.

Tray No. 4

Burn injuries and hemorrhages

21. Third degree burn, charring of tissue (on front of thigh just above the knee, or high on the hip, or on small of back)

The "injured person" indicates great pain by moaning and explaining how injury occurred, i.e. trying to start a charcoal fire. After a sterilized dressing and bandages are applied, he appears much calmer but still complains of the pain. He should be made up with pale make up.

22. Second degree burn, with blistering (on the outside of wrist – back of hand, or on front of shoulder, or on outside of upper arm)

The "injured person" indicates that the injury is causing great pain. He tells the rescuer that the injury occurred when he was scalded by hot oil or that he fell against a hot pipe, or hot stove. The pain is reduced after the application of a dry, sterilized dressing and bandage.

23. First degree burn, with erythema (on outside of upper arm, or immediately above the knee, or on front of ankle – side of foot)
The "injured person" indicates that injury is painful and informs the rescuer how the injury occurred, i.e. accidentally touched a hot-water heater, or scalding water was accidentally spilled on him.

24. Burn due to gamma ray exposure (on back of left hand)

The "injured person" indicates great worry and excitement. He indicates that he threw himself down and managed to cover all of his body, except his left hand. The direction of radiation from the atomic explosion caused the burn on the left hand. He calms down after an application of a dry sterilized dressing and bandage.

25. Incised wound with arterial bleeding (on the side of the wrist, or on the side of the neck, or behind the knee)

The "injured person" is quite afraid and apprehensive as to what will happen to him. He must remember to press the bulb in order to simulate the spurting blood from the pulsating artery. The rubber tube from the bulb should be concealed under the shirt sleeve and lead down behind the neck to the opposite trouser pocket. The pump should be filled with light-colored blood. Upon questioning, he explained how the injury occurred, i.e. a knife slipped and cut him, or he was hit by a piece of shrapnel. If the wound is to be placed on the side of the throat, the "injured person" indicates difficulty in breathing. The "injured person" calms down after a compress dressing is applied. After dressing is applied, he can stop the pumping.

26. Wound demonstrating venous bleeding (on the side of the under arm, or on the side of the thigh, or on inside of ankle, behind the ankle bone)

The "injured person" does not indicate too much pain; however, is apprehensive because of the heavy bleeding. He tells the rescuer how the accident occurred, i.e. either by a knife slipping and cutting him, or by being stuck with a barbed wire fence or nail. He must remember to use the blood pump in order to produce an even stream of blood coming from the wound. The pump and rubber tubing is concealed in the same manner as in No. 25. The pump is filled with dark-colored blood. He calms down after a proper compress has been applied. The pump and rubber tubing is concealed in the same manner as in No. 25.

Tray No. 5

Miscellaneous wounds

27. Laceration

The "injured person" complains of pain and explains that the injury was caused by a factory explosion and is anxious about the safety of his co-workers. He calms down when a good bandage is applied and upon hearing that his co-workers are in good health.

28. Contused wound (on the shoulder blade, on the shin or on the hip)

The "injured person" expresses great pain and explains that he fell from a tree onto hard, rocky ground.

29. Superficial abrasions (just below the elbow, on the palm of the hand or on the knee)

The "injured person" explains that he has sustained his injuries after a fall from a bicycle.

30. Incised wound (on the inside of the arm, on the upper arm, or on the stomach)
The "injured person" complains of pain. He explains the cause of the injury i.e. a knife or shrapnel. He calms down after a good bandage is applied.

31-32. Gun shot wounds – entrance and exit (entrance through left jaw, exit through right cheek, or entrance through left shoulder, exit through lower part of the back, or entrance through outside of leg, exit through inside)
The "injured person" is apprehensive and in pain and has difficulty in talking clearly. If wound is placed on front of shoulder the "injured person" has difficulty in breathing, and also coughs up some blood. If the wound is placed on the leg, he clutches the wound with his hand but appears cheerful, since he is relieved that the wound is not more serious.

33. Deeply lacerated wound (on the thigh or on the back of upper arm or on the back of leg)
The "injured person" indicates pain by moaning and is quite apprehensive. He informs the rescuer how the injury occurred, i.e. that an industrial explosion took place and something hit him. The accident occurred so quickly that he cannot give any other definite details. If any of his friends were also in the area of the explosion, he anxiously asks about their welfare. He calms down upon hearing that they are not harmed and after a good bandage and dressing has been applied (the application of a splint may be needed).

CONTENTS

Tray No. 1

Fractures and special injuries

1. Closed fracture of mandible
2. Closed fracture of orbit with hemotoma
3. Compound fracture of nasal bones
4. Compound fracture of ribs
5. Incised wound of abdominal wall with protrusion of small intestine
6. Wound of orbit with evulsion of eye

Tray No. 2

Fractures, wounds and bites

8. Compound fracture of fibula
9. Compound fracture of humerus
10. Dog bite
11. Incised wound with glass fragment
12. Incised wound with large metal fragment
13. Snake bite
14. Incised wound

Tray No. 3

Fractures and injuries

15. Closed fracture of fibula
16. Closed fracture of radius
17. Compound fracture of fingers
18. Contused wound of nail bed
19. Compound fracture of toes with contusion
20. Frost bite of toes

Tray No. 4

Burn injuries and hemorrhages

21. Third degree burn, charring of tissue
22. Second degree, with blistering
23. First degree, with erythema
24. Burn due to gamma ray exposure
25. Incised wound demonstrating arterial bleeding
26. Wound demonstrating venous bleeding

Tray No. 5

Miscellaneous wounds

27. Laceration
28. Contusion
29. Superficial abrasions
30. Incised wound
- 31/32. Gun shot, entrance and exit
33. Deep laceration

Inside compartment:

| | |
|---------------------|--|
| Plastic blood drops | to be glued on |
| Blood | 1 tube, 100 grams concentrated artificial blood to be diluted in water |

Mixture:

| | |
|----------------------|--------------------------------------|
| Light arterial blood | 1 part concentrate to 10 parts water |
| Dark venous blood | 1 part concentrate to 2 parts water |

| | |
|------------------|-----------------------------------|
| Blood | 2 empty 100 ml bottles with spout |
| Plastic tube | 2 x 1.4 meters |
| Special adhesive | 1 bottle 100 ml |
| Spatula | 1 |
| Marking pencils | 3 (red, blue and black) |
| Masking ointment | 1 box |
| Plastic bulbs | 2, to simulate bleeding |
| Gauze compress | 2 |

On the inside of the lid is a sheet containing colored illustrations of the wounds and a numbered index.

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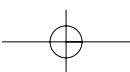
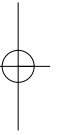
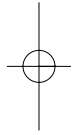
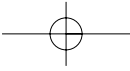
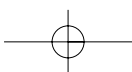
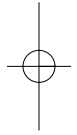
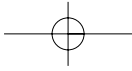
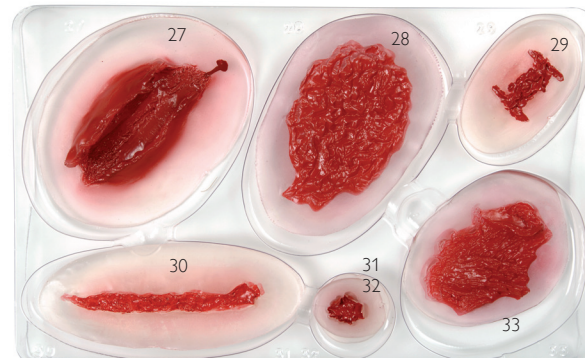
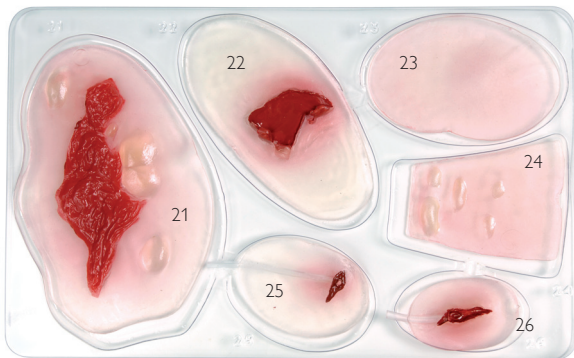
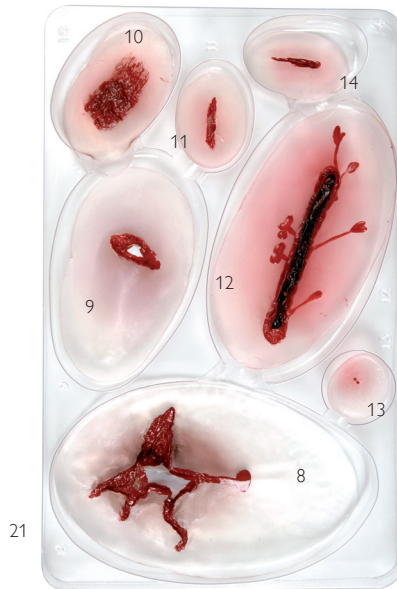
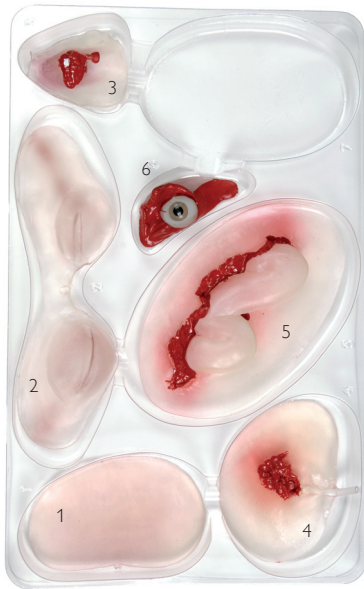
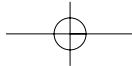
- 1 Closed fracture mandible
- 2 Closed fracture, orbit
- 3 Compound fracture, nose
- 4 Compound fracture, ribs
- 5 Incised wound, abdomen
- 6 Wound, orbit w/evulsion

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- 8 Comp. fracture, fibula
- 9 Comp. fracture, humerus
- 10 Dog bite
- 11 Incised wound (glass)
- 12 Incised wound (metal)
- 13 Snake bite
- 14 Incised wound

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- 15 Closed fracture, fibula
- 16 Closed fracture, radius
- 17 Comp. fracture, fingers
- 18 Contused wound, nail bed
- 19 Compound fracture, toes
- 20 Frost bite, toes

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- 21 Burn, grade 1, 2 and 3
- 22 Burn, grade 1 and 2
- 23 Burn, grade 1
- 24 Burn due to gamma ray
- 25 Wound, arterial bleeding
- 26 Wound, venous bleeding

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- 27 Laceration
- 28 Contusion
- 29 Superficial abrasions
- 30 Incised wound
- 31 Gun shot, entrance
- 32 Gun shot, exit
- 33 Deep laceration

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