


PreemieNatalie

Preterm Simulator

User Guide

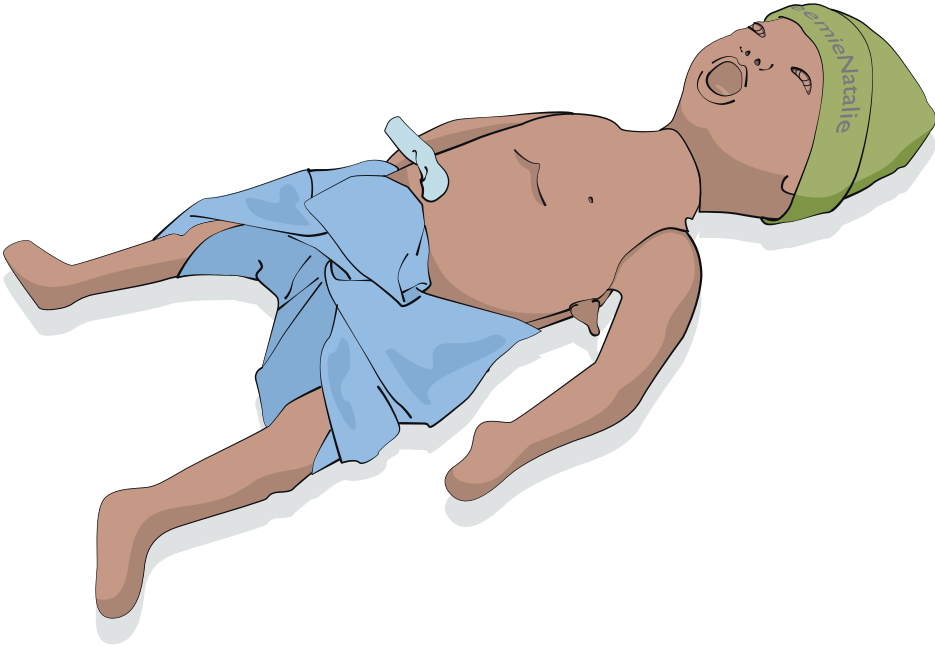

PreemieNatalie is a Preterm Newborn which has a realistic size and appearance of a preterm baby simulator (1.6 kg, 32 week gestational age).


Supports training in:

- Nasogastric placement
- Nasogastric tube feeding
- Breastfeeding positioning and attachment

PreemieNatalie can be used together with:

- MamaBreast Breastfeeding Simulator to practice initiation of breastfeeding
- CarePlus Wrap to practice Kangaroo Mother Care
- MamaNatalie Birthing Simulator to practice training of delivery and essential care for mother and premature babies.


Items Included	4
Setup	5
Before Each Use	6
Nasogastric Tube Placement	7
Nasogastric Feeding	10
After Each Use	11
Storage	12
Cleaning	13
Specifications	14

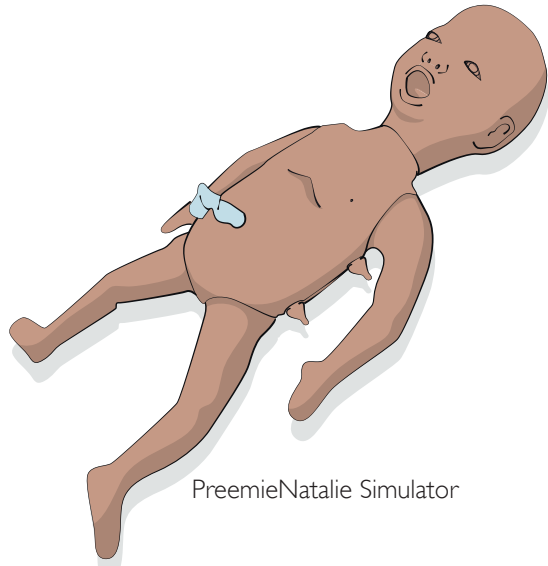

PremieNatalie Simulator




Hat

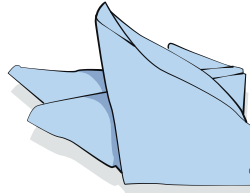


Diaper




Training Stethoscope

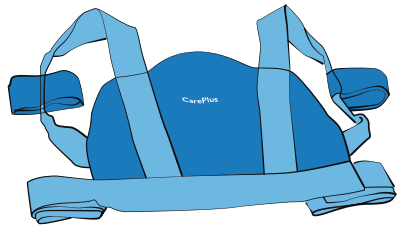


CarePlus Wrap

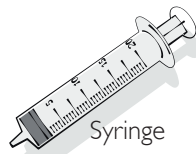

Syringe

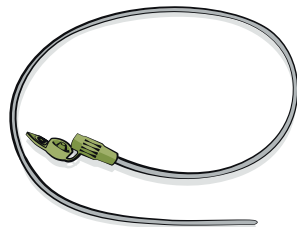

Nasogastric Tube (size 6)




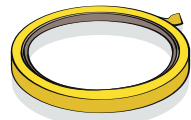


Tape




Storage Bag


Filling PremieNatalie

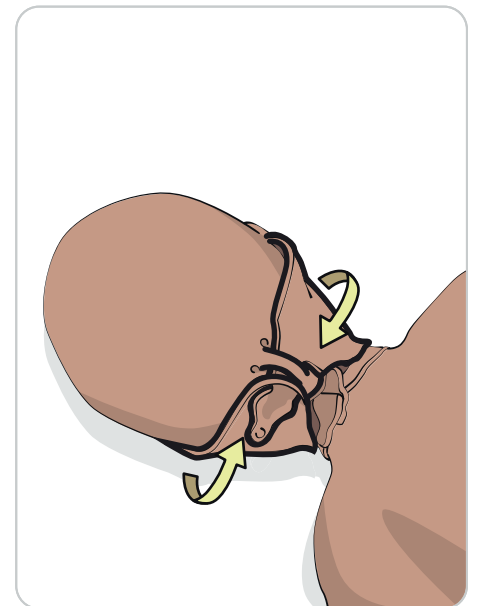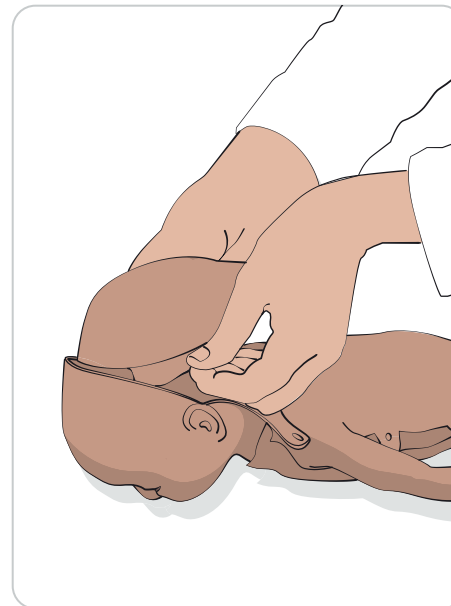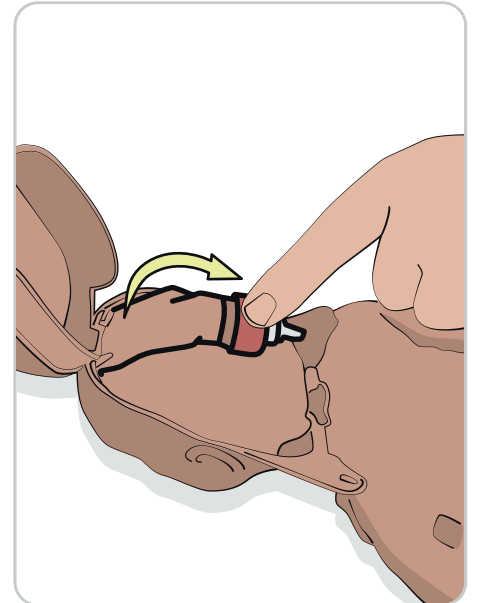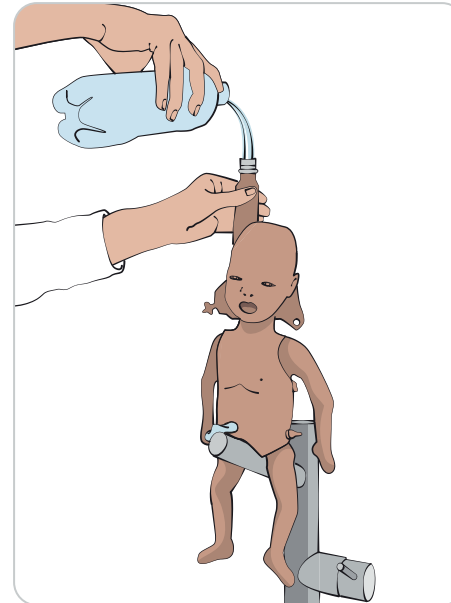


Weigh PreemieNatalie

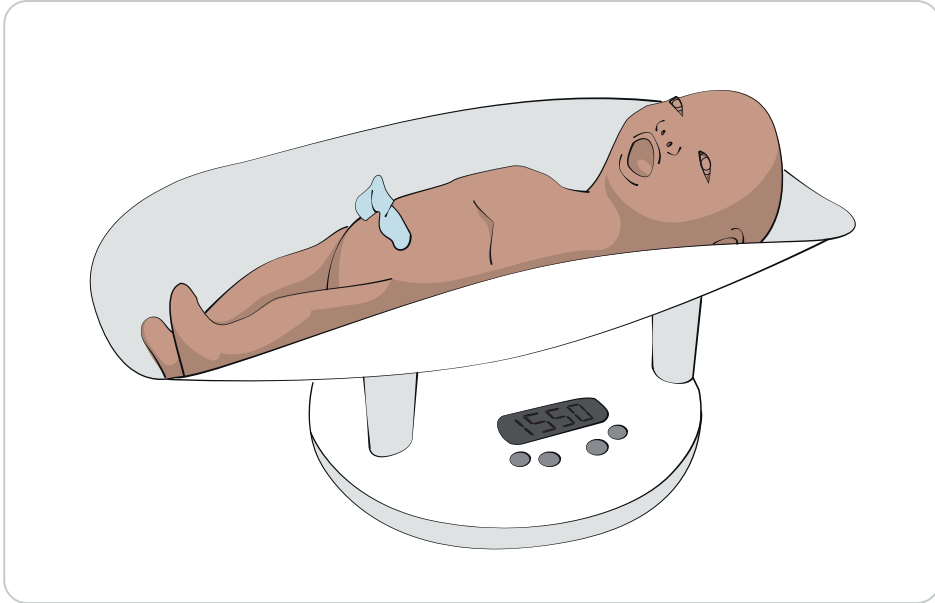


Fill Stomach to Prepare for Training

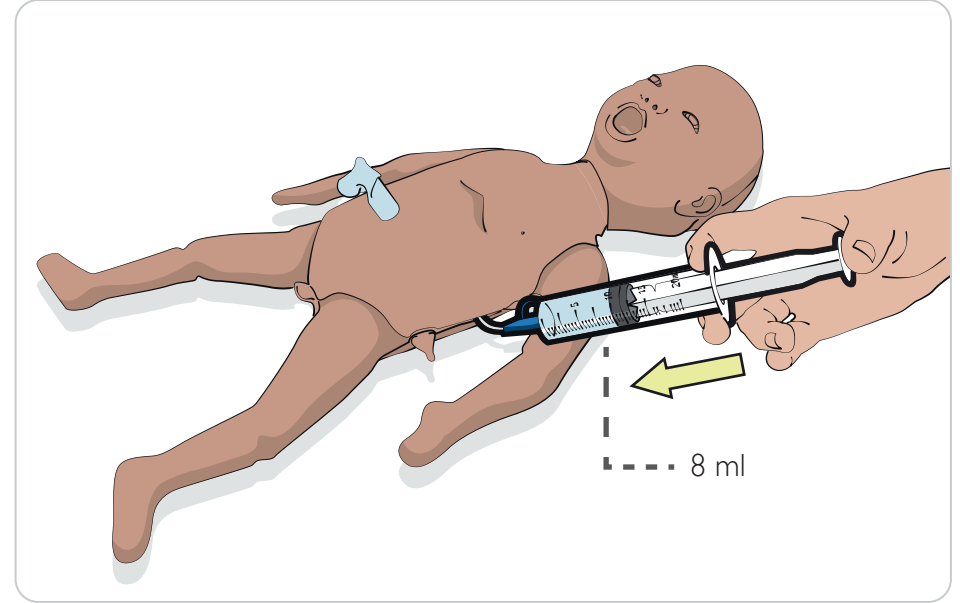



Empty Stomach

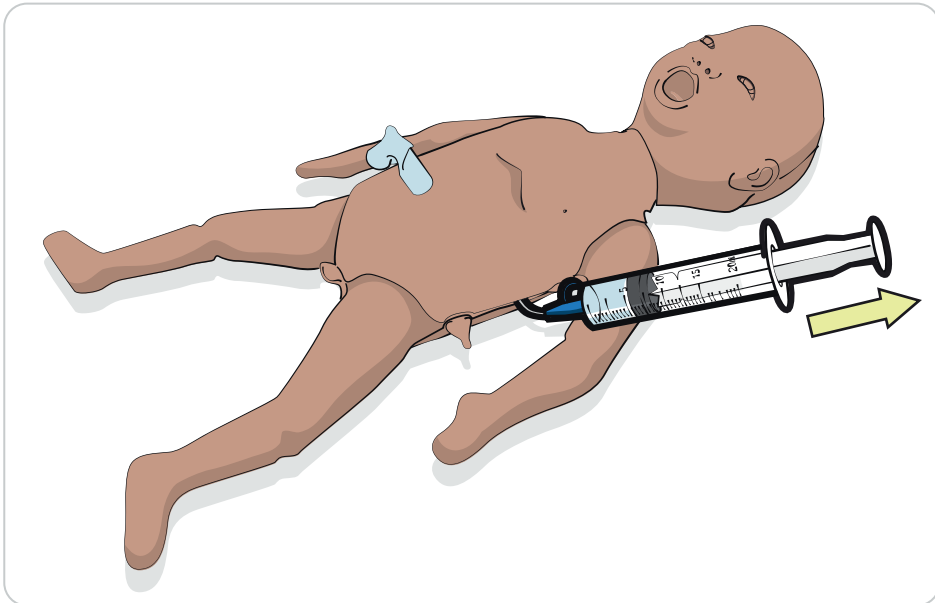



Measure Length for Correct Placement


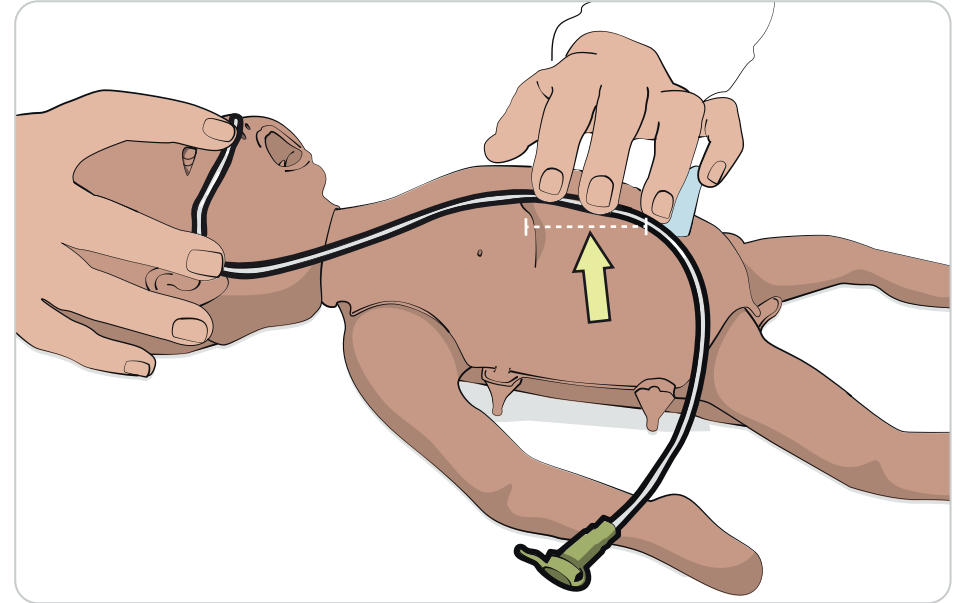


Insert the Nasogastric Tube through the nose or mouth

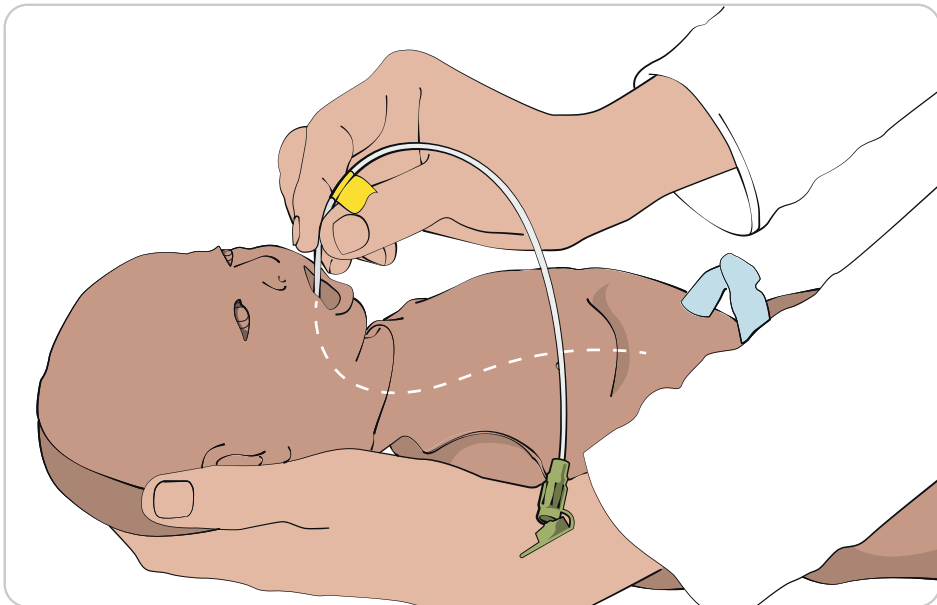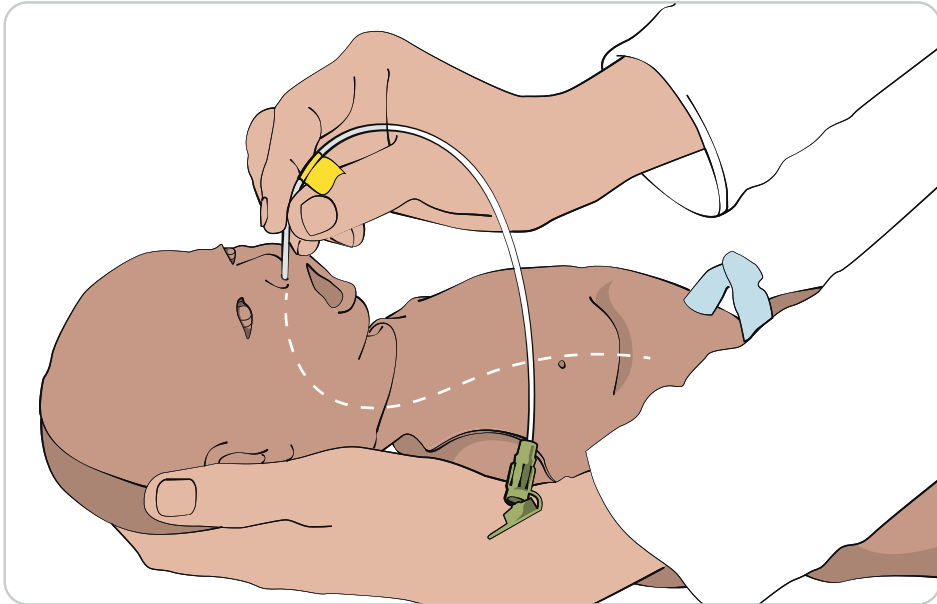


Check for stomach sounds or fluid to show correct placement

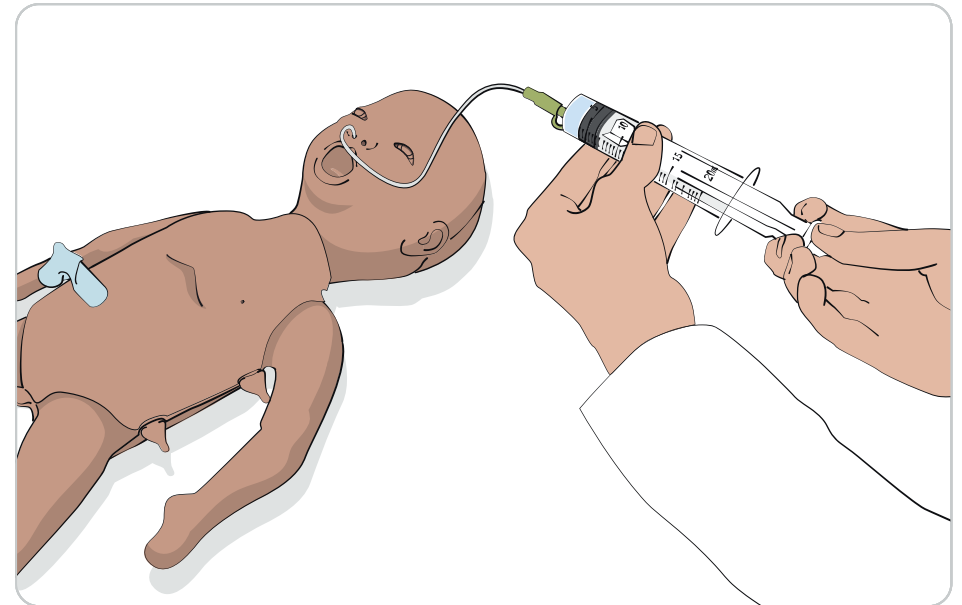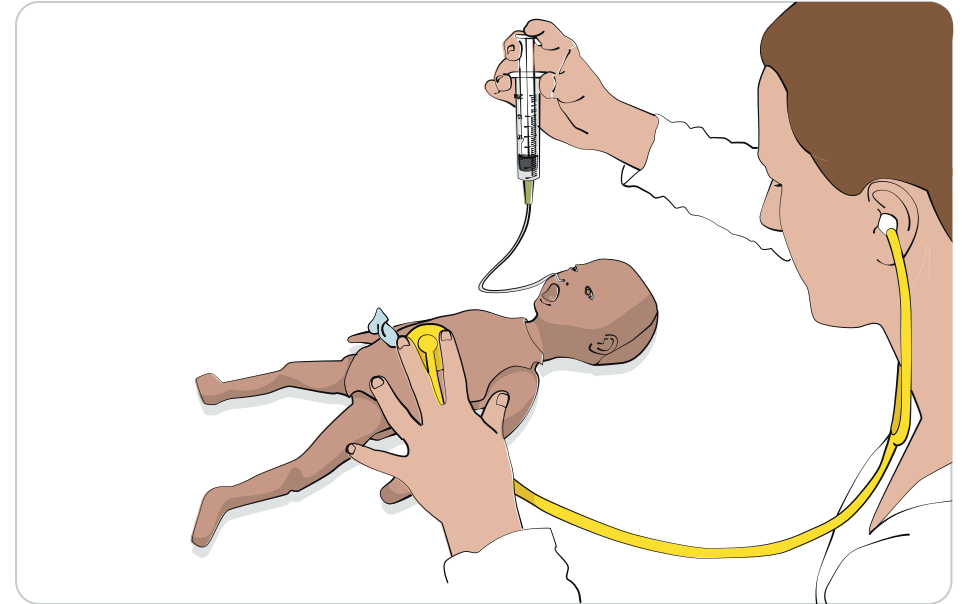


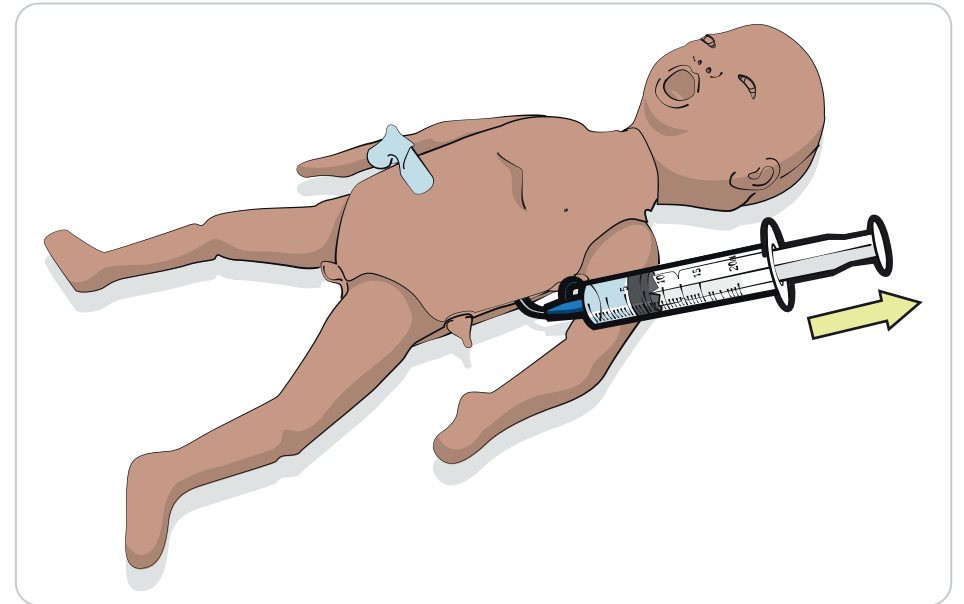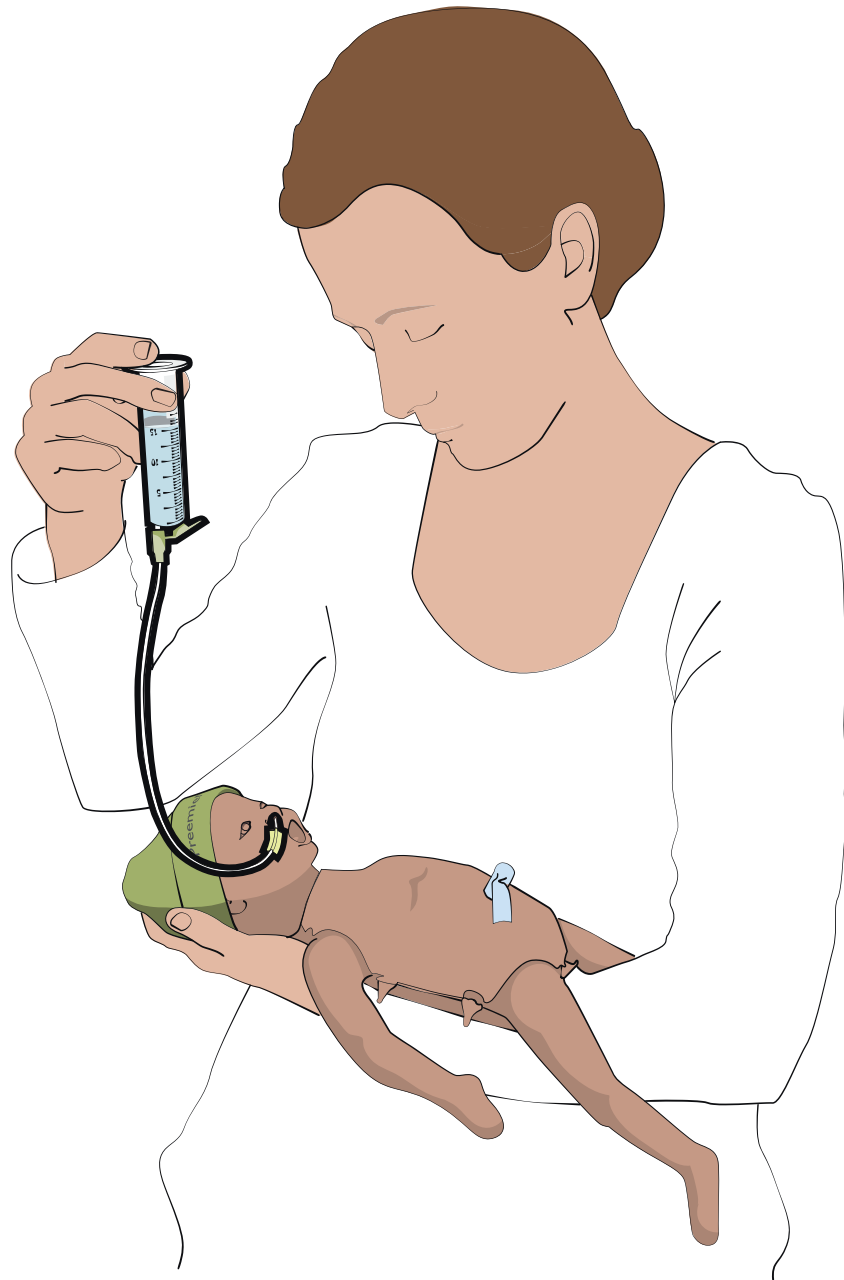

Before Storage

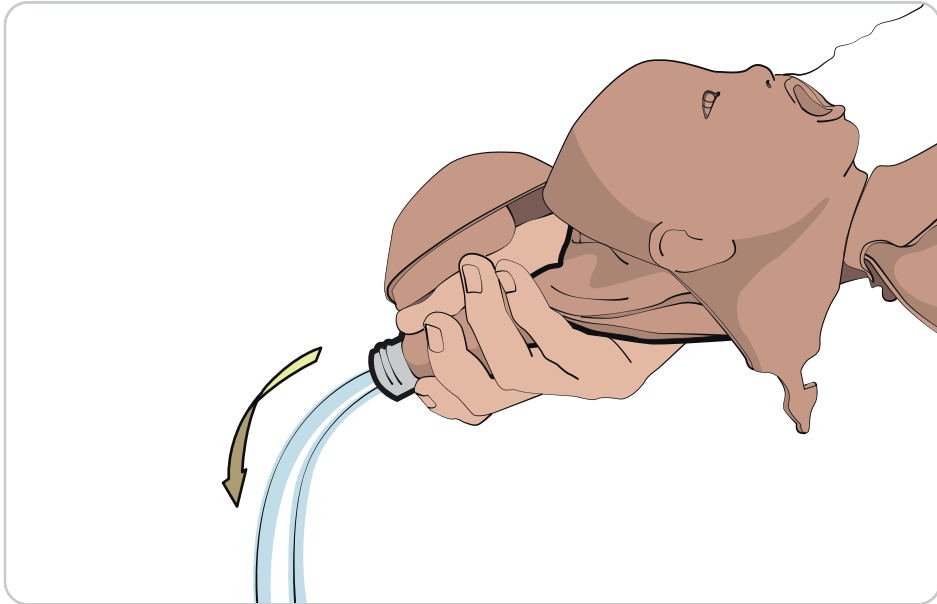

Packaging

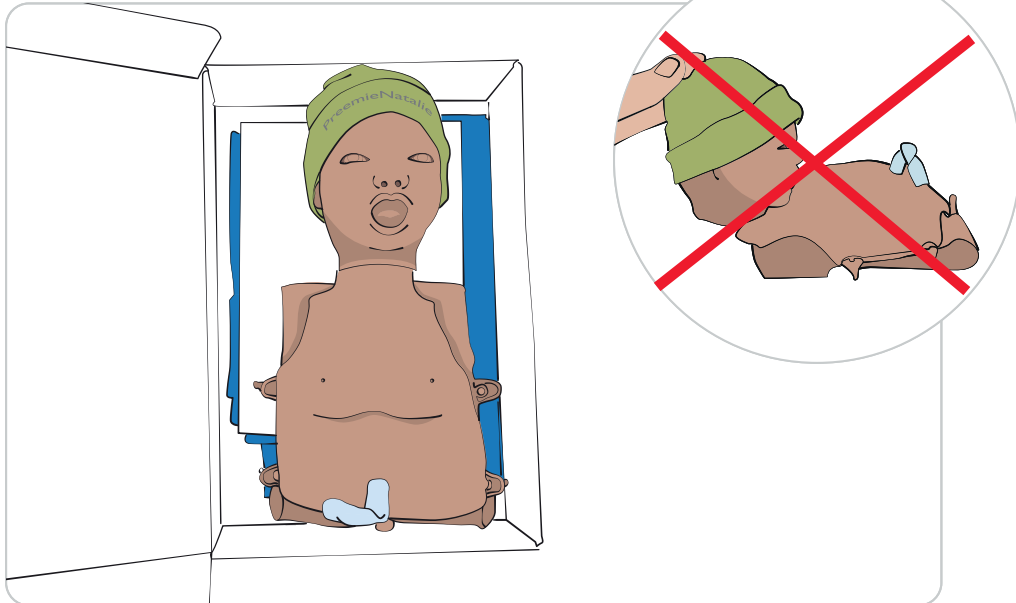

Clean PremieNatalie with a damp soft cloth.

When required, the chest skin can be lifted. Use a damp soft cloth to wipe clean.

Specifications

Operating conditions	
Relative Humidity	5-100% RH
Temperature	15 °C to 40 °C (59 °F to 104 °F)
Storage conditions	
Temperature (unfilled)	-10 °C to 50 °C (14 °F to 122 °F)
PremieNatalie Simulator	
Weight	1550g (filled with water)
Dimensions	43 cm

© 2015 Laerdal Medical AS. All rights reserved.
Laerdal® is a registered trademark of Laerdal Medical AS.

Manufactured in China for: Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway
T: (+47) 51 51 17 00

Printed in China.

20-08023 Rev A

www.laerdalglobalhealth.com



Laerdal
helping save lives