Intended Use

EN

The CPRmeter 2 Silicone Cover is intended to be used with the CPRmeter 2. It provides a larger patient contact area during use and does not require a patient adhesive.

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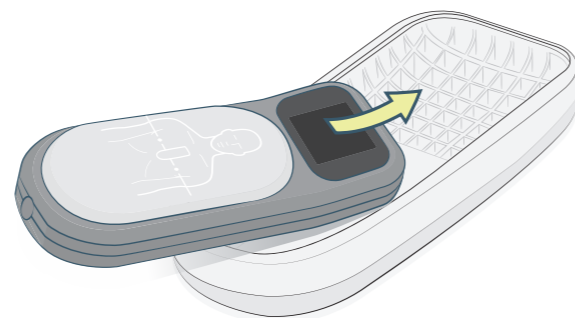
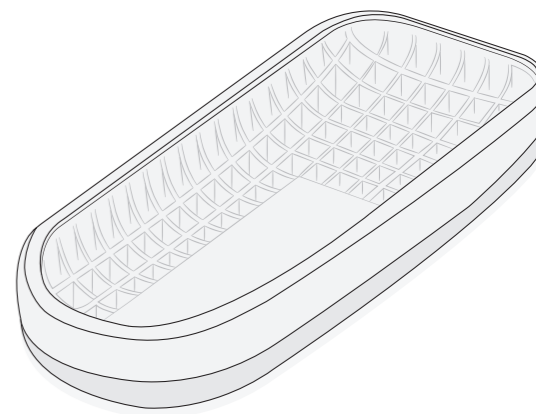
Laerdal® is a registered trademark of Laerdal Medical AS.
© 2020 Laerdal Medical AS. All rights reserved.

Manufactured in China by Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30,
4002 Stavanger, Norway
T: (+47) 51511700

Printed in China

20-14242 Rev D

Cat. no. 801-10662

EN
FR
ES
JA

CPRmeter 2 Silicone Cover

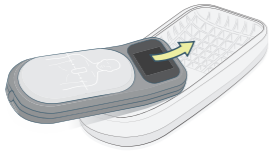
Patient >8 years

CPRmeter 2 Silicone Cover - Patient >8 years

Rx ONLY	Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Not for use on children under 8 years old
CE	This product is in compliance with the essential requirements of EU Council directive 93/42/EEC as amended by EU Council directive 2007/47/EC.

Applying the Silicone Cover

1. Check that the silicone cover has been cleaned and disinfected after last use.
2. Insert the CPRmeter 2 into the cover.

 Cautions

- In case of slippage or movement, maintain correct placement of the CPRmeter 2 on the patient's chest.
- Do not use the Silicone Cover with patient adhesive.

 Note

- Do not delay CPR. If the Silicone Cover is generally used with the CPRmeter 2, it should be applied to the CPRmeter 2 in advance.

Use and Maintenance

The CPRmeter 2 use or maintenance is not changed by use of the Silicone Cover. The Silicone Cover must be cleaned first for effective disinfection or sterilization. Disinfect or sterilize the Silicone Cover using the respective method.

Instruction for Cleaning:

1. Rinse the device in a sink under running, lukewarm water (22-43 °C) from a tap for at least 30 seconds. Submerge parts in lukewarm tap water (22-43 °C) ensuring that all surfaces are in contact with water for at least 2 minutes before exposure to detergent.
2. Prepare a cleaning solution by mixing 1 tsp. (5 ml) of mild dishwashing liquid in 4 l of hot tap water (60-70 °C).
3. Fully immerse the device in the cleaning solution for a minimum of 5 minutes.
4. Using a soft brush, scrub the device for a minimum of 3 minutes.
5. Rinse the device under running, lukewarm tap water (22-43 °C) for a minimum of 1 minute.

Disinfection in ortho-phthalaldehyde:

1. Immerse the device completely in Cidex OPA Solution for a minimum of 12 minutes in room temperature (20-25 °C).
2. Remove the device from the solution.
3. Rinse the device under running, lukewarm tap water (22-43 °C) for a minimum of 3 minutes.

Sterilization:

Autoclave with distilled water for 4 minutes at 132 °C in a wrapped sterilization configuration using autoclave cycles with Pre-Vacuum Air Removal.

 Warning

With the use of the Silicone Cover, the CPRmeter 2 is not considered a disinfected or sterilized device.

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