





## SimPad – Job Aid Register SimPad Online

**Note:** To register your SimPad on SimStore online, be sure to have the SimPad connected to a network with internet access.

Select SimStore icon.

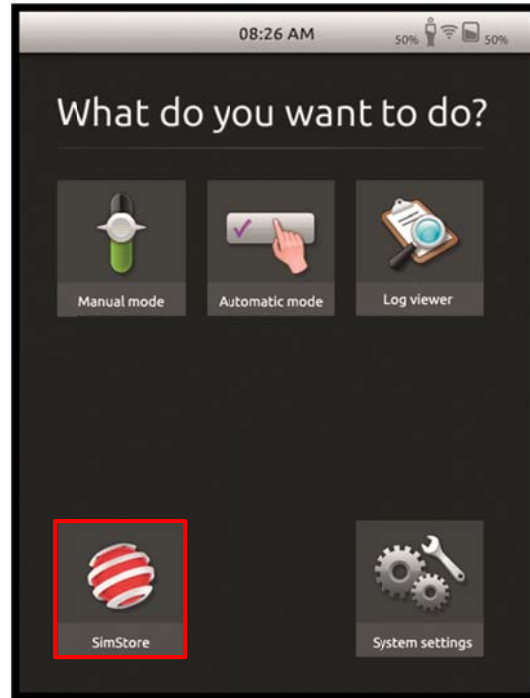


Select Register SimPad.

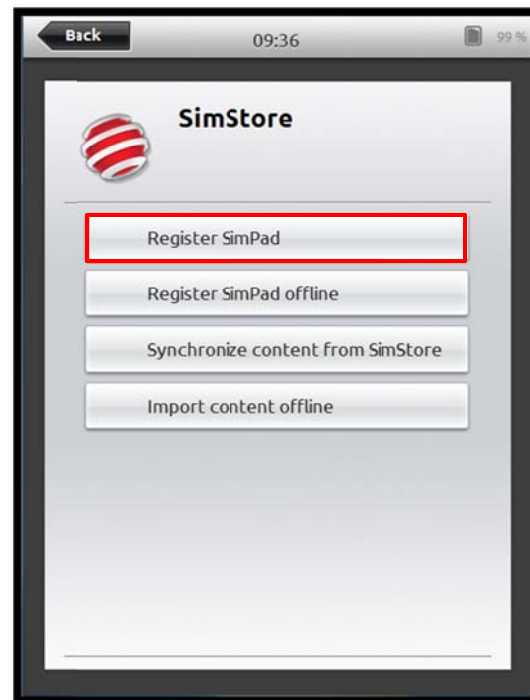



## SimPad – Job Aid Register SimPad Online

Touch the Institution field to reveal the keyboard.

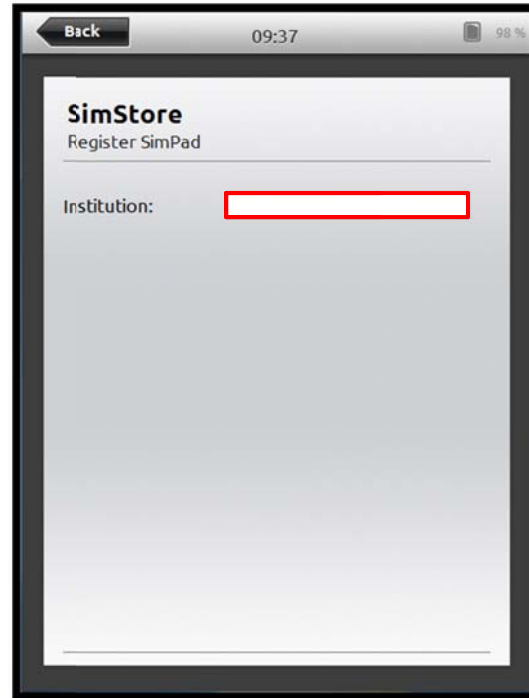

Type your Institution's name then select OK.


**Tip:** Avoid using generic terms like university and hospital as they will yield a large number of search results.






## SimPad – Job Aid Register SimPad Online

**Note:** This screen is displayed as SimPad contacts the SimStore server.

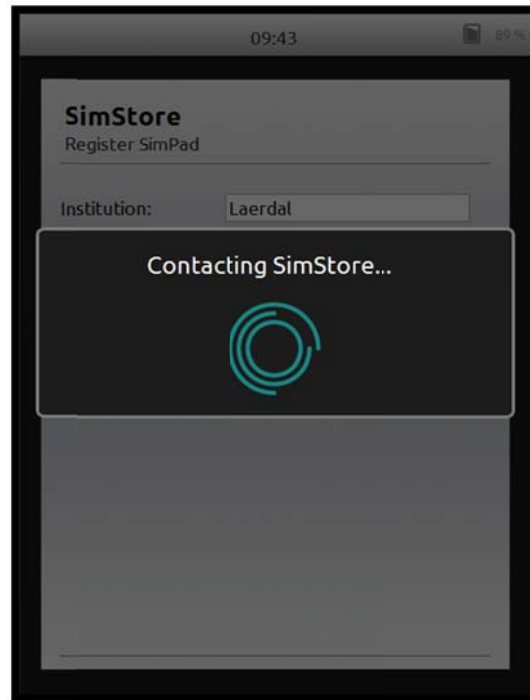

**Note:** This screen shows the results of the search.

**Select your Institution's name.**

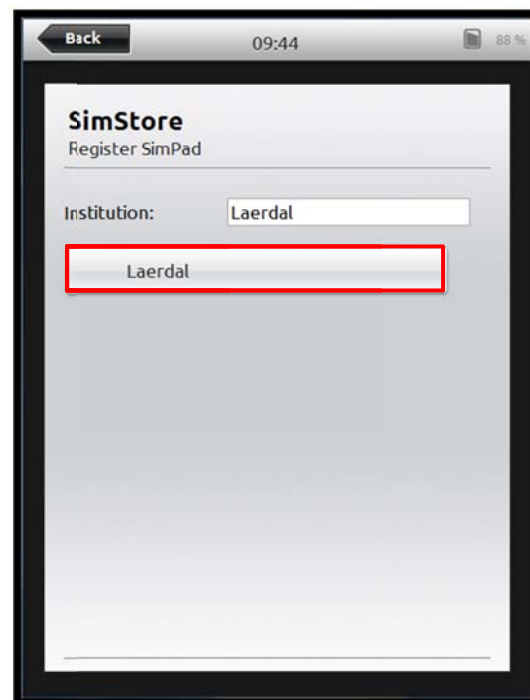



## SimPad – Job Aid Register SimPad Online

Touch the User name field to reveal the keyboard.

The screenshot shows the 'SimStore Register SimPad' form. At the top, there is a 'Back' button, the time '09:44', and a battery icon at '87%'. Below the title, the 'Institution' is set to 'Laerdal' with a 'New search' button. The 'User name' field is highlighted with a red box, and the 'Password' field is empty. A 'Register SimPad' button is visible at the bottom.

Type your user name and password then select OK.

The screenshot shows the same form with the keyboard open. The 'User name' field now contains 'MyUser'. The 'Password' field contains 'MyPassword'. The 'OK' button is highlighted with a red box. The keyboard is visible at the bottom of the screen.



## SimPad – Job Aid Register SimPad Online

**Note:** When both fields contain valid input the Register SimPad button appears.

**Select the Register SimPad button.**

The screenshot shows a mobile application interface for 'SimStore' with the title 'Register SimPad'. At the top, there is a 'Back' button, the time '09:45', and a battery icon at '86%'. Below the title, there is a search bar with 'Institution' set to 'Laerdal' and a 'New search' button. Two input fields are present: 'User name:' with the value 'MyUser' and 'Password:' with the value 'MyPassword'. A 'Register SimPad' button is located below these fields and is highlighted with a red rectangular border.

**Note:** This screen appears when SimPad is successfully registered.

You are now able to synchronize your SimPad with content purchased on SimStore.

The screenshot shows the same 'SimStore Register SimPad' screen as above, but with a dark overlay dialog box in the center. The dialog box contains the text 'SimPad successfully registered ✓' and a green 'OK' button. The background of the screen is dimmed.