




SimPad – Job Aid Synchronize Content from SimStore

Note: To synchronize SimStore content with SimPad, be sure to have the SimPad connected to a network with internet access.

Select SimStore icon.

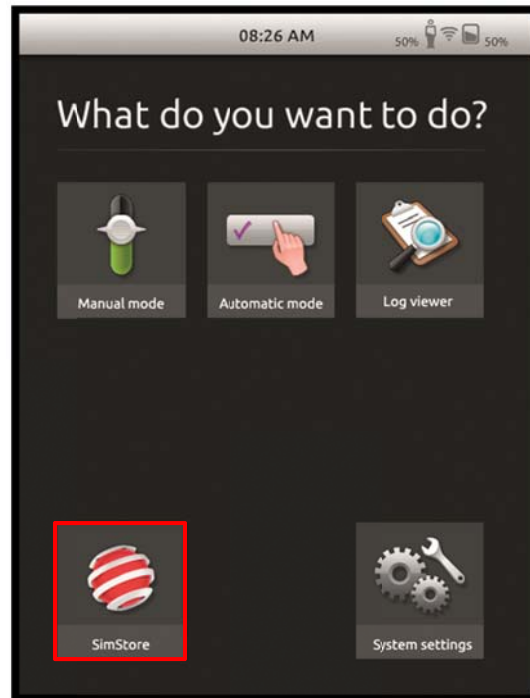


Select Synchronize content from SimStore.






SimPad – Job Aid Synchronize Content from SimStore

Select Synchronize.

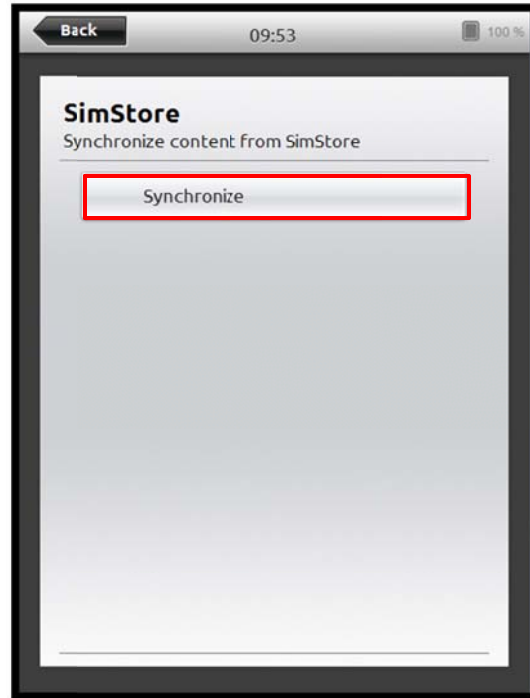

Note: The Contacting SimStore screen appears as SimPad is downloading content from your account.

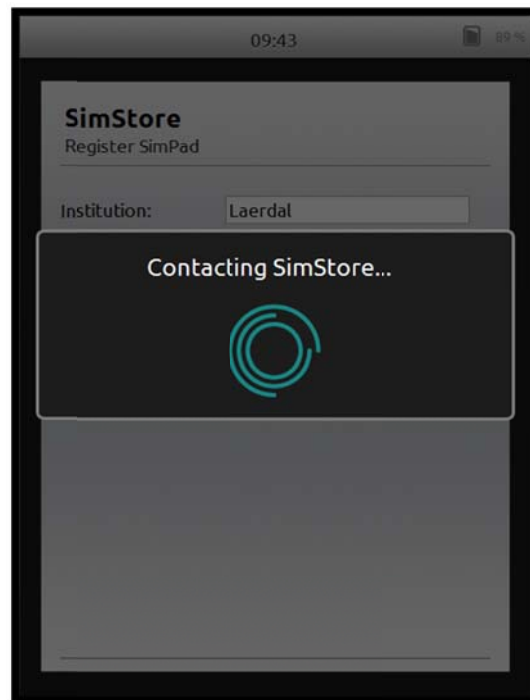



SimPad – Job Aid Synchronize Content from SimStore

Note: The date and time are displayed when SimPad completes the synchronization process with SimStore.

