
# Multi-Venous IV Training Arm for Resusci Anne Modular System

## Directions for Use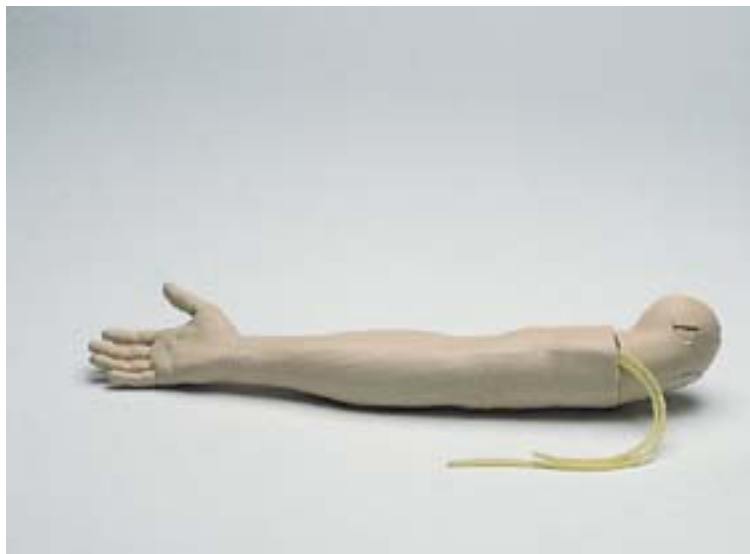

312025

---

# TABLE OF CONTENTS

Recommendations	3
Items Included	4
Skill Taught	4
Anatomy Represented	4
Instructions for Use	4
Replacing Skin and Veins	4
Replacing Veins	4
Care and Maintenance	5
Limited Warranty	6
Replacement Parts	7
Directions for Use ( <i>French Version</i> )	8

---

---

---

Model 312025 Multi-venous IV Training Arm for Resusci Anne Modular System is a full-size lifelike arm, which realistically simulates that of a patient. It is specifically designed for training professionals in the practice of venipuncture.

As a result of quality construction and the ease of replacing individual parts, this simulator should provide many sessions of training when reasonable care and maintenance are practiced.

---

---

## **Recommendations**

Catheter – Size 21-gauge or smaller

When using a catheter or other venipuncture device, lubricate them with silicone spray for easier insertion.

## **Instructor Notes:**

**Items Included:**

- (1) R.A. Right Multi-Venous Training Arm
- (1) Lubricant Spray
- (1) Simulated Blood
- (1) Set Replacement Skin & Veins
- (1) Set Syringes/21-gauge Needles (5)
- (1) Carry Case

**Skills Taught:**

- Adult Venipuncture

**Anatomy Represented:**

- Dorsal veins of hand (3)
- Cephalic
- Median
- Basilic
- Antecubital

**Directions for Use:**

1. Attach IV tubing to either latex vein.
2. Allow fluid to flow through arm and out other latex vein.
3. Clamp off flow of water.

Training arm is now ready to practice venipuncture.

When excessive leaking occurs at puncture sites, either a new multi-vein system or skin should be installed to reduce loss of fluid. The recommended working place is near a sink.

**Replacing skin and veins:**

1. Cut off skin. This can be done with a sharp knife or scalpel. (See Fig. 1)


**Replacing veins:**

1. Lubricate inside of skin with liquid detergent.
2. Begin at top of arm and roll skin down and off arm and hand. (Thumb will detach with skin.)

**Note:** If reusing skin after replacing tubing, make sure thumb insert is in place before continuing. A thumb is inserted in new skin.

3. Remove tubing from track in mandrel. Tack glue may need to be cut away to allow removal.
4. Rinse and dry vein grooves well and swab with alcohol. Be sure to remove any excess glue.
5. Place new veins along grooves, spot gluing as needed. (See Fig. 2)
6. Rinse inside of arm skin with soap and water. Lather arm mandrel with liquid soap.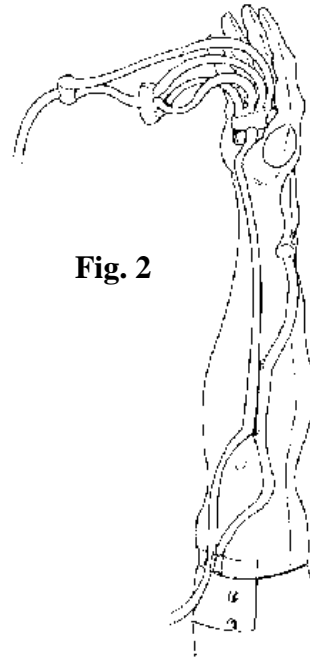

7. Slide arm skin over mandrel and work mandrel into fingers. (See Fig. 3)
8. Rinse off excess soap and dry.

**Fig. 3**

**Care and Maintenance:**

1. Clean with mild soap and water; do not submerge in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin and module should be cleaned after each training session and a general inspection should be conducted regularly.

4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed. After use of injection pads (*use water only*), accumulated water should be squeezed out. **Do not store wet foam pads in the skin.** To prevent mildew or mold, pads can be soaked in a mild solution of disinfectant and water or bleach and water. Squeeze excess solution from pads, allow them to dry, then store or reinsert in manikin.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.

### **Limited Warranty**

Laerdal Medical warrants to the purchaser that its products are free from defects in material and workmanship for a period of one (1) year from the date of purchase by the original user. During the designated one (1) year period, MPL will, upon receipt of a product found to be defective due to materials or workmanship from the purchaser and notification in writing of the defect, at its option repair or replace any parts found to be defective or the entire product. Warranty period does not renew with replacement or repair of original product.

Products found to be defective and notification of defects may also be sent to the authorized Laerdal Medical dealer from whom the product was purchased. All postage, shipping or handling charges shall be the sole responsibility of the purchaser.

Laerdal Medical is responsible for the effects of safety, reliability and performance of its product(s) only if:

- service, repair, readjustment or modification is carried out by Laerdal Medical or persons authorized by Laerdal Medical.
- the electrical installation of the room where the product is used complies with pertinent equipment requirements.
- the product is used in the proper manner in strict compliance with its Directions for Use.

Laerdal Medical shall not be liable under this warranty for incidental or consequential damages, or in the event any unauthorized repairs or modifications have been made or attempted, or when the product, or any part thereof, has been damaged by accident, misuse or abuse. This warranty does not cover batteries, fuses, normal wear and tear, excessive use, staining, discoloration or other cosmetic irregularity, which does not impede or degrade product performance.

Some states in the USA do not allow the exclusion or limitation of incidental or consequential damages, so those limitations or exclusions may not apply to you.

There are no other express or implied warranties, whether of merchantability, fitness or purpose, or otherwise, on the product, its parts and accessories.

**REPLACEMENT PARTS:**

The following replacement part numbers are for simulators with white skin tone.

<u>Part Number:</u>	<u>Description:</u>
312029	Replacement Skin & Multi-Vein System
300-00650	Manikin Lubricant
300-00750	Simulated Blood Red

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Sweden: LAERDAL MEDICAL AB
United Kingdom: LAERDAL MEDICAL LTD.
Laerdal Medical Corporation
Medical Plastics Laboratory, Inc.