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ENGLISH Directions for Use

Original Stifneck Extrication Collar

- Measure the patient**
Align the head to neutral or "eyes forward" position unless contraindicated by your protocol.
- Match the collar size to the patient**
Choose from 4 adult and 2 child size collars.
Plastic edge to hole or to black post if assembled
- Assemble the original collar**
Insert the black post into the hole.
- Preform the collar**
- Apply the collar while manually maintaining neutral head position**
Place the chin support well under the chin. If a different size is needed, remove, and re-apply the collar.
For a supine patient, slide the rear panel behind the neck before placing the chin support.

Storage: Do not store collar in folded position. Store flat.
Storage Temperature Range: -34°C to 52°C (-30°F to 125°F)
Operating Temperature Range: -18°C to 43°C (0°F to 110°F)

IMPORTANT

All Stifneck products should only be used by persons who have received adequate training. In cases of suspected spinal injury, proper cervical immobilization is only one part of an immobilization system.

It is important that the patient be properly immobilized to prevent movement of the spine (per local protocol).

Do not use an improperly sized collar. Too large a collar may hyperextend a patient's cervical spine; too small a collar may not provide appropriate stability. Special sizes of Stifneck collars are available for children and others with small bodies.

WARNING

To insure proper use, please review all material in this DFU.

Intended for single patient use.

The product is in compliance with the essential requirements of Council Directive 93/42/EEC Medical Devices Directive.

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© 2005 Laerdal Medical AS. All Rights Reserved.