

# Pediatric Pocket Mask

## User Guide

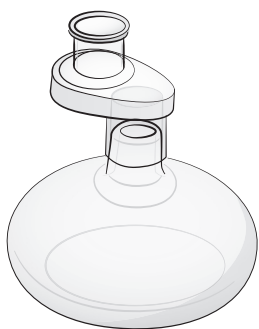www.laerdal.com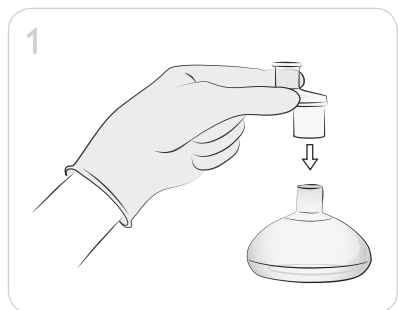

EN  
FR  
IT  
ES  
NO  
NL  
SV  
DA  
FI  
DE  
JA  
PL

### ENGLISH

Inspect and verify that this product is complete and without defects before use.

#### Intended Use

The Laerdal Pediatric Pocket Mask is designed for mouth-to-mask ventilation of a non-breathing child or infant. For use by trained personnel only.

1. Attach one-way valve to mask port by twisting it until seated firmly (figure 1).
2. Open patient's airway.
3. Apply mask over patient's mouth or mouth and nose. If applying over mouth only (as used for children) be sure to pinch nostrils to avoid air leakage (figure 2).
4. Seal as illustrated. Open airway. Blow slowly until chest rises (figure 3).
5. Remove your mouth. Allow patient to exhale.

Note

If patient vomits, remove mask and clear patient's airway. Clear mask thoroughly of foreign material. Blow through valve to verify operation. Reapply mask to patient and continue ventilation.

**To disassemble**, hold mask with one hand, gripping the one-way valve between your thumb and two fingers of the other hand. Separate by twisting valve off.

#### Patient use

Mask and accessories are single use and should be disposed of after use. Can be used on same patient during a 24-hour period. Laerdal is not responsible for any consequences of re-use.

#### Training use

Mask can be cleaned and reused only for future training. One-way valve must be disposed of after use.

#### Cleaning and Disinfection

Wash and scrub in warm soapy water. Rinse in clean water. Submerge for 10 min. in a 0.5% sodium hypochlorite solution. Rinse again and allow to air dry.

#### Temperature limits

Operating: -10°C (14°F) to 40°C (104°F).  
Storage: -40°C (-40°F) to 50°C (122°F).

#### Expiratory resistance for patient

Mask & one-way valve: Approximately 4,3 cm H<sub>2</sub>O

#### Inspiratory resistance for patient

Mask & one-way valve: Approximately 3,3 cm H<sub>2</sub>O

#### Deadspace 90,5 ml

Performance data are nominal values only.

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© 2020 Laerdal Medical AS. All rights reserved
**Distributed in USA by:** Laerdal Medical Corporation
**Distributed in Canada by:** Laerdal Medical Canada Ltd.