

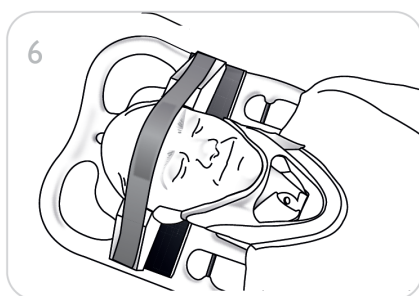
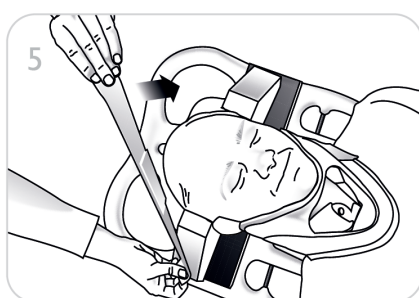
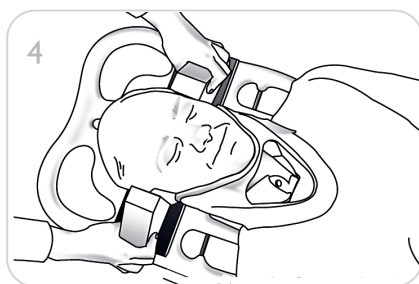
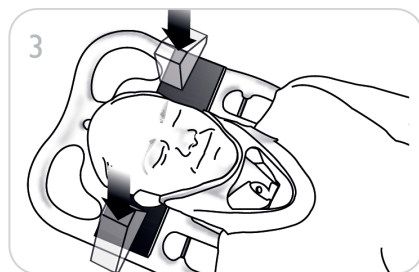
Sta-Blok Head Immobilizer

User Guide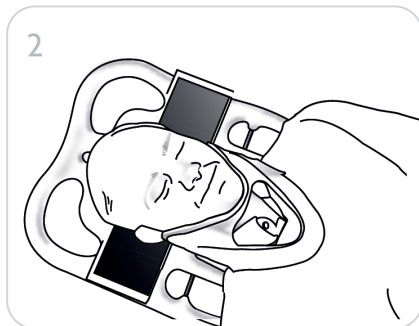

REF Cat. No. 700-00001 QTY: 30 pieces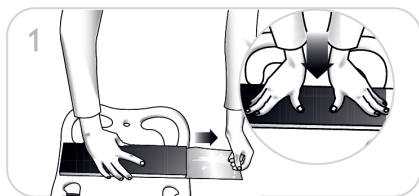

The Sta-Blok Head Immobilizer is to be used as a part of a total immobilization system in conjunction with other immobilization products and techniques to help reduce potential for movement of the patient's head and neck in cases where spinal fracture, dislocation and/or spinal cord injury is suspected.

Warnings

- In cases of suspected spinal cord injury, proper head immobilization is only one part of a total spinal immobilization process.
- Only qualified healthcare providers trained in the use of this and other immobilization products should use the Sta-Blok Head Immobilizer. Failure to use this product as directed, and under prescribed protocols from the local cognizant medical control authority, may result in patient injury and/or disability.

Cautions

- All healthcare provider personnel should read and fully understand these instructions prior to using the product.
- Store product in a cool, dry place.
- Do not store in excessive heat or in direct sunlight.

Note

These Directions are general guidelines and instructions for proper use of the Sta-Blok Head Immobilizer. They are not intended to be and should not be construed as teachings, recommendations or directions for clinical practice.

Components

Each sta-blok package contains the following

- Occipital Pad - Foam pad with blue loop (hook and loop) fastener top surface.
- Side Support Blocks - Triangular foam blocks with hook (hook and loop) fastener surfaces.
- Standard Strap - White loop (hook and loop) fastener strap.
- User Guide

Positioning the sta-blok occipital pad

1. Place the spineboard on a stable surface. Pull the liner from the Occipital Pad. Press Pad down firmly to secure it to the Spineboard.
2. Place the patient on the spineboard. Use an extrication collar if available.

Positioning side support blocks

3. Place a Side Support Block against either side of the head, just above the ears.
4. Press the Blocks firmly down onto the Occipital Pad.

Securing patient's head

5. Apply the Standard Strap with the foam centered on patient's forehead.
6. Attach each hook and loop end to the Side Support Blocks.

Ensuring a neutral position

For children under 8 years old or a patient with helmet, use spinal alignment padding if needed to obtain a neutral position for the spine.

Note

If circumstances prevent either the Standard Straps from being used, suitable lengths of conventional medical adhesive or other tape may be substituted.

| Technical Specifications | |
|--------------------------------|------------------------------------|
| Operating temperature | -15 °C to 38 °C (5 °F to 102 °F) |
| Storage temperature | 0 °C to 52 °C (32 °F to 125 °F) |
| Operating and storage humidity | 5-95% RH non-condensing. |
| Non-sterile | |
| Symbol Glossary | |
| | Single Patient Use. Do not re-use. |
| | Not made with natural rubber latex |
| | Lot number |

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© 2023 Laerdal Medical AS. All rights reserved.
Sta-Blok is a trademark of Laerdal Medical AS