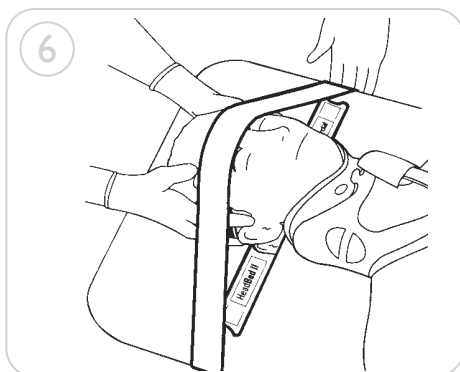
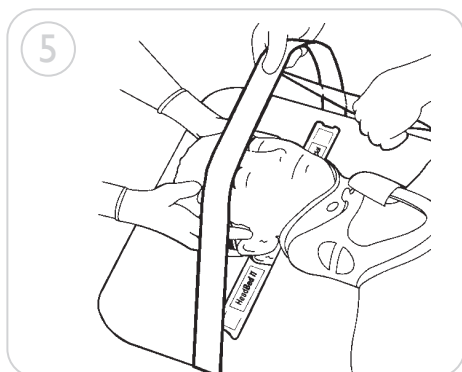
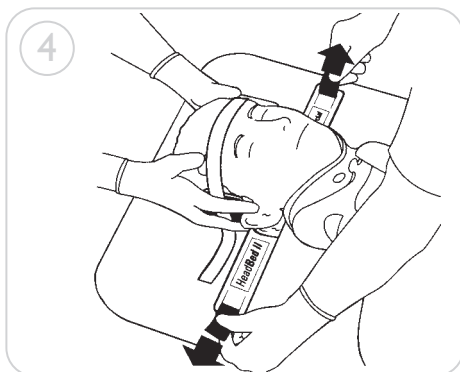
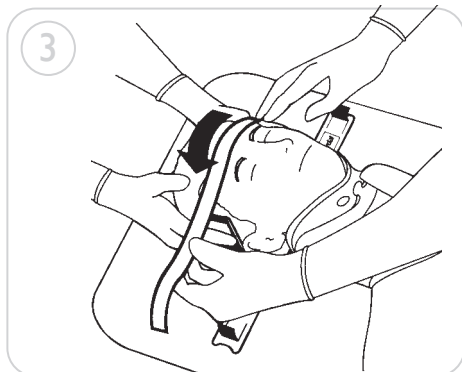
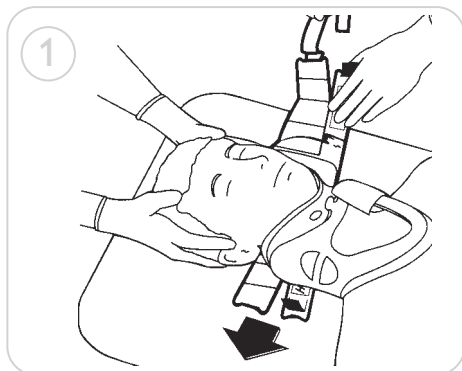
HeadBed II Cervical Immobilization Device

User Guide

Rx ONLY

LATEX

MD

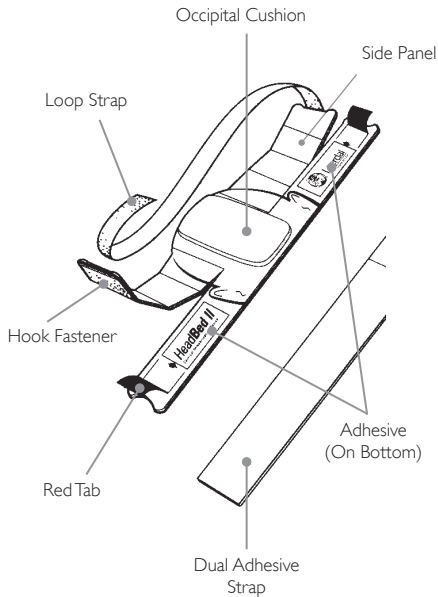
www.laerdal.com

Laerdal

helping save lives

Components

Setup Instructions

- 1. Position under head**
Pad if necessary. (See padding guidelines.)
- 2. Wrap side panels up**
Apply panels to contact the head at the top of the ears.
- 3. Apply loop strap**
Apply loop strap across the forehead above eyebrows.
- 4. Affix to board**
Pull both red tabs and press firmly on white areas.
- 5. Strap across forehead**
Attach one end of the Dual Adhesive Strap to board as shown. Position strap so it adheres to the eyebrows and the loop strap.

Note

To avoid tangling the strap, peel the liner off as you go rather than all at once.

- 6. Secure strap to board**
Secure the strap to the opposite side of the board.

Note

Apply strap snugly but not so tightly as to cause patient discomfort.

Temperature limits

Storage	-34 °C to 52 °C (-30 °F to 125 °F)
	5% to 95% relative humidity
Operating	0 °C to 43 °C (32 °F to 110 °F)

Symbol Glossary

	Caution: Federal law (US) restricts this device to sale by or on the order of a physician or appropriately licensed practitioner.
	Single Patient Use. Do not re-use.
	Medical Device
	Not made with natural rubber latex

⚠ Warnings

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

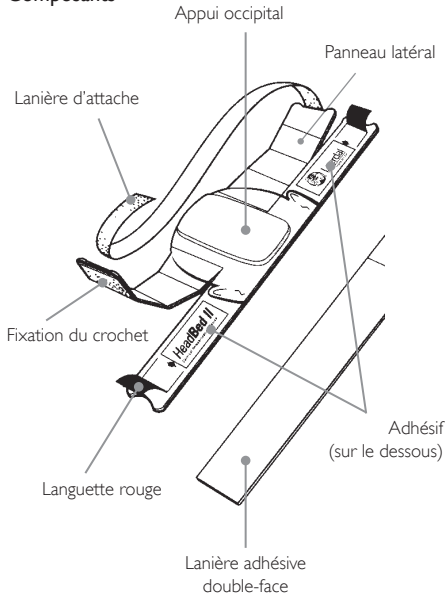
- Manual immobilization of the head must be maintained until application is complete.
- The HeadBed II Cervical Immobilization Device (CID) is only one part of a spinal immobilization system. Proper patient care requires that both the head and the body are effectively immobilized.
- This device should only be used by trained personnel in accordance with the User Guide.
- Designed for single patient use only. Do not re-use. Re-use will lead to increased risk of cross-contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.
- Prior to use, inspect all parts to ensure they are free of damage and working properly.
- The patient should be constantly monitored while this product is in use.

⚠ Cautions

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the Thomas Tube Holder.

- Application pressure, surface condition, and temperature can affect the bond strength of the adhesive. The HeadBed II CID is most effective when firmly attached to the spine board.
- In wet conditions, a piece of tape placed directly over the dual adhesive strap and completely encircling the spine board may be appropriate.

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