Laerdal Pedi-Pad Backboard Pad

for use with Sta-Blok/SpeedBlocks Head Immobilizers

FOR CHILDREN 8 YEARS OR YOUNGER

INSTRUCTIONS FOR USE


For proper body position when transporting small children requiring head immobilization with either the Sta-Blok or SpeedBlocks Head Immobilizers, use the Laerdal Pedi-Pad by placing it under the child's back and shoulders as shown at right and below. Properly secure the patient. Ensure that the patient is in neutral position and follow local protocols for immobilization.

Storage temp.: -34 °C to +52 °C
Operating temp.: -25 °C to +43 °C

Single Use Warning: Designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.

Catalog No.: 701-000 40 - Qty: 6 Pads
Product: Laerdal Pedi-Pad for use with Sta-Blok and SpeedBlocks Head Immobilizers

LOT Number:

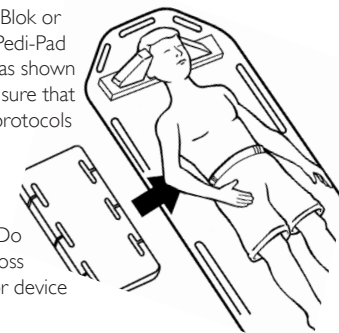	Not made with natural rubber latex.		Medical Device		Do not re-use
	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.				

©2020 Laerdal Medical AS. All rights reserved.

Manufactured in Mexico for Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30, 4002 Stavanger, Norway
5220 rev I

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