



Laerdal[®]

helping save lives



Realistic Clinical Skills Training

Nursing Anne

Nursing Anne is educationally effective for clinical training targeting key skills from basic patient care to advanced nursing competencies. Nursing Anne paired with the SimPad System provides an economical and relevant training platform for women's health, wound assessment and general patient care.



laerdal.com

Visit us to learn more
about Nursing Anne



Nursing Anne

PATIENT CARE:

- Head with anatomical landmarks, trachea, and esophagus, along with simulated lungs and stomach, allow the practice of many procedures
- Irrigation of the eye and ear (simulated)
- Application/instillation of medications in the eye, ear and nose including nasal packing
- Mouth and denture care procedures
- Insertion and suctioning of oropharyngeal and nasopharyngeal airways
- Insertion, securing, and care of endotracheal tubes
- Tracheostomy care and tracheal suctioning
- Various oxygen delivery procedures
- NG tube insertion, care, medication administration, and removal
- Gastric lavage and gavage
- Full range of motion for realistic patient handling
- Fingers and toes are spread to allow bandaging
- Interchangeable stomas depict colostomy, ileostomy and suprapubic cystostomy
- Colostomy may be irrigated and will retain an indwelling catheter

INTERCHANGEABLE MALE AND FEMALE GENITALIA:

- Complete urinary catheterization
- Can be attached to urinary and colon reservoirs via connector valves
- Female genitalia capable of vaginal douching
- Will retain indwelling or straight catheter
- Enema procedures may be performed using fluid for realistic return
- When used with manikin, fluid may be used for realistic return
- Urinary valves give the natural resistance felt when catheterizing
- Anal valves simulate the internal anal sphincter

CIRCULATORY SKILLS AND IV DRUG ADMINISTRATION:

- Articulating IV training arm with replaceable skin and infusible vein system allows peripheral intravenous therapy and site care
- Venipuncture possible in the antecubital fossa and dorsum of the hand
- Accessible veins include median, basilic and cephalic
- Subclavian, jejunostomy and Hickman catheter openings (placement only)
- Deltoid, dorsogluteal, and vastus lateralis IM injections possible

BLOOD PRESSURE / PULSES:

- Articulating blood pressure arm for auscultated and palpated blood pressure simulation
- Korotkoff sounds synchronized with ECG
- Korotkoff sounds volume control in 10 steps, 0-9
- Systolic and diastolic pressure may be set individually in steps of 2 mmHG
- Systolic 0-300 mmHG, Diastolic 0-200 mmHG
- Auscultative Gap, with on/off feature
- Pressure accuracy +/- 4 mmHG
- Calibrate function to adjust pressure sensor and cuff gauge
- Palpated pulse simulator controls brachial and radial pulses
- Pulses only active when palpated
- Pulses synchronized with programmable ECG
- Pulse strengths dependent on Blood Pressure
- Manually generated carotid pulse

SOUNDS:

- Heart sounds synchronized with ECG
- Auscultated lung sounds synchronized with breathing, 0 - 60 BPM
- Individual lung sound selection
- Normal or abnormal bowel sounds
- Vocal sounds: Computer-generated sounds, recorded vocal sounds and real-time voice input via headset
- User generated vocal sounds

LOGGING / SCENARIO FUNCTION:

- Time stamped activities, vital signs, and notes are captured in the data log
- The SimPad log files can be viewed on the device or on a PC for post-simulation reflection and debriefing
- Upload Scenarios and Themes from a PC via a USB or download scenarios directly from SimStore

SIMULATED PATIENT MONITOR (OPTIONAL):

- Touch screen simulated patient monitors provides concise clinical feedback for physiological parameters
- The monitor's color screen is configurable and provides multiple simulated parameters, each presenting multi-level alarms
- Simulated parameters include HR, ECG, SpO2, BP, RR, Temperature, and etCO2

ORDERING INFORMATION:

325-05050 Nursing Anne Light Skin
325-05050T Nursing Anne Tan Skin
325-05050B Nursing Anne Brown Skin

Includes: Adult Manikin (SimPad-capable), Hospital Gown, Airway Lubricant, Assembly Tool Kit and Directions for Use.

200-30001 SimPad System

Includes: SimPad, Link Box, 2 AC Power Supplies, Rechargeable Lithium-Ion Battery, Wrist Strap for SimPad, Manikin Strap for Link Box, Manikin Adapter Cable, SimPad Sleeve, USB Cable, Ethernet Cable and Directions for Use.

Accessories and Consumables

200-09201 12" Portable Laptop/Tablet Patient Monitor
200-29301 23" Bedside PC Patient Monitor
325-00450, 325-00450T, 325-00450B Fundus Skills and Assessment Module
325-00550, 325-00550T, 325-00550B Wound Care and Assessment Set
325-00650, 325-00650T, 325-00650B Mastectomy Module
325-00750 Breast Exam Module
205-10150, 205-10150T, 205-10150B Adult IO Leg

Visit laerdal.com for a complete listing of wound modules, accessories and consumables.

Operation

Nursing Anne is controlled by the SimPad® System and can be utilized with an optional simulated patient monitor:

**** SimPad Unit required for operation. Sold separately ****



laerdal.com

SimCenter

Visit www.laerdal.com/simcenter to learn how to optimize your simulation training using SimCenter™ and Nursing Anne



SimStore™

Access to validated, standardized content from industry experts

SimManager™

A suite of online tools that allows you to fully optimize your entire simulation center operation

SimView™

An integrated debriefing solution giving you the ability to study simulation events more closely

Services

Laerdal offers comprehensive Educational Services to support your curriculum, and Technical Services to meet your product servicing needs.



Laerdal®
helping save lives