

# Laerdal NetMobile

## How to change the router channel



**Laerdal**  
helping save lives

# First, try to power cycle the router

(it will then do a new search for the least occupied channel)

If this does not help, set a fixed channel on the router:

1.

Start by choosing a **LaerdalNetMobile** connection and enter the password: **SimStore**

(The last four letters 98-fb is the last four letters in the LAN code on the backside of the router.)



2.

Open Internet Explorer and type the following IP address to access the router: <http://192.168.190.254>

Username: **admin**

Password: **LittleAnne**

3.

Click on “Wireless”, then choose “Basic” and set the Channel Number:

(Use WiFi-analyzer or similar App to identify the least occupied channel.)

