


SimMan 3G

Quick Setup Guide

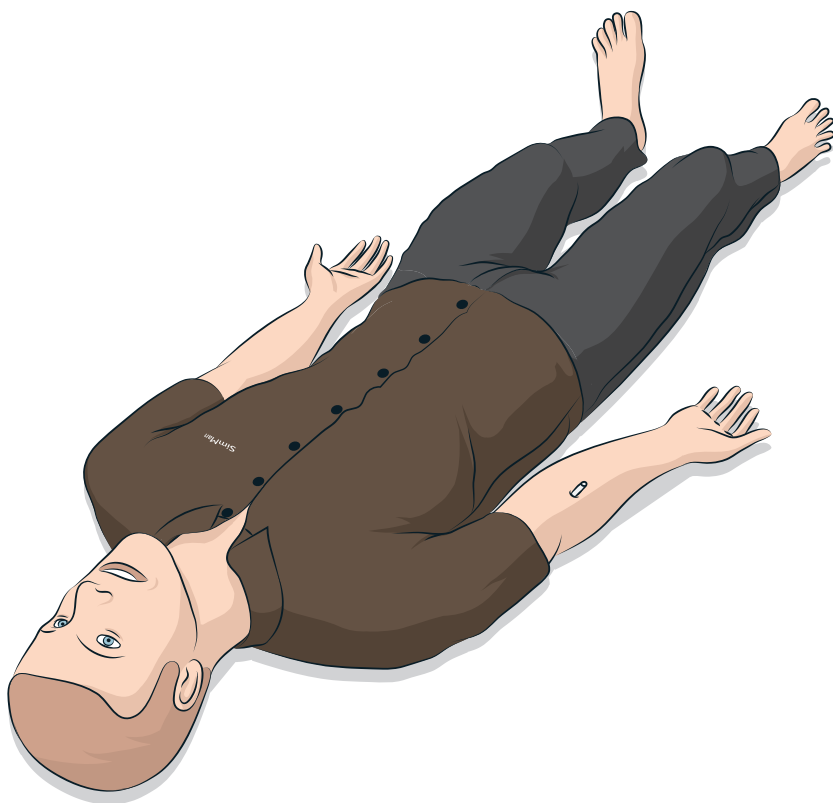


This Quick Setup Guide describes how to assemble and setup your SimMan patient simulator. For information about how to use SimMan, please refer to LLEAP helpfiles and SimMan User Guide.

Visit www.laerdal.com/download to download the latest User Guide and software.




COMPONENTS

Torso

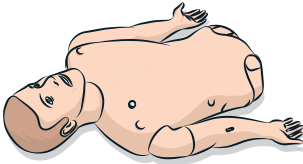

Legs

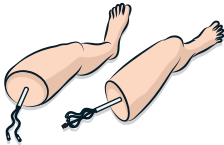


Clothes




Hard Upper Teeth



Male & Female Genitalia

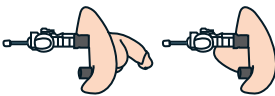


ID Band

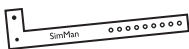

SimMan Arm Adapter



Inline Filter Blood
x 2

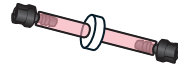

Defib ECG Tightner



ECG Adapter Electrode



Neck Skin
x6



Lung Compliance O-ring
x 10



Fluid & Blood Refill Bottles



Wounds Kit



Chest Drain Pleura
x 6



Blood Concentrate



Airway Lubricant



Blood Pressure Cuff



IV bag



IV Catheter Extension



Cricothyroid Tape




Tibia Pads & Bone
x3



IM Pads



Pneumothorax

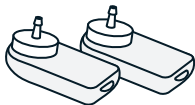

User Guide &
Important Product Information



Headset



Keyboard & Mouse



SpO2 probe



RFID Kit



Tools



External Power Supply




Battery Charger
x 2




1

CHARGE PC BATTERIES

Patient Monitor


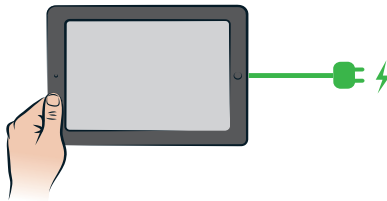





Insturctor PC



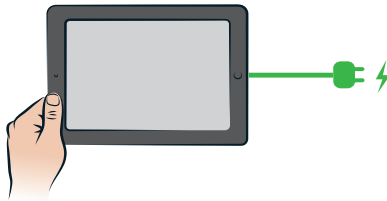


2

UNZIP TORSO SKIN

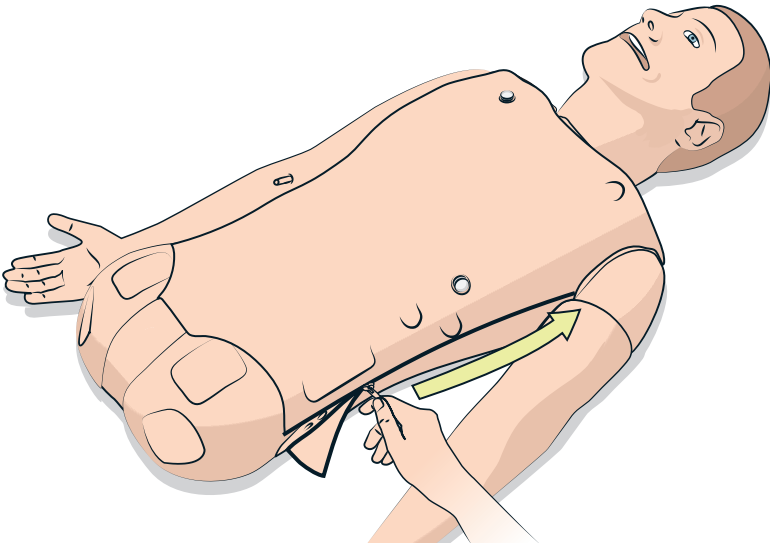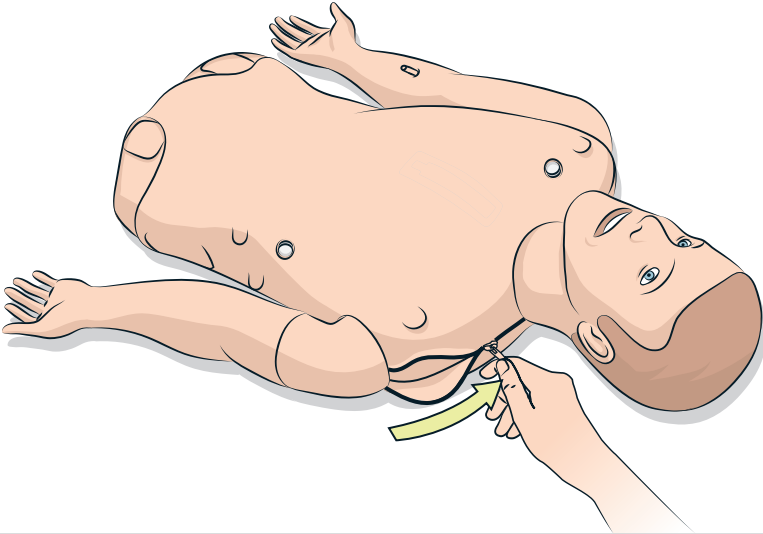


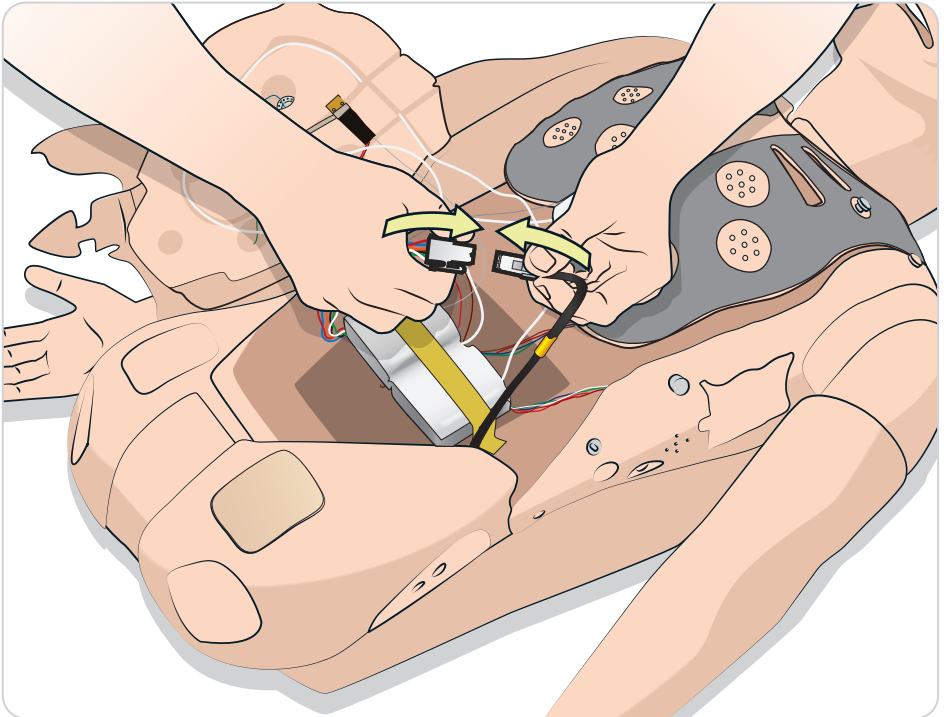
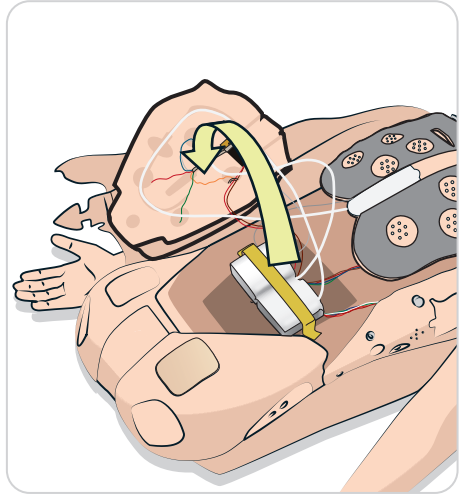
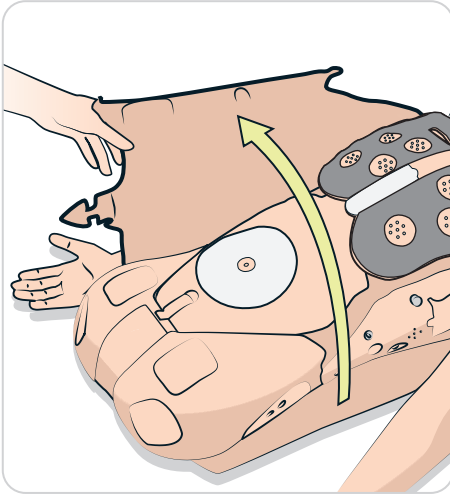


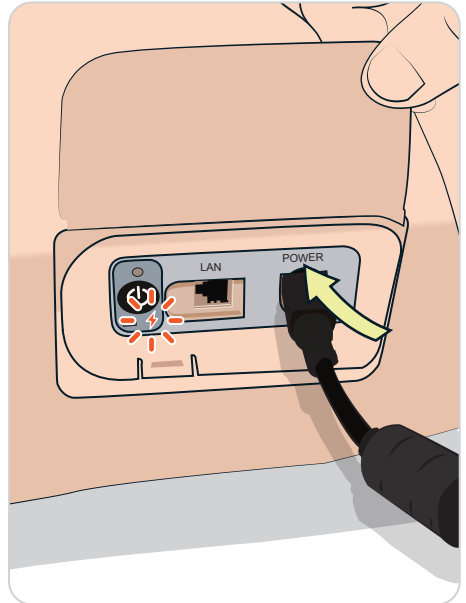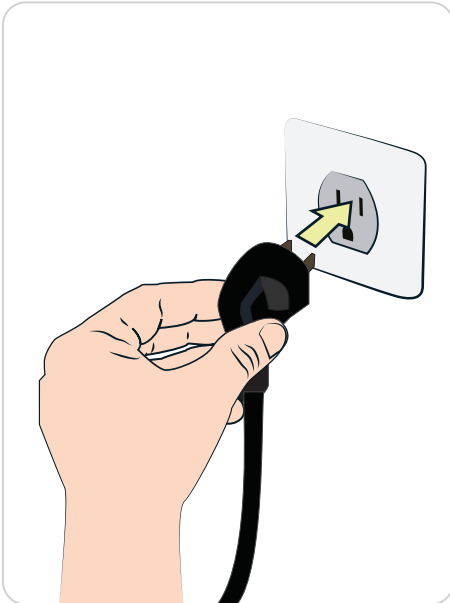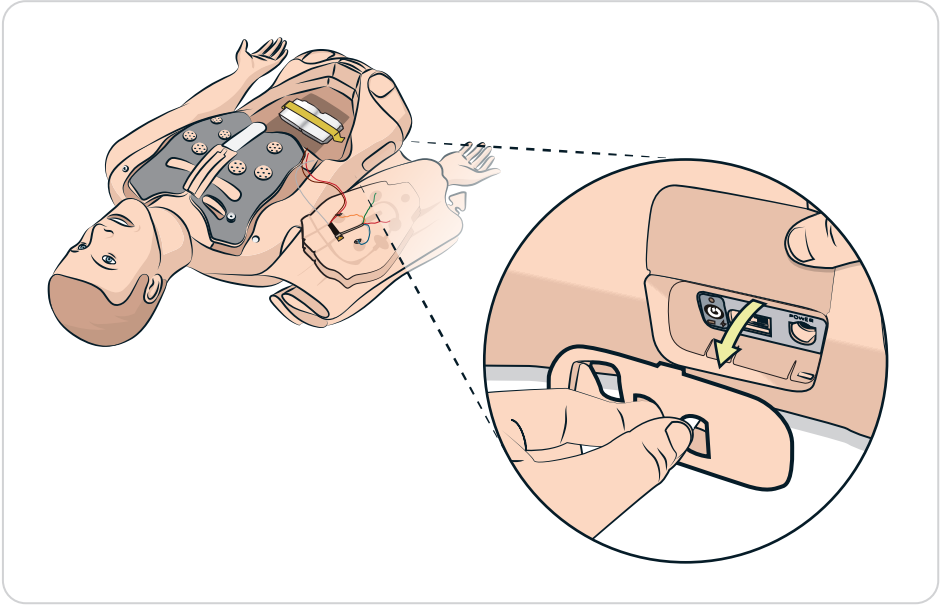

5


ATTACH LEFT LEG



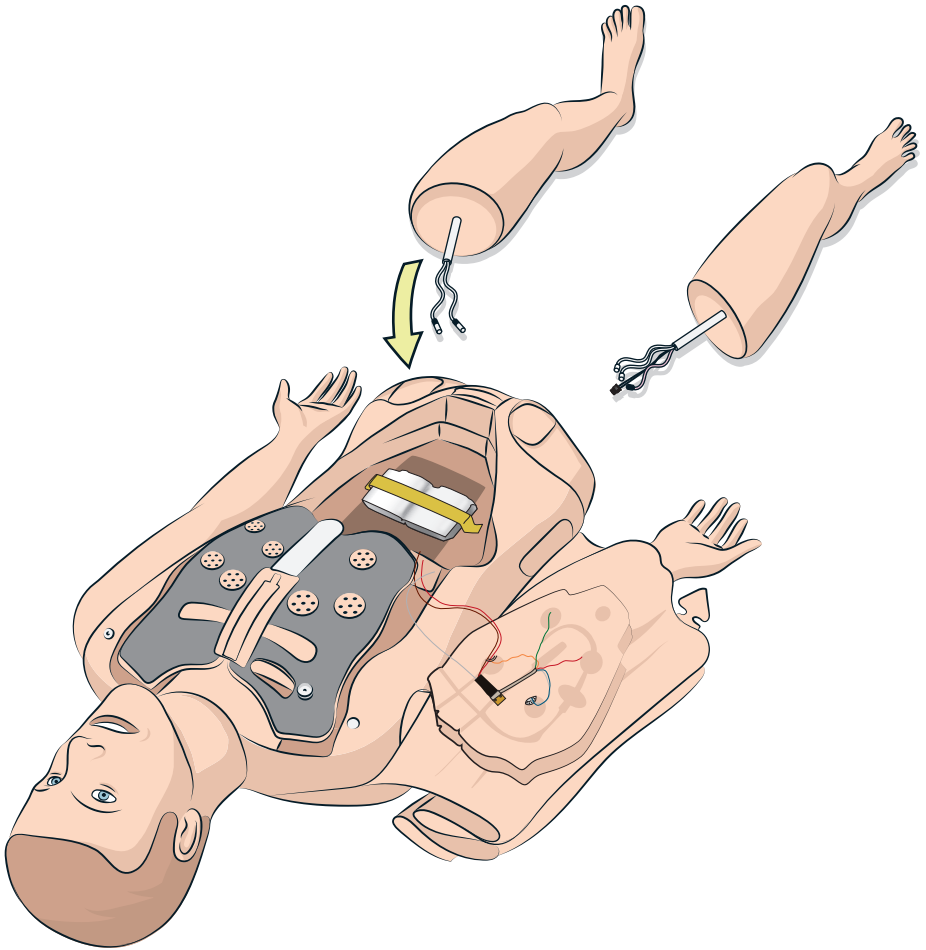


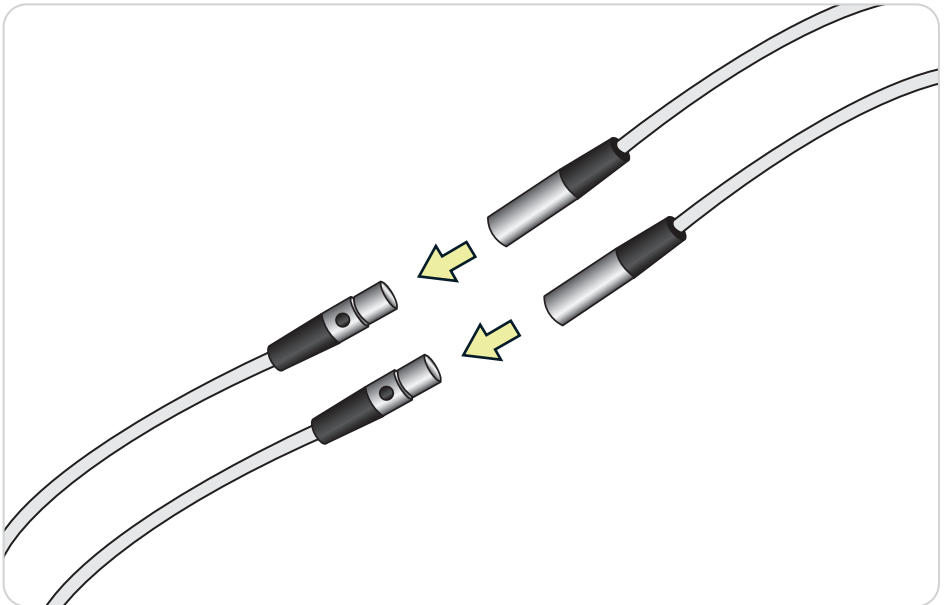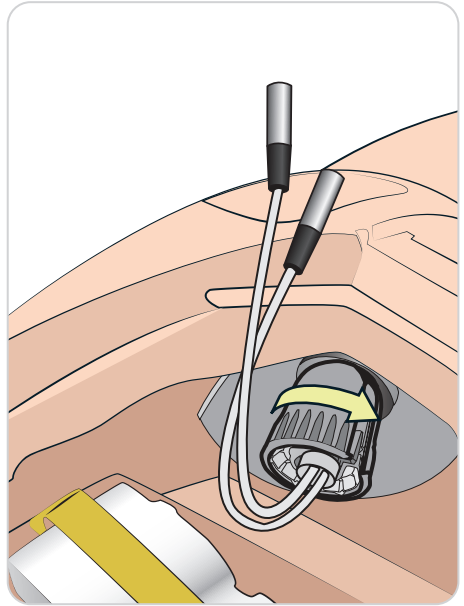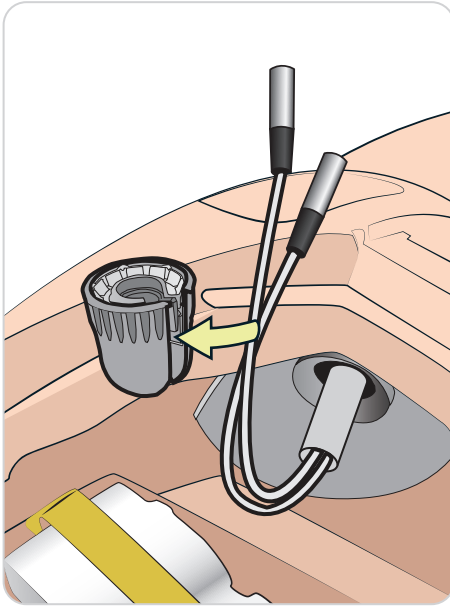

6

ATTACH RIGHT LEG

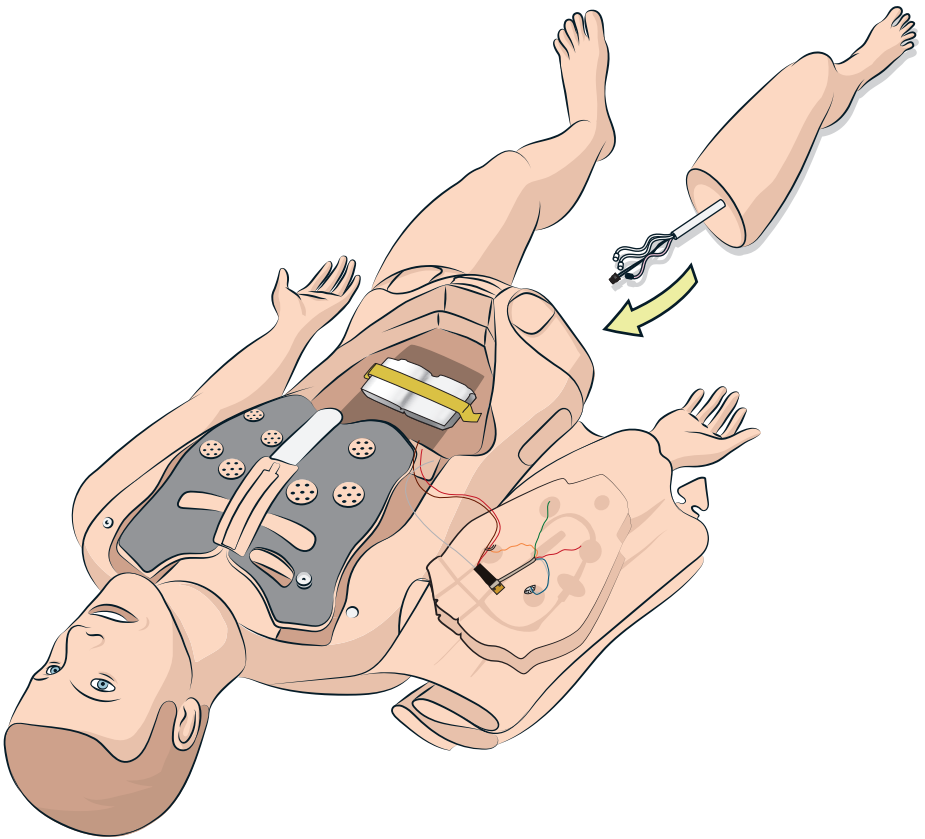

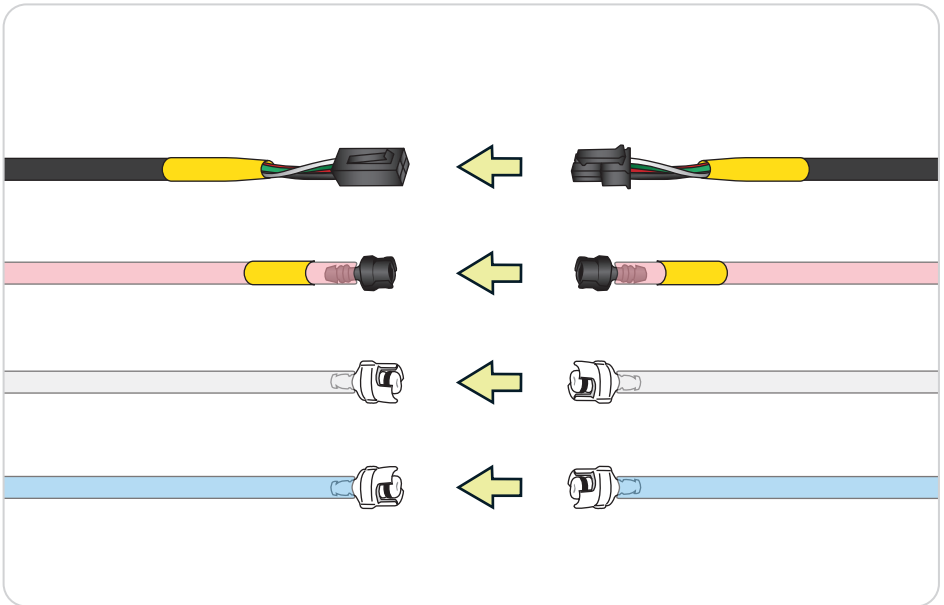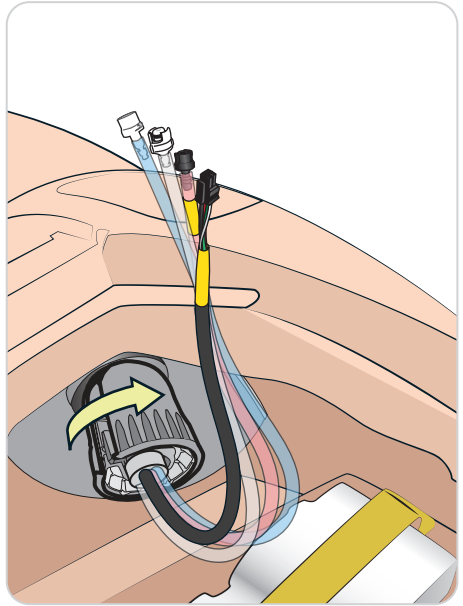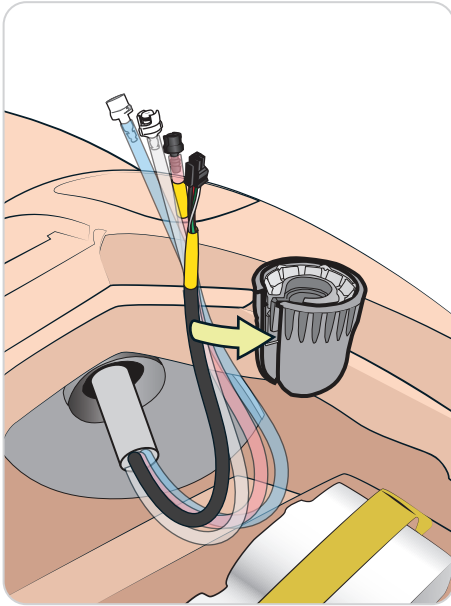


7_A

WIRELESS SETUP

Headset




Instructor PC

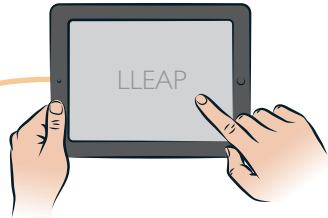


SpO2



Patient Monitor

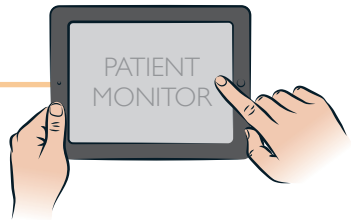

USB cable —

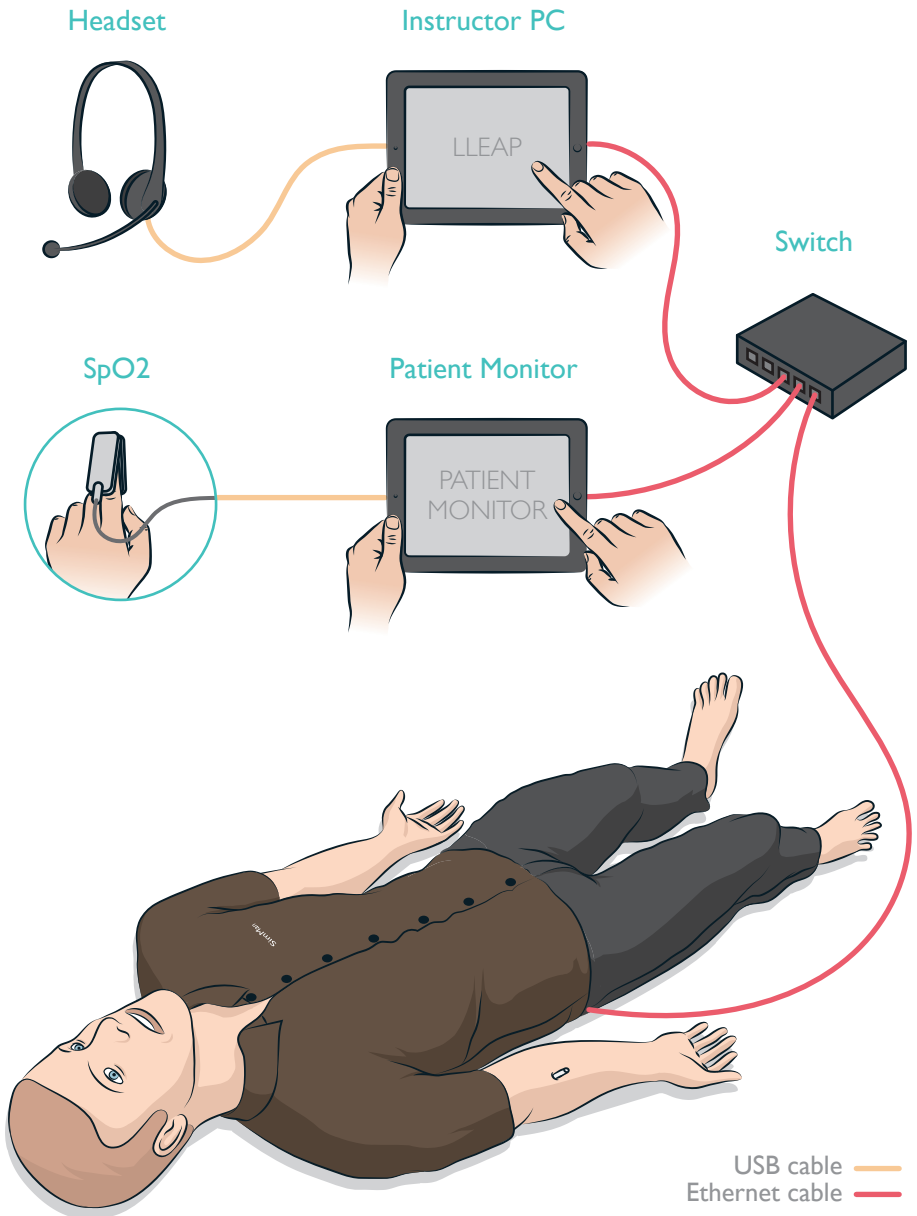

© 2016 Laerdal Medical AS. All rights reserved.
Printed in Norway

20-07889 Rev B

www.laerdal.com



Laerdal
helping save lives