


# SimMan Essential

Quick Setup Guide

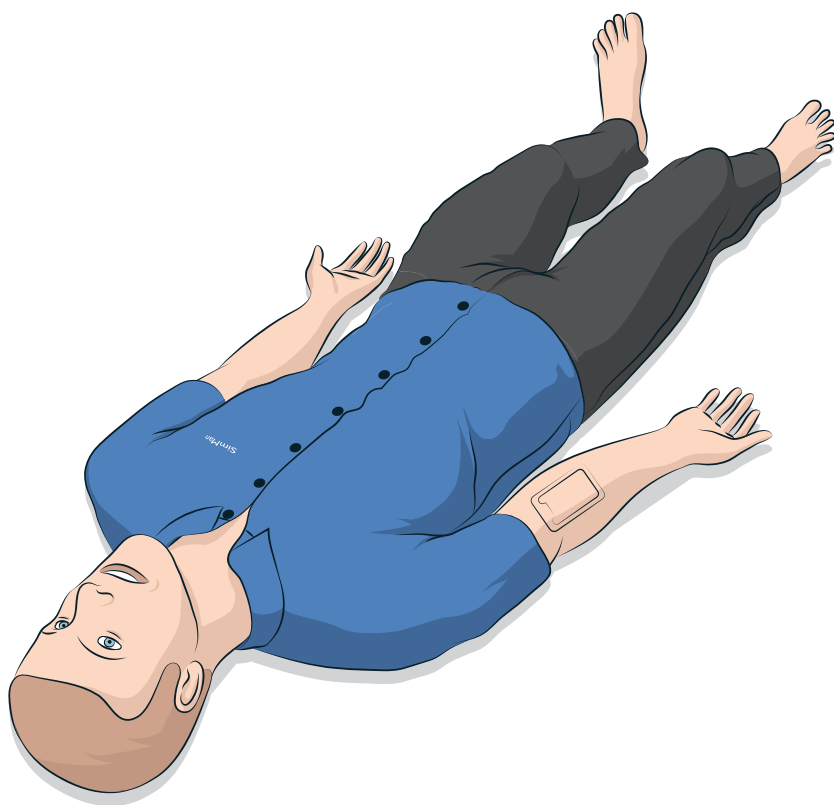




This Quick Setup Guide describes how to assemble and setup your SimMan patient simulator. For information about how to use SimMan, please refer to LLEAP helpfiles and SimMan User Guide.

Visit [www.laerdal.com/download](http://www.laerdal.com/download) to download the latest User Guide and software.




# COMPONENTS

Torso

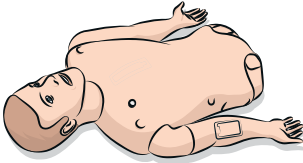

Legs

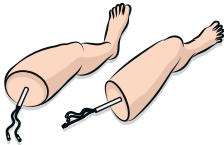


Clothes




Hard Upper Teeth



Male & Female Genitalia

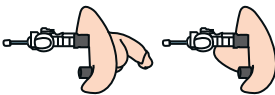

ID Band

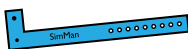

SimMan Arm Adapter



Syringe



Defib ECG Tightner



ECG Adapter Electrode



Neck Skin  
x6



Lung Compliance O-ring  
x 10



Fluid Refill Bottle



IV Arm Pads  
x3



Chest Drain Pleura  
x 6



---

Blood Pressure Cuff



Airway Lubricant



Eye Inserts Kit



IV bag



Cricothyroid Tape




Tibia Pads & Bone  
x3



IM Pads



Pneumothorax

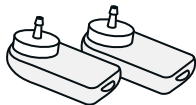

User Guide &  
Important Product Information



Headset



Keyboard & Mouse



SpO2 probe



Ethernet Cable



Allen Key



External Power Supply



Battery Charger  
x 2




1


## CHARGE PC BATTERIES



---

Patient Monitor

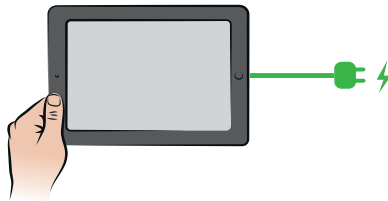




Insturctor PC

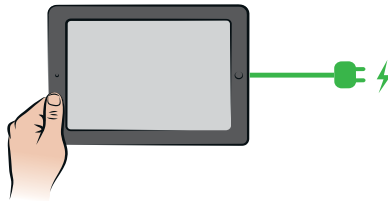


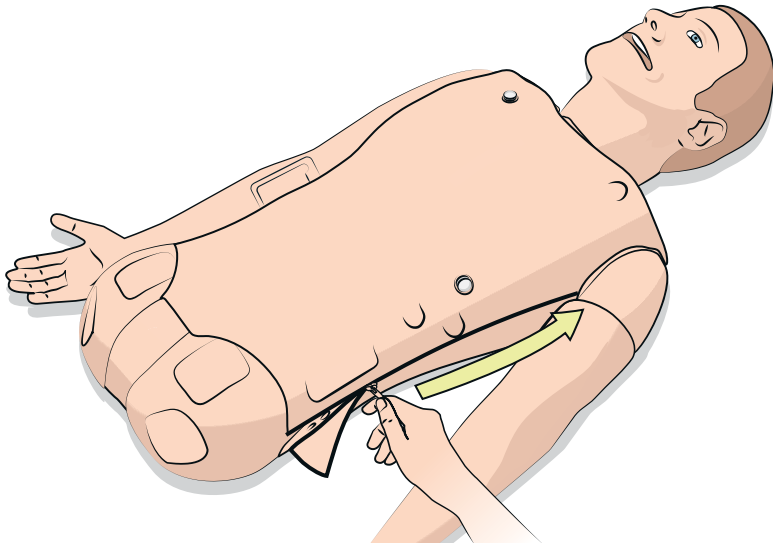
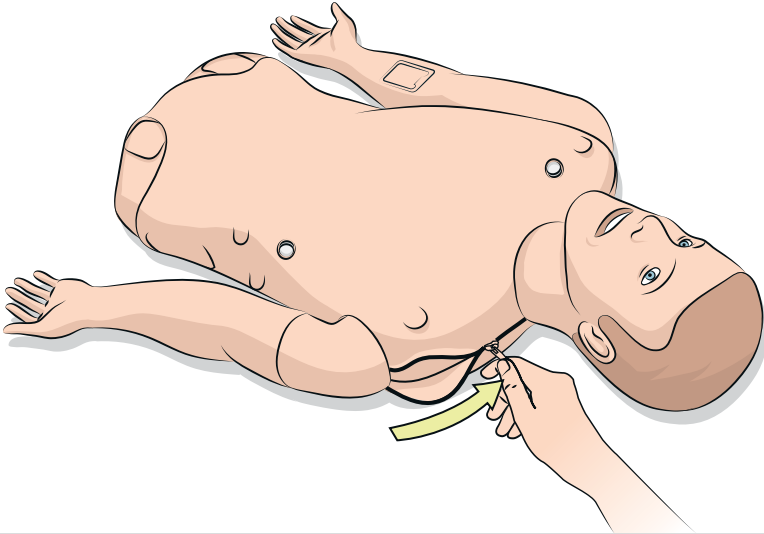


3

## CONNECT BATTERY

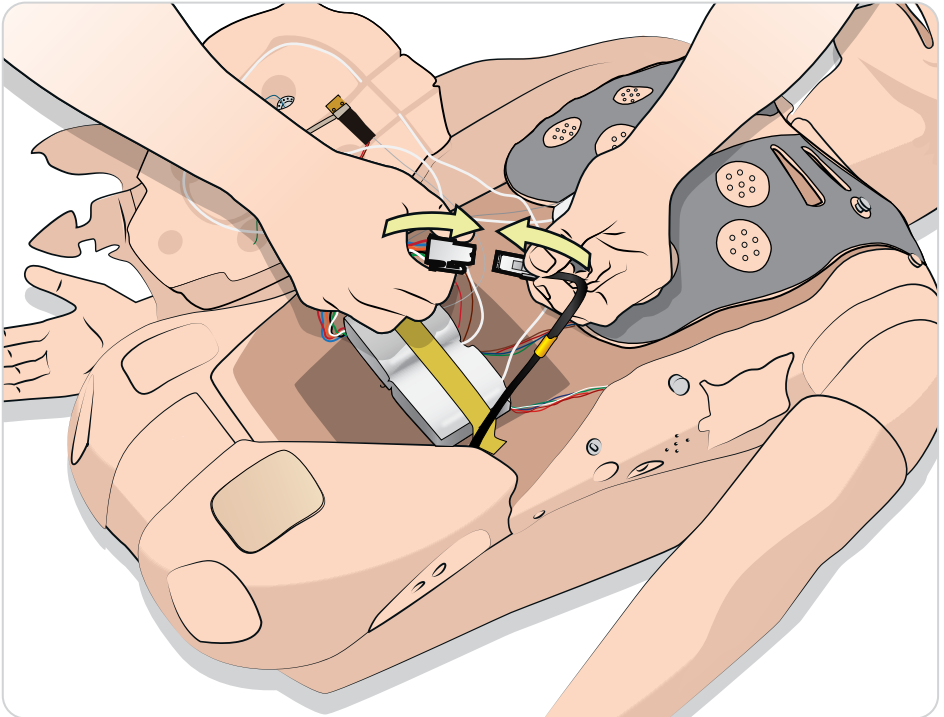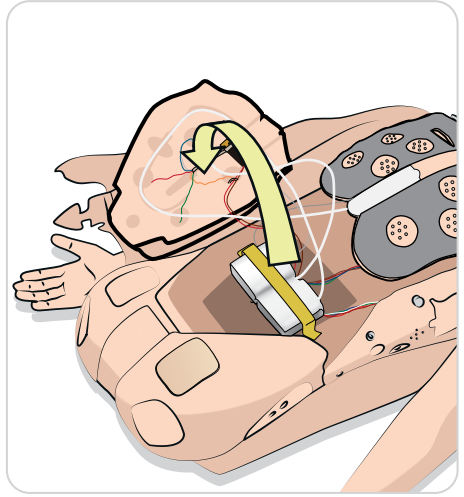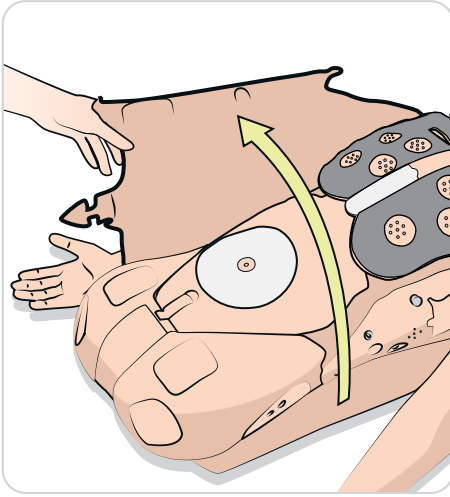







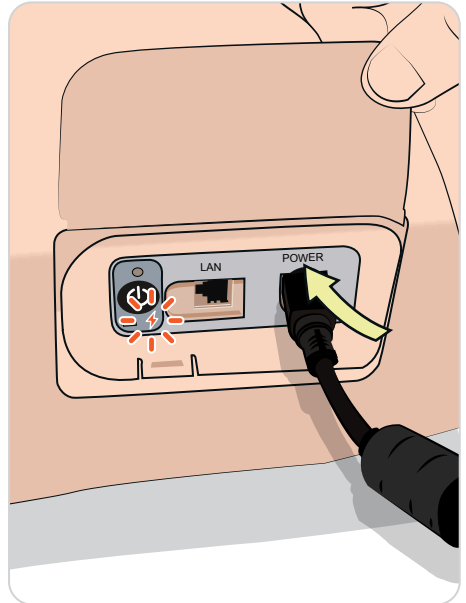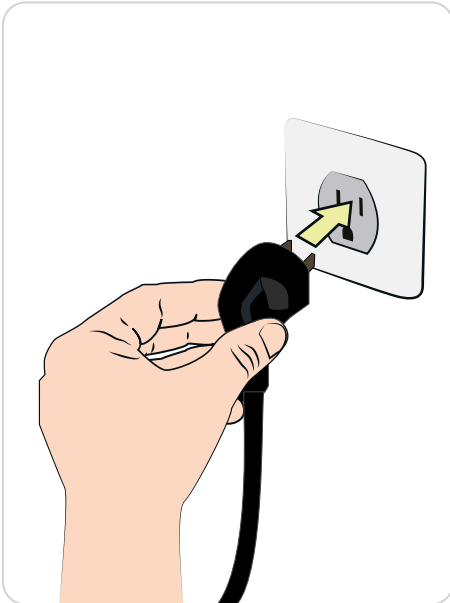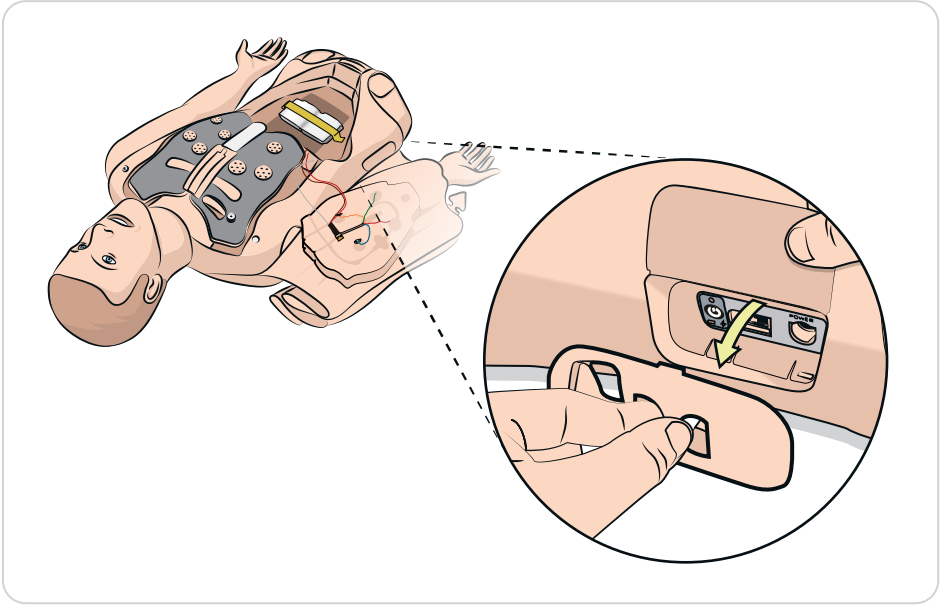

5


ATTACH LEFT LEG



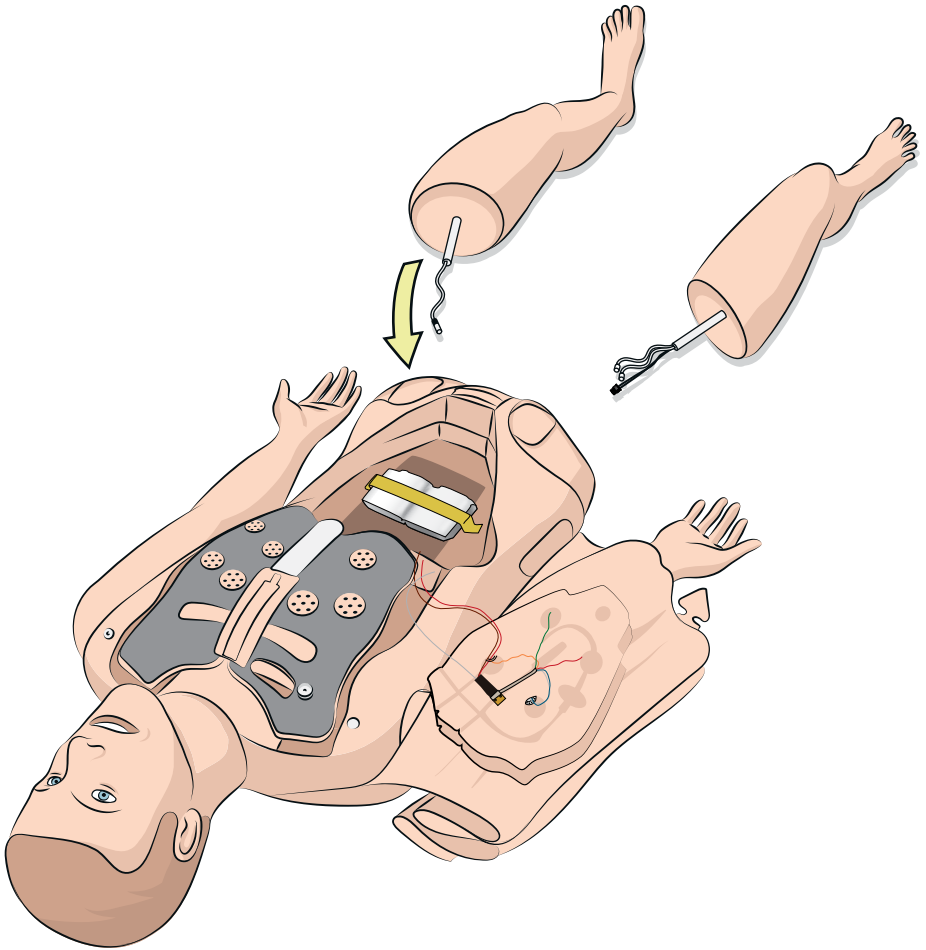


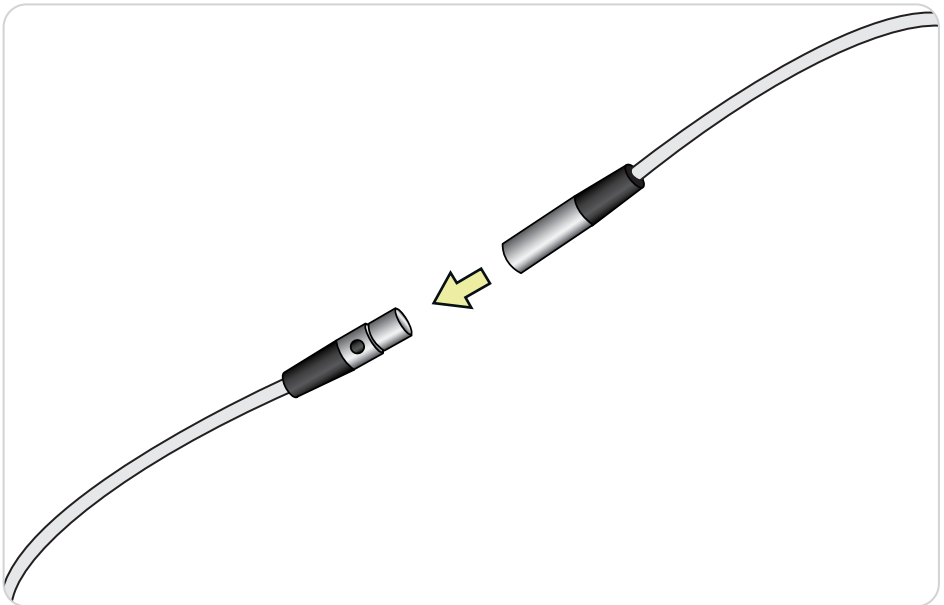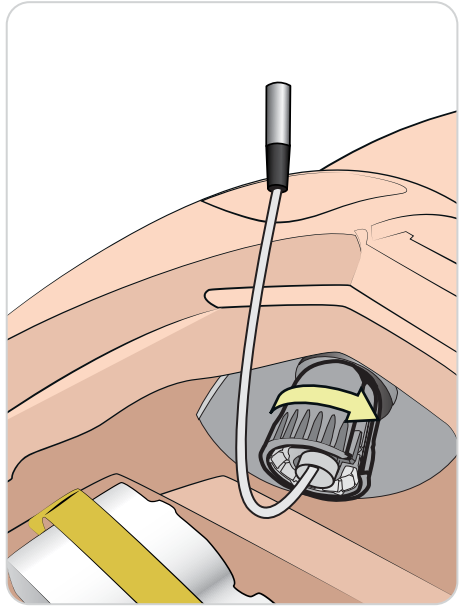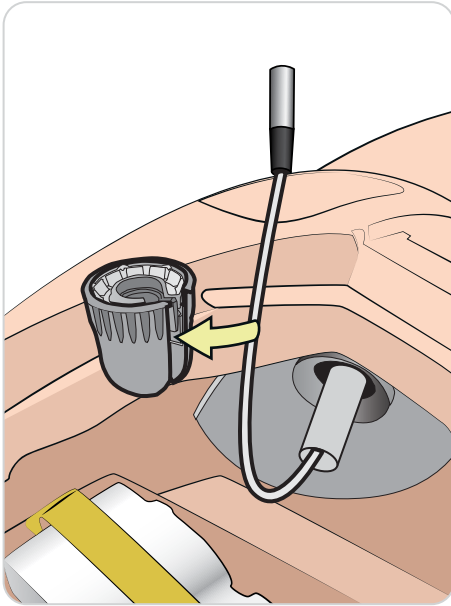

6

ATTACH RIGHT LEG

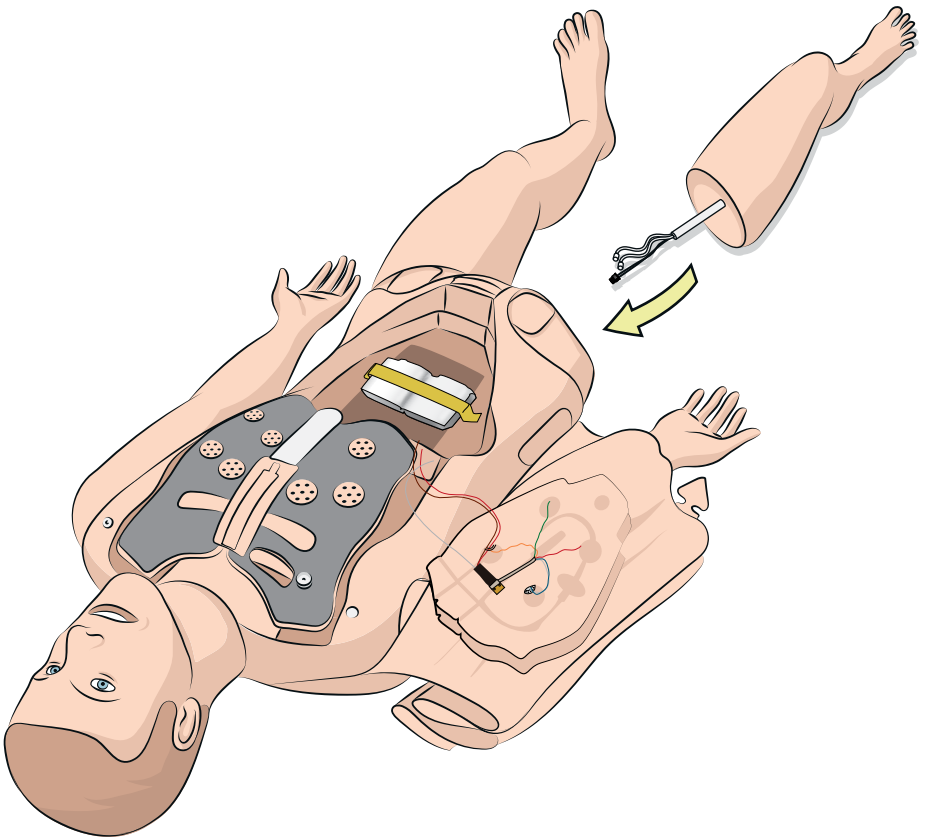

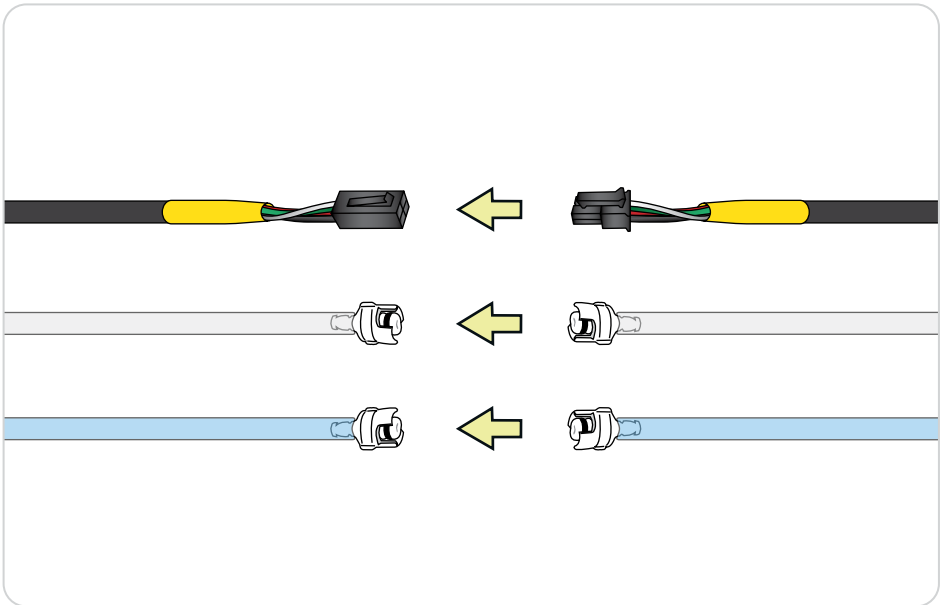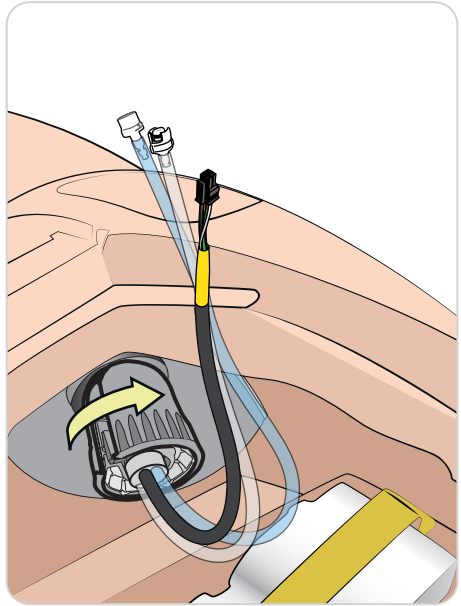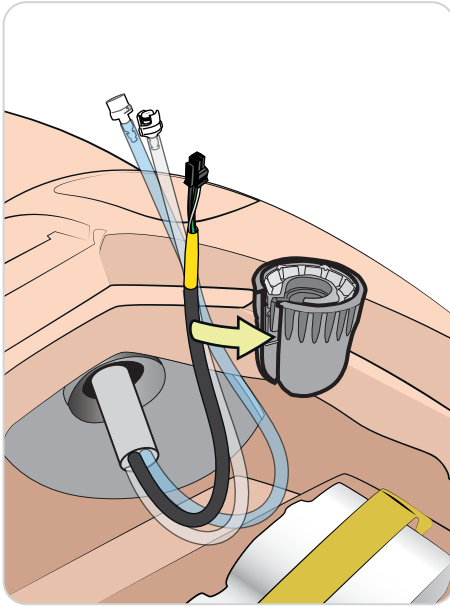


# 7<sub>A</sub>

## WIRELESS SETUP

Headset




Instructor PC

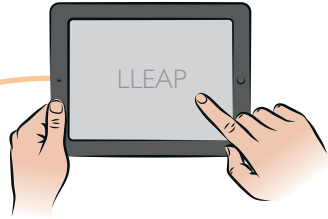


SpO2



Patient Monitor

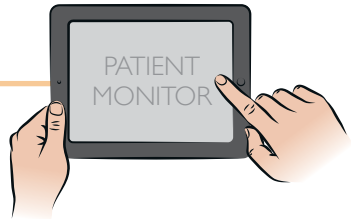

USB cable —

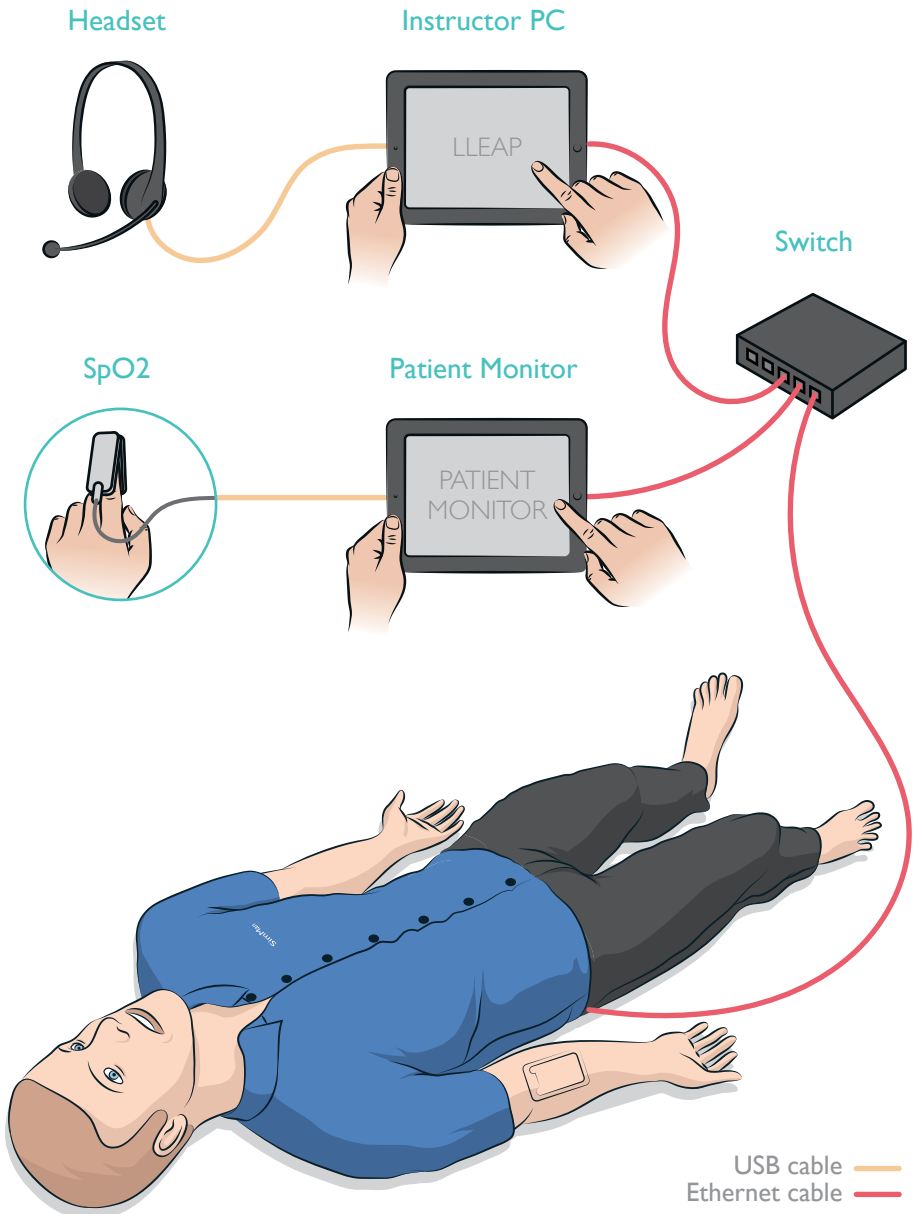

© 2014 Laerdal Medical AS. All rights reserved.  
Device Manufacturer: Laerdal Medical AS, P.O. Box 377,  
Tanke Svilandsgate 30, 4002 Stavanger, Norway  
T: (+47) 5151 1700

20-07892 Rev A

---

[www.laerdal.com](http://www.laerdal.com)



**Laerdal**  
helping save lives