


SimMan

Essential Bleeding

Quick Setup Guide

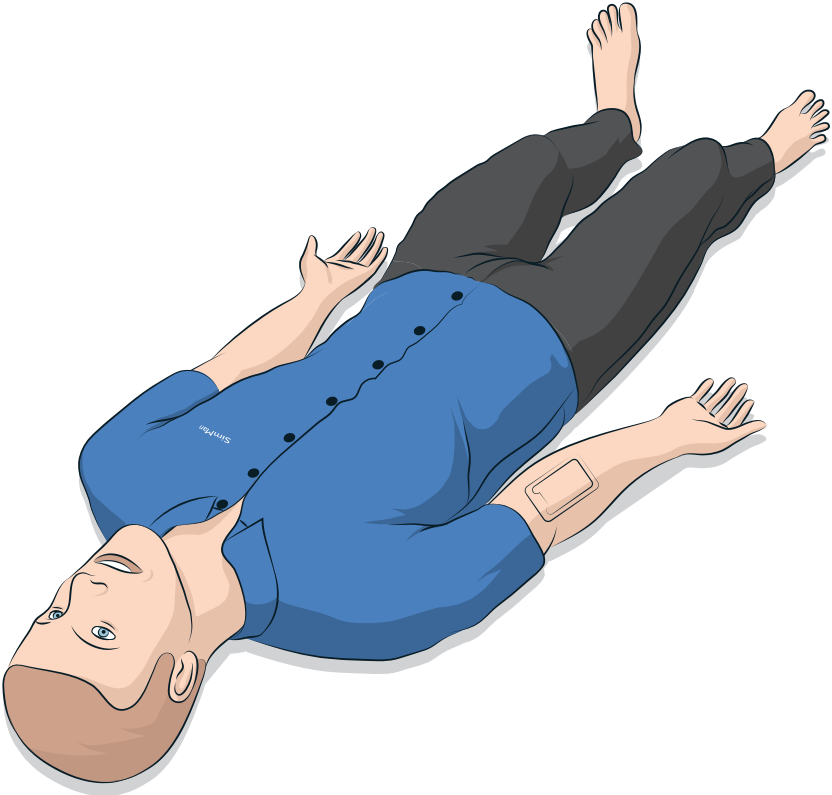

SIMMAN ESSENTIAL BLEEDING


This Quick Setup Guide describes how to assemble and setup your SimMan patient simulator. For information about how to use SimMan, please refer to LLEAP helpfiles and SimMan User Guide.

Visit www.laerdal.com/download to download the latest User Guide and software.




COMPONENTS

Torso

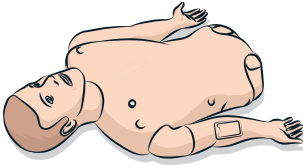

Legs

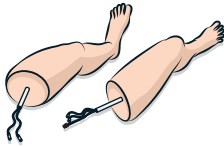


Clothes



Hard Upper Teeth



Male & Female Genitalia

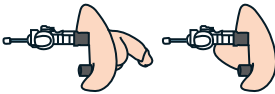


ID Band



SimMan Arm Adapter



Wounds Kit

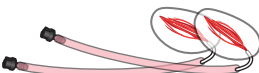

Syringe



Defib ECG Tightner




ECG Adapter Electrode



Neck Skin
x6



Inline Filter Blood
x 2

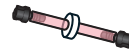

Lung Compliance O-ring
x 10




Blood Refill Bottle



IV Arm Pads
x3



Chest Drain Pleura
x 6

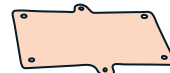

Blood Concentrate



Airway Lubricant



Eye Inserts Kit



IV bag



Cricothyroid Tape



IV Catheter Extension




Tibia Pads & Bone
x3



IM Pads



Pneumothorax

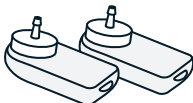

User Guide &
Important Product Information



Headset



Keyboard & Mouse



SpO2 probe



Ethernet Cable



Allen Key



External Power Supply



Blood Pressure Cuff




Battery Charger
x 2




1



CHARGE PC BATTERIES




Patient Monitor



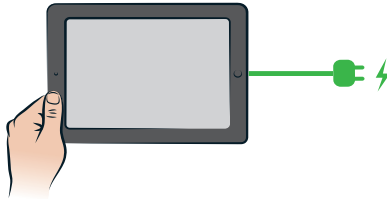



Insturctor PC



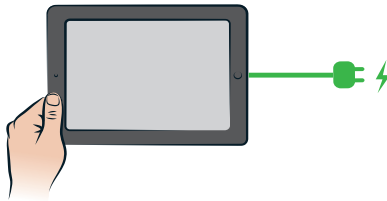


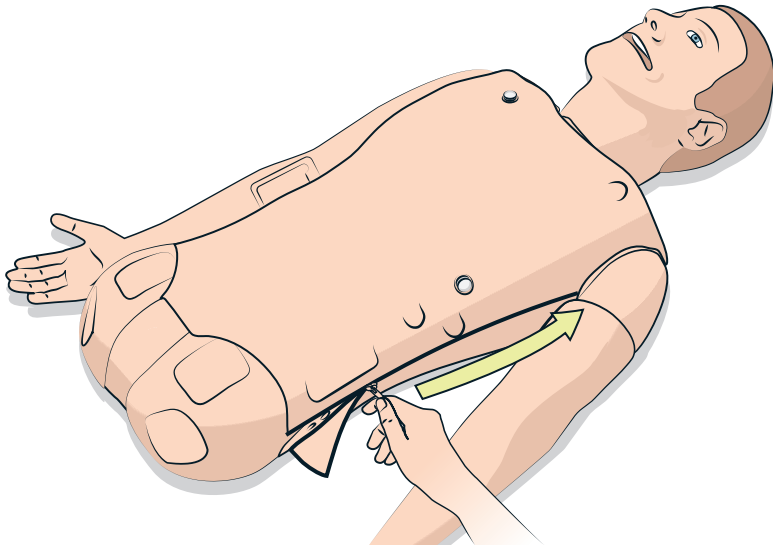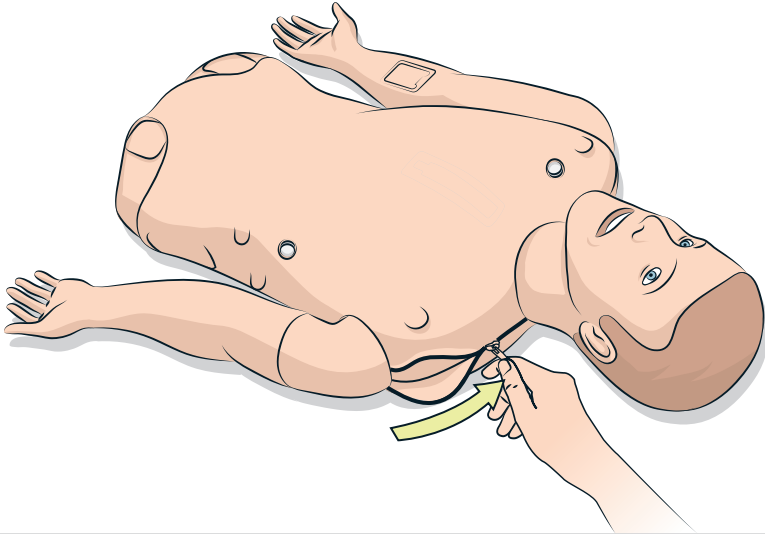

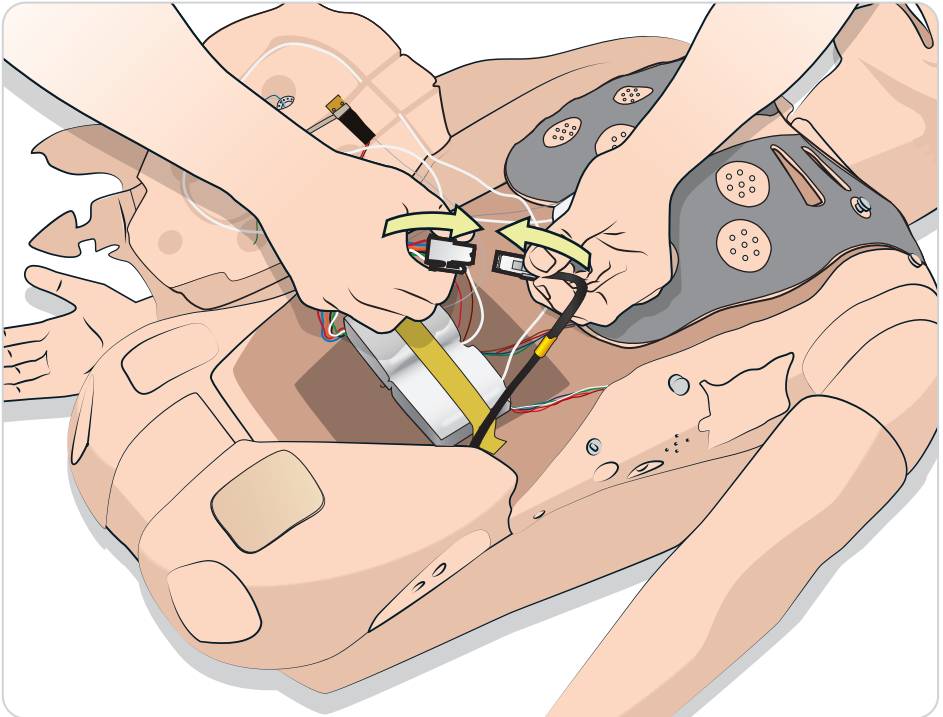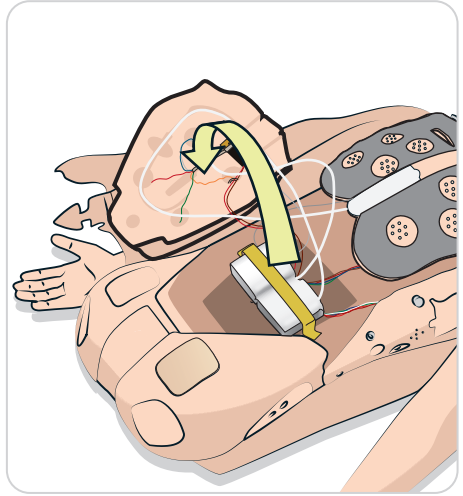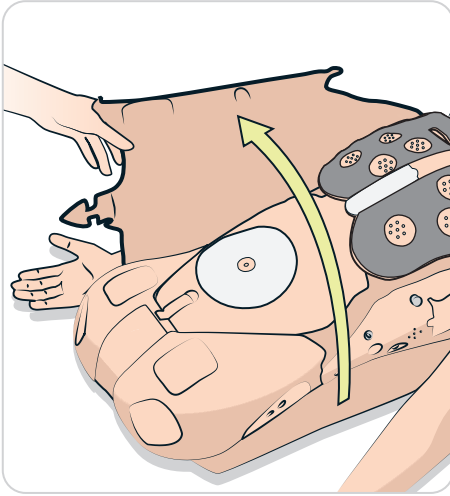

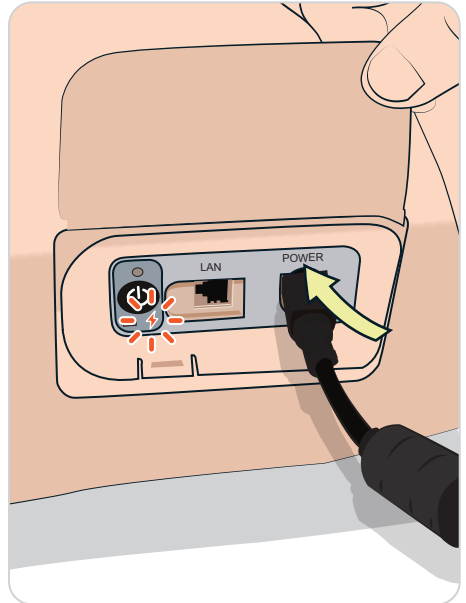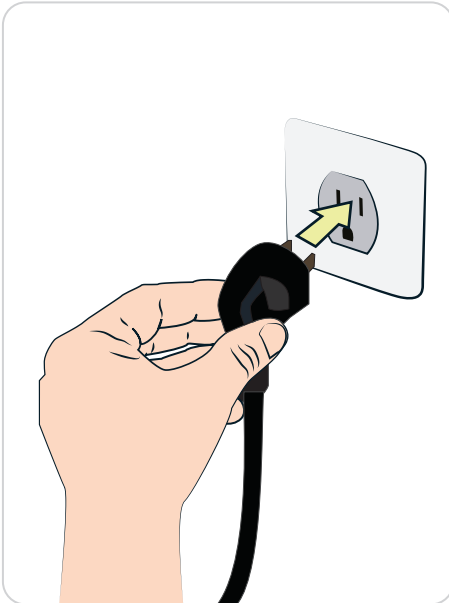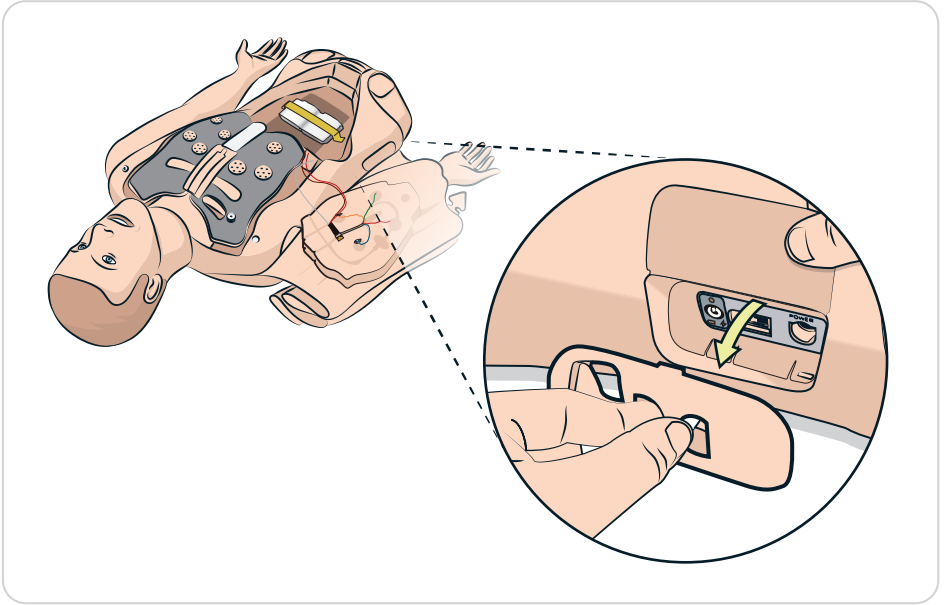




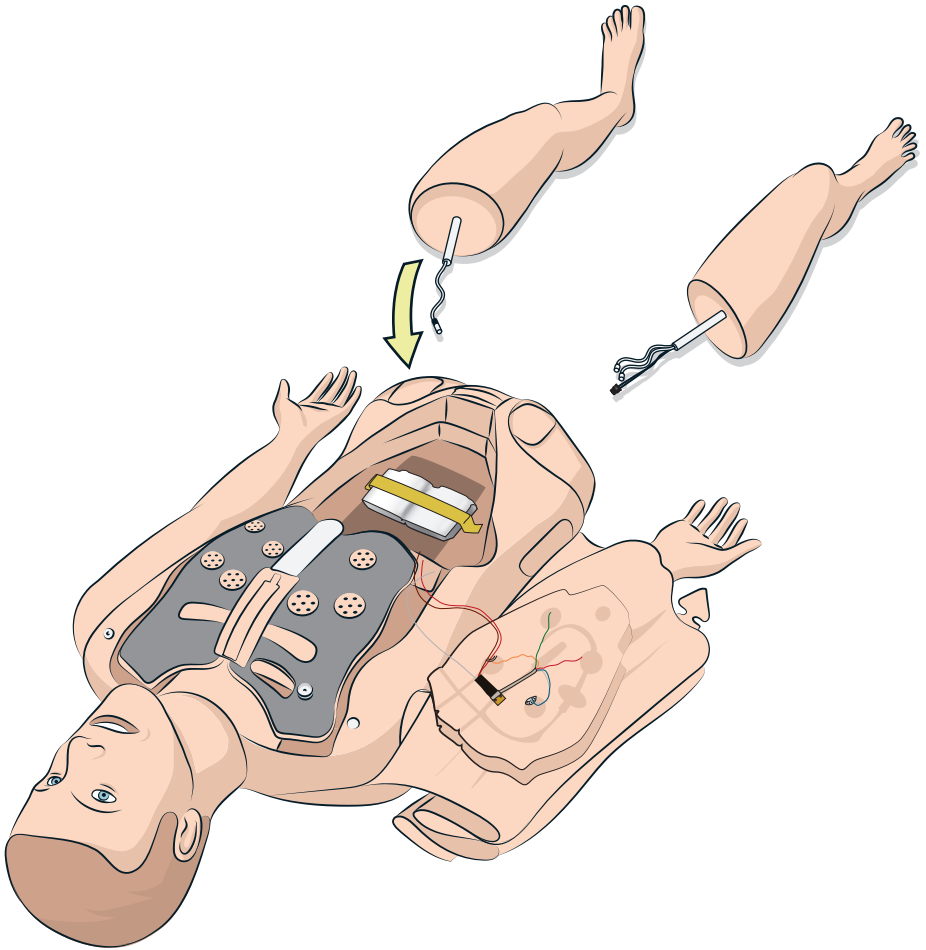


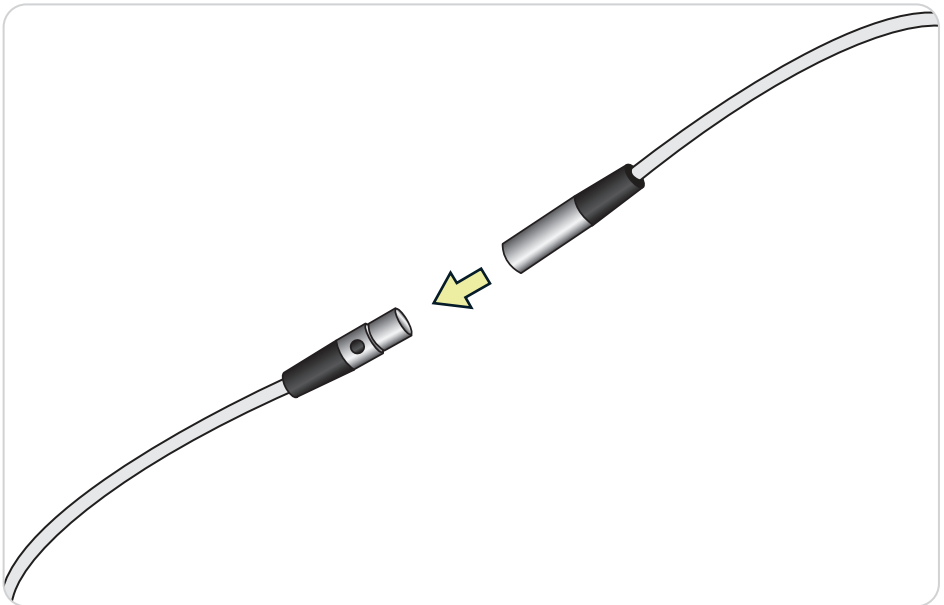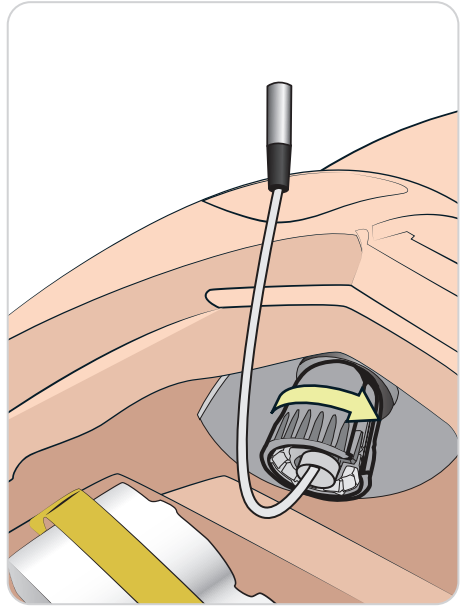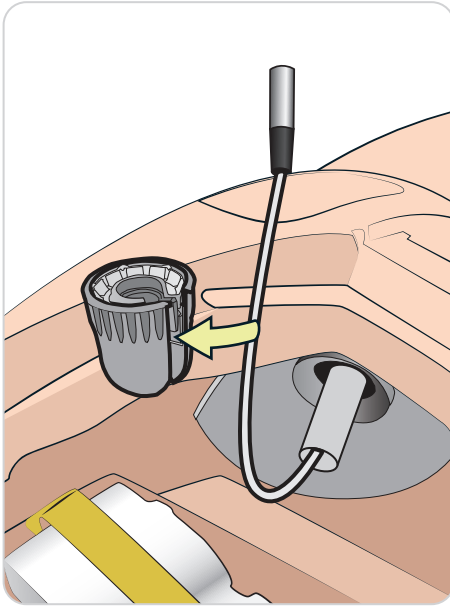

6

ATTACH RIGHT LEG

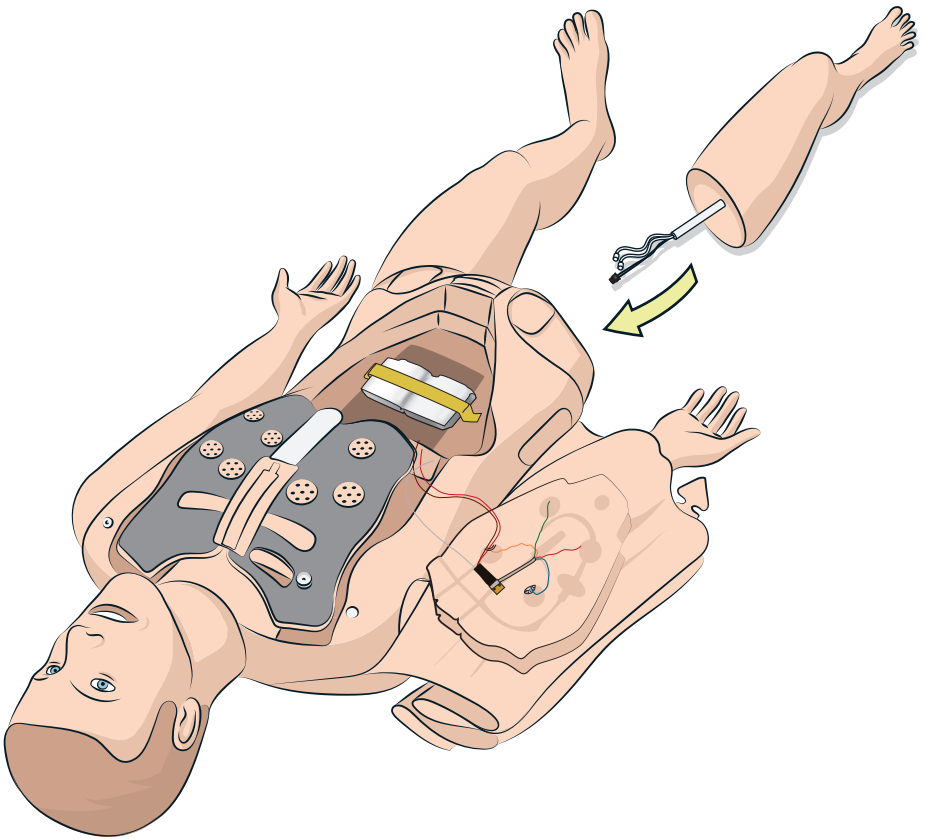

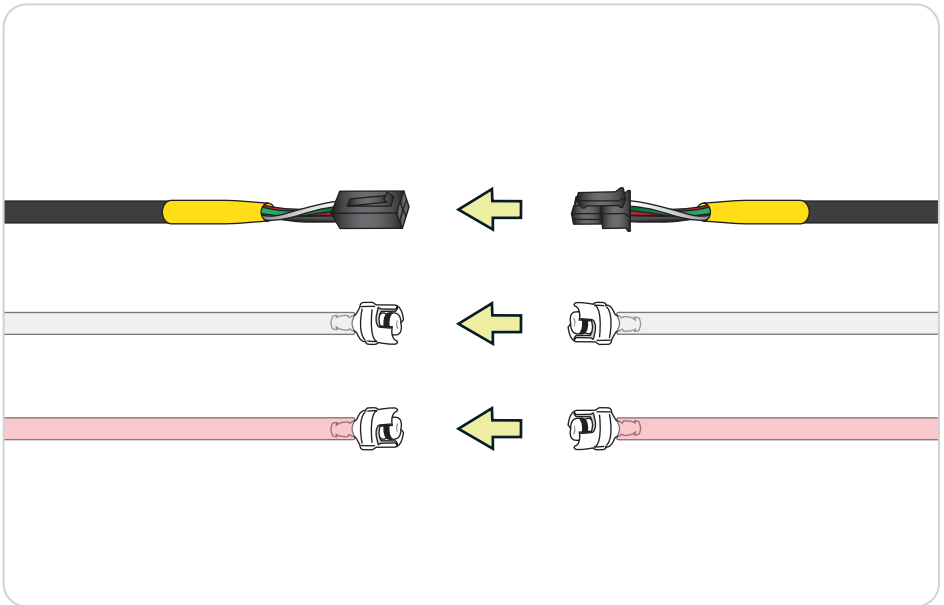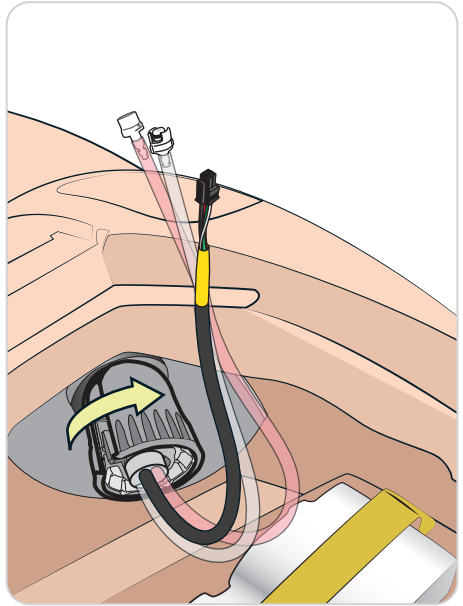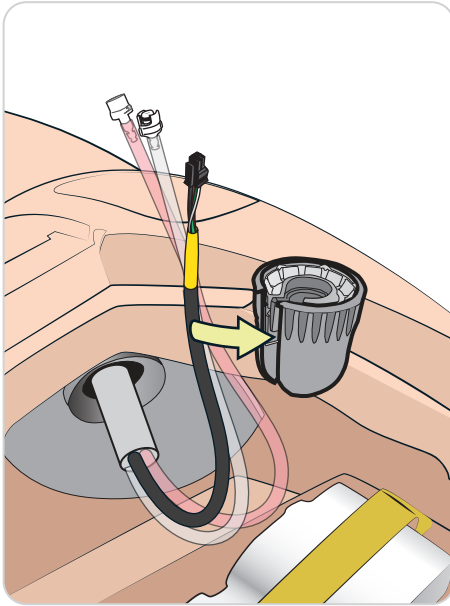


7_A

WIRELESS SETUP

Headset




Instructor PC

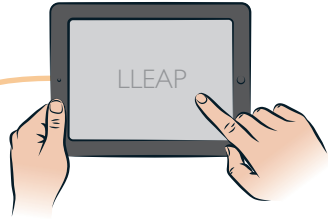


SpO2



Patient Monitor

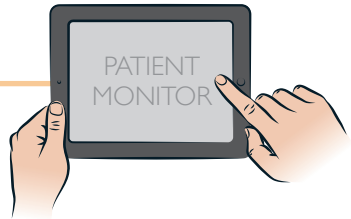

USB cable —

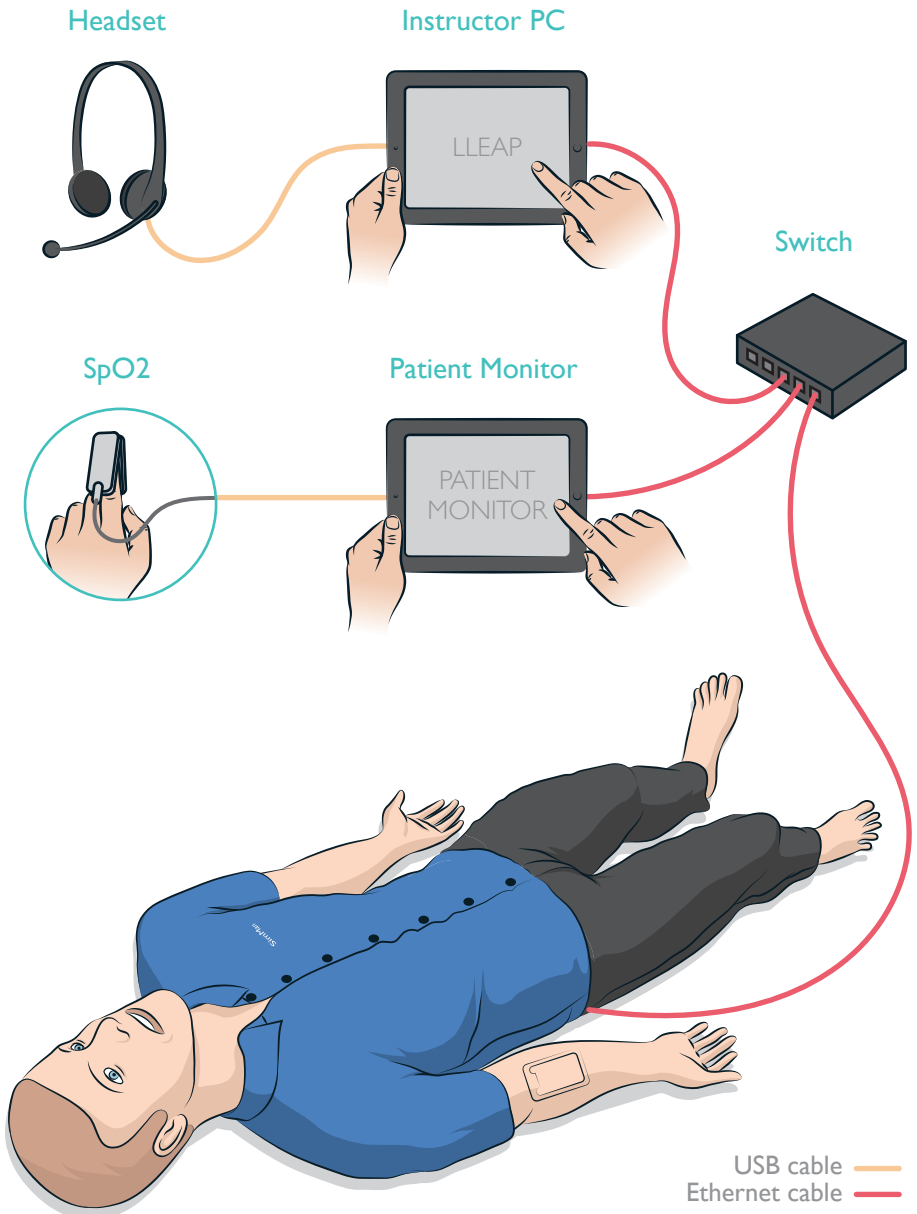

© 2016 Laerdal Medical AS. All rights reserved.
Printed in Norway

20-07895 Rev B

www.laerdal.com



Laerdal
helping save lives