

Fundus Skills and Assessment Task Trainer

Directions for Use



Table of Contents

Items Included	4
Features	4
Directions for Use	5
Care and Maintenance	5
Replacement Parts	5

The Fundus Skills and Assessment Task Trainer (375-22501) is designed for training in postpartum care.

The manufacturing excellence of this trainer should provide many sessions of training when reasonable care and maintenance is practiced.

Items Included:

- (1) Fundus Skill Assessment Trainer
- (1) Internal Components (Platform Fundus Stomach, Fundus Small Ball & Fundus Large Ball)
- (1) Genitalia
- (1) Simulated Blood Concentrate
- (5) Peri Pads
- (1) Talcum Powder
- (1) Laerdal Global Warranty Booklet
- (1) Directions for Use

Features:

- Normal anatomy of the postpartum female abdomen
- Landmark of the symphysis pubis
- Umbilicus
- Fundus offset secondary to full bladder
- Anal hemorrhoids visible
- 2nd degree midline episiotomy
- Simulates either firm or boggy fundus
- Fundus consistencies:
 - Approximately a 3” diameter ball to simulate a firm, well contracted uterus
 - Approximately a 4” ball to simulate a “boggy” uterus that is not contracted
- Genitalia elongated with swelling of clitoris area
- Vaginal orifice is visible and open
- Labium minus flapped to expose vaginal canal

Directions for Use:

1. Apply baby powder to inner skin.
2. Insert ball to simulate “firm” or “boggy” uterus.

Care and Maintenance:

1. Clean with mild soap and water; do not submerge modules or parts in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, manikin and module should be cleaned after each training session and a general inspection should be conducted regularly. A generous dusting of powder will assist in protecting modules and promoting longevity.
4. Store properly between teaching sessions.

Replacement Parts:

<u>Part Number:</u>	<u>Part:</u>
375-22650	Pelvis Skin
375-22750	Urinary Bladder
	Reservoir Assembly
375-22850	Female Fundus
	Genitalia Assembly
375-22950	Urinary Valve
	Connector
375-23050	Pubis Bone Pelvis Pin
375-21250	Right Stump
	w/Injection Pad
375-21150	Left Stump w/Injection
	Pad
325-00150	Ventral Gluteal Pad
325-00250	Dorsal Gluteal Pad
375-23150	Small Foam Fundus Ball
375-23250	Large Foam Fundus Ball
277-00150	Baby Powder
300-00750	Blood Concentrate
375-23350	Peri Pads (5)

Please contact Customer Service Representatives for more information on Replacement Parts in other skin tones.

(C) Copyright Laerdal Medical, Gatesville TX. All rights reserved
1004885 rev E

