


Sounds Trainer

Directions for Use

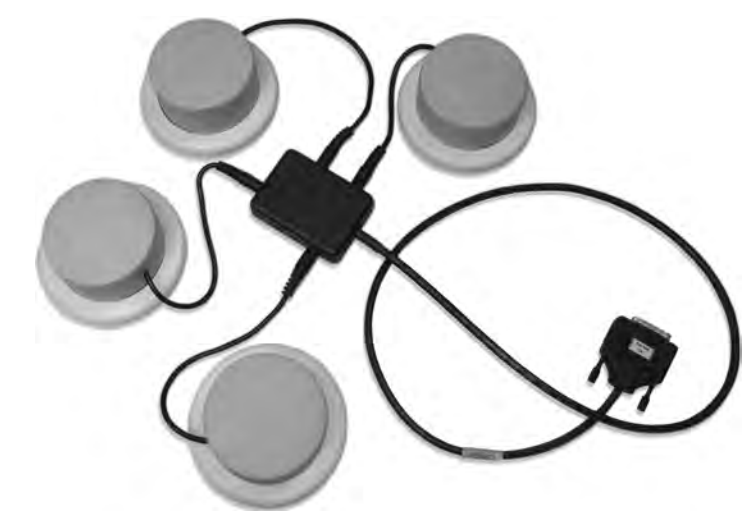

© 2015 Laerdal Medical AS. All rights reserved.
Manufacturer: Laerdal Medical Corporation
P.O. Box 38, 226 FM 116, Gatesville, Texas 76528 USA
T: +1 (254) 865-7221

Printed in USA



Rev C



Table of Contents

Laerdal Recommends..... 1

Items Included..... 1

Skills Taught..... 1

Auscultation of Heart, Breath & Bowel Sounds.....2

Replacement Parts.....2

Global Warranty.....2

The SimPad Sounds Trainer is an accessory to the SimPad System that enables the auscultation and recognition of normal and abnormal heart, breath and bowel sounds.

Laerdal Recommends

For use with standard stethoscope.

Items Included:

- (4) SimPad Sounds Trainer External Speaker
- (1) Connector Cable *
- (1) Carry Case

* Must be used with SimPad System (sold separately)

Skills Taught:

- Auscultation of Normal and Abnormal Heart, Breath & Bowel Sounds.

Sounds Include:

See SimPad DFU for complete list of available sounds.

Auscultation of Heart, Breath and Bowel Sounds-

1. Secure connector cable to back of SimPad Link Box. (Photo 1)



Photo 1

2. Attach each external speaker to the connector cable via attached cord.
(Photo 2)



Photo 2

3. Auscultate sounds by placing a stethoscope against the top surface of an external speaker

See SimPad Directions for Use for complete instructions and operation of the Heart, Breath and Bowel Sound features.

Replacement Parts

For latest version of Spare Parts and Accessories, visit www.laerdal.com

Global Warranty

See the Laerdal Global Warranty Booklet, or www.laerdal.com