

# Advanced Trauma Modules

## Directions for Use



**Laerdal**<sup>™</sup>  
*helping save lives*



---

---

# Table of Contents

Items Included	4
Skills Taught	4
Instructions for Use	5
Care and Maintenance	5
Replacement Parts	6

---

---

---

---

The Advanced Trauma Modules (276-00001) are specifically designed for training professionals in the practice of emergency care.

As a result of quality manufacturing and the ease of replacing individual parts, this kit should provide many sessions of training when reasonable care and maintenance is practiced.

---

---

**Items Included:**

- (1) Trauma Head
- (1) Industrial Hand with Forearm
- (1) Contused Ankle and Foot (Left Leg)
- (1) Crushed Foot with Amputation of the Small Toe (Right Foot)
- (1) Projectile Entry/Exit Wounds (Set of Four)
- (1) Open Fracture Radius
- (1) Seat Belt Contusion
- (1) Exposed Viscera
- (1) Impaled Object
- (1) Open Fracture Femur
- (1) Closed Fracture Tibia & Fibula (Right Leg)
- (1) Crushed foot
- (1) Laerdal Global Warranty Booklet
- (1) Directions for use

**Skill Taught:**

- Emergency Care Management

### **Instructions for Use:**

For installation procedures, please refer to the *Directions for Use* that came with your manikin.

### **Care and Maintenance:**

1. Clean with mild soap and water; do not submerge manikin or parts in cleaning fluids or water.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin and module should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.



#### **Cautions and Warnings**

This product contains Natural Rubber Latex which may cause allergic reactions when in contact with humans.

**Replacement Parts:**

For latest version of Spare Parts and Accessories, visit [www.laerdal.com](http://www.laerdal.com)

**Global Warranty**

See the Laerdal Global Warranty Booklet, or [www.laerdal.com](http://www.laerdal.com)





**1001701**

© 2015 Laerdal Medical. All rights reserved.  
Printed in USA  
1001701 Rev D



**Laerdal**<sup>™</sup>  
*helping save lives*