
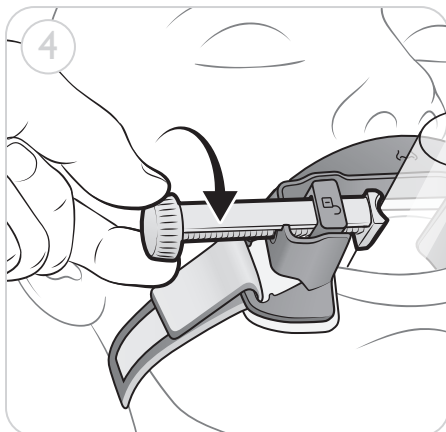
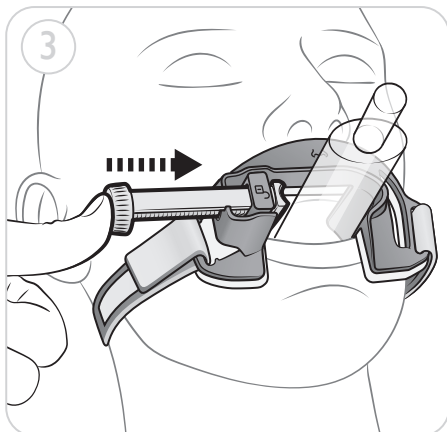
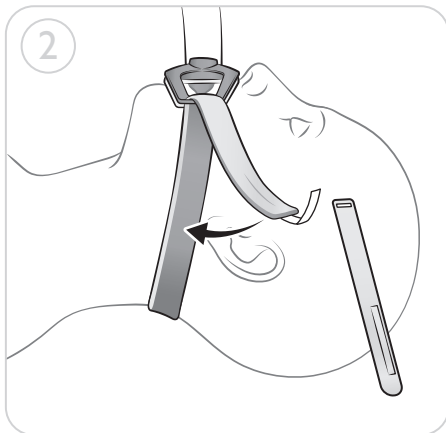
Thomas Select Tube Holder - Adult

User Guide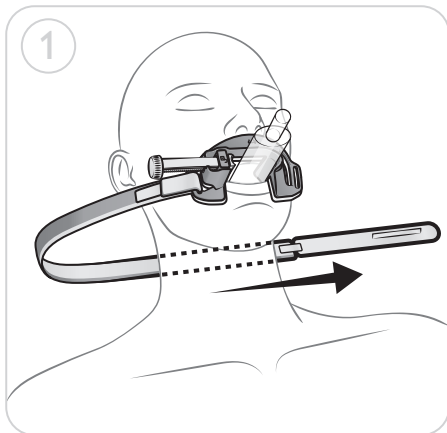

 Size 3 - 6

 < 8 years

 ONLY

REF 600-40000 (qty.1)

 LATEX

 ≥ 6.5 mm

 -34° C to +52° C
(-30° F to +124° F)

 CE



MD


Laerdal
helping save lives


A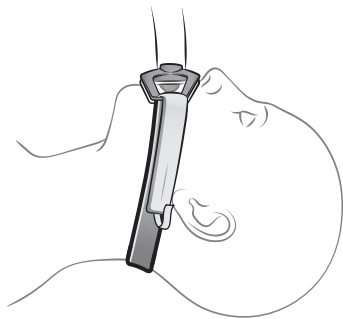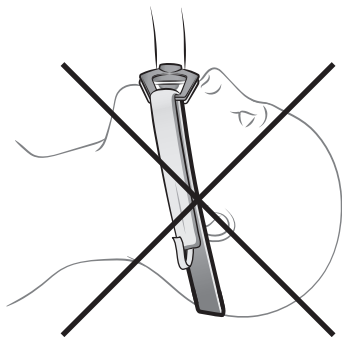

B

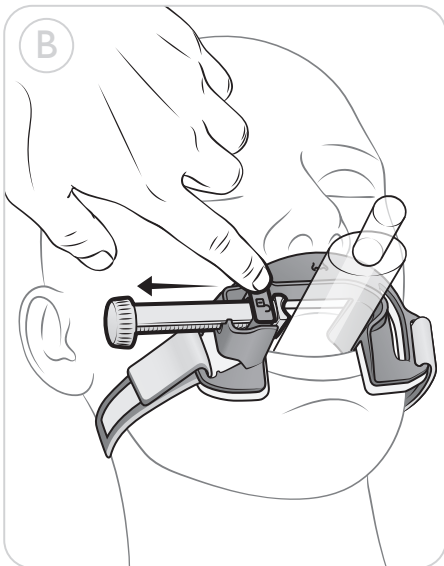C

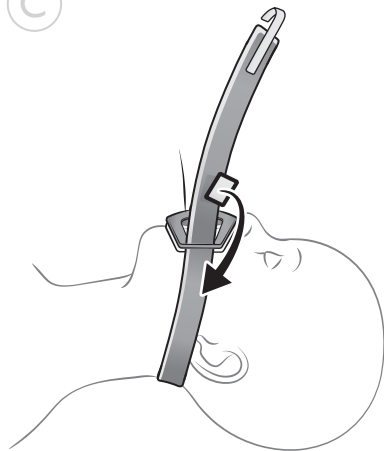Indications for use

Thomas Select Tube Holder is designed to secure advanced airway devices, e.g. Endotracheal Tubes (ETs), Laryngeal Mask Airways (LMAs) and other wider Supraglottic Airway devices (SGAs), after insertion into the trachea or esophagus to reduce the risk of accidental extubation. It is intended for use on persons of at least 8 years old with a short-term need (less than 24 hours) for airway-tube device management.

Setup Instructions

- Secure the product as shown (Fig. 1 - 4).
- Ensure correct position of head strap to prevent it from sliding (Fig.A).
- Press the unlock button for quick release of the tube clamp (Fig.B).
- If necessary, position removable hook tab on strap to accommodate smaller neck size (Fig. C).

Notes

Important information about the product or its operation.

- The Thomas Select Tube Holder meets the following Standards: EN 1789, ISO 13485, ISO 14971, ISO 10993-1.
- The Thomas Select Tube Holder fits most advanced airway management devices but always check size compatibility prior to implementation.

Warnings

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

- This device should only be used by trained personnel in accordance with the User Guide.
- Apply just enough pressure against the tube to hold securely in place. Overtightening of the screw may restrict airflow.
- Do not use on children younger than 8 years old.
- Designed for single patient use only. Do not re-use. Re-use will lead to increased risk of cross-contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.
- Prior to use, inspect all parts to ensure they are free of damage and working properly.
- The patient should be constantly monitored while this product is in use.

Cautions

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the Thomas Select Tube Holder:

- The device should only be attached to the patient after insertion of an advanced airway device.
- Do not use the device on a patient if it has been used during training.
- Do not disinfect or sterilize the product.
- The device is for short term use only. Remove after 24 hours.
- Storage temperature: -34 °C to +52 °C (-30 °F to 124 °F).
- Operating temperature: -18 °C to +43 °C (-0 °F to 109°F).

Symbol Glossary

	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.
	Caution: Federal law (US) restricts this device to sale by or on the order of a physician or appropriately licensed practitioner.
	Single Patient Use. Do not re-use.
	Reference order and number
	Unique Device Identification
	Medical Device
	Not for use on children under 8 years old
	Storage temperature (Maximum and Minimum)
	Size range for supraglottic devices
	Size range for tracheal tubes (Inner Diameter)
	Unlock button for quick release of clamp
	Symbol indicates placement under the nose
	Not made with natural rubber latex

Materials

Part	Material
Foam	Polyethylene foam - PE
Strap	Polyurethane foam - PU Polyester flannel - PET Polyamide - PA
Other parts	Polypropylene - PP

The materials have not been tested for flammability.

Dispose of according to local protocol.

Thomas Select Tube Holder is protected by US patent # 9,707,364 and # 10,046,130 and other international patents. It is also protected by US Design patent # D834,185 and other international design registrations. Thomas and Laerdal are trademarks owned by Laerdal Medical AS. Ownership and all rights reserved.

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© 2022 Laerdal Medical AS. All rights reserved.