Resusci Baby

QCPR

Important Product Information
重要产品信息

EN

FR

DE

ES

IT

BR

NL

NO

SE

DA

FI

JA

ZH

KO

RU

PL

www.laerdal.com

Laerdal
helping save lives

English

Resusci Baby QCPR

Read these instructions thoroughly. Observe all warnings, precautions and instructions on the product, in the User Guide and in this Important Product Information booklet.
Retain this booklet for future reference.

Warnings and Cautions

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the product.

Note

A note states important information about the product or its operation.

General

Caution

Use of automatic chest compression machines may damage the manikin.

Care, Maintenance and Cleaning

For instructions on cleaning Resusci Baby faces, refer to the Cleaning Advice for Laerdal manikin faces insert.

- Keep the manikin clean and in a hygienic condition.
- Replace the airway assembly after every class, especially if mouth to mouth resuscitation is practiced.
- We recommend that you use a separate Laerdal Manikin Face for every student.
- If several students use one manikin face, thoroughly sanitize the manikin face after every use.
- Clean all skin parts regularly. Use warm soapy water or Laerdal manikin wipes.

These can discolor the manikin:

- Pigments from lipstick and pens
- Latex gloves
- Using clothes other than Resusci Baby clothing.

Federal Communications Commission (FCC) and Industry Canada (IC) Statements

This device complies with part 15 of the FCC Rules and RSS-210 of IC rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Changes or modifications not expressly approved by Laerdal Medical could void the user's authority to operate the equipment.

The term "IC" before the equipment certification number only signifies that the Industry Canada technical specifications were met.

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This device contains licence-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

1. This device may not cause interference.
2. This device must accept any interference, including interference that may cause undesired operation of the device.

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Resusci Baby Q CPR Contains
FCC ID: QHQ 20-10494
IC 20263-2010494

Canada

This Class B digital apparatus complies with Canadian ICES-003.

English

Resusci Baby QCPR

UK

UKCA (U.K. Conformity Assessed): This product complies with the requirements of U.K. legislation of Electromagnetic Compatibility Regulations (S.I. 2016/1101).

UKCA (U.K. Conformity Assessed): This product complies with the requirements of U.K. legislation of Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment Regulations (RoHS) Regulations (S.I. 2012/3032).

EU

This product is in compliance with the essential requirements of Council Directive 2014/30/EU on electromagnetic compatibility (EMC) and Council Directive 2011/65/EU on Restriction on the use of certain hazardous substance (RoHS).

Waste Handling

The manikin contains electronic components. Dispose of it at the applicable recycling facility in accordance with local regulations.

European Directive 2012/19/EU (WEEE)

WEEE: this appliance is marked according to the European directive 2012/19/EU on Waste Electrical and Electronic Equipment (WEEE). By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

The symbol on the product, or on the documents accompanying the product, indicates that this appliance may not be treated as household waste. Instead it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment. Disposal must be carried out in accordance with local environmental regulations for waste disposal. For more detailed information about treatment, recovery and recycling of this product, please contact your local city office, your household waste disposal service or the Laerdal representative where you purchased the product.

Certification, Compliance and Labels

	CE Mark
	UKCA Mark
	Australian EMC Compliance Mark
	Manufacturer
	Date of Manufacture
	WEEE Symbol
	Reference order number

Specifications

Resusci Baby QCPR	
Dimensions	23.2" x 10.2" x 5.5" (59 cm x 26 cm x 13.5 cm)
Weight	4.5 lbs (2.05 kg)
Operation temperature	32°F to 104°F (0°C to +40°C)
Humidity	< 95% relative humidity
Storage temperature	5°F to +122°F (-15°C to +50°C)

Warranty

The Laerdal Resusci Baby QCPR has a two-year limited Warranty. Refer to the Laerdal Global Warranty for terms and conditions. www.laerdal.com

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© 2022 Laerdal Medical AS. All rights reserved.