# THE BAG II

## Laerdal Disposable PEEP Valve Assembly For use with all models of Laerdal brand resuscitators

### Directions for Use

**REF** Cat. No. 845140      **QTY** 1

**REF** Cat. No. 845240      **QTY** 10

**LOT**

Assembly Package Contains:

One – 30mm OD Expiration Diverter (Fig 1, item 1)

One – Disposable PEEP Valve – 30 mm ID (Fig 1, item 2)

### PEEP Valve Assembly

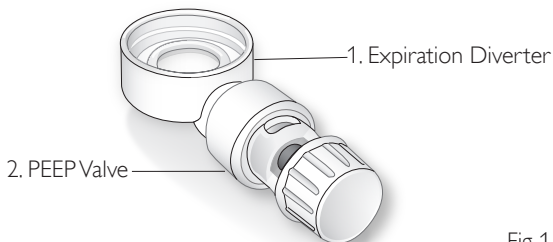
Fig 1

Single Patient Use - Non Sterile - Do Not Sterilize

# CONTENTS

---

English	Directions for Use	3
Deutsch	Bedienungsanleitung	6
Français	Mode d'emploi	9
Nederlands	Gebruiksaanwijzing	12
Italiano	Istruzioni per l'uso	15
Español	Instrucciones	18
Português	Instruções de Utilização	21
Norsk	Bruksanvisning	24
Svenska	Bruksanvisning	27
Dansk	Brugervejledning	30
Suomi	Käyttöohje	33
Polska	Instrukcja obsługi	36
简体中文	使用说明	39
한국어	사용설명서	42
日本語	取扱説明書	45

## The PEEP Valve

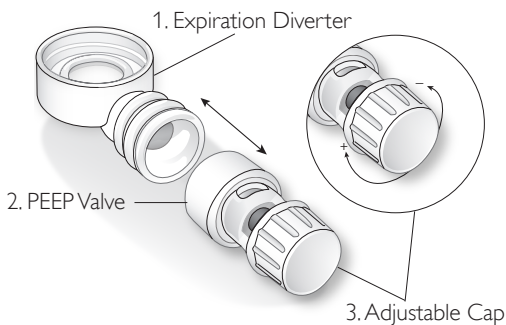
Fig 2

## Laerdal Resuscitator Assembly

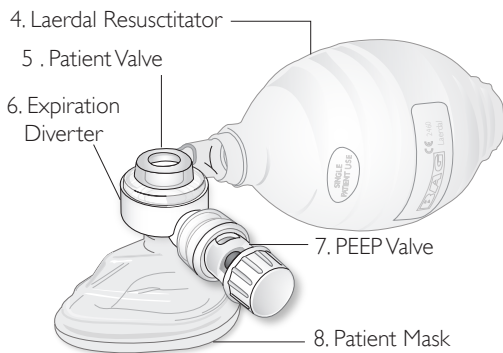
Fig 3

## Intended Use

The Laerdal Disposable PEEP Valve Assembly is a single patient use accessory for use with all models of Laerdal brand resuscitators to enable clinicians to apply and adjust the level of PEEP therapy being administered to a patient. The PEEP Valve connector is the standard 30 mm ID size.

## Operating Instructions

1. Upon removal from package, inspect to ensure the assembly is complete (items 1 and 2) and undamaged, and that the Adjustable Cap (3) moves easily and will hold its position.
2. Connect PEEP Valve to the resuscitator. Ensure that items 1 and 2 are connected firmly, then press the Expiration Diverter (1) onto the Laerdal resuscitator patient valve.
3. Turn Adjustable Cap (3) so that its bottom edge is set to the desired PEEP level setting.

### Operating Environmental Limits:

-10 °C to 40 °C (14 °F to 104 °F)

### Storage Environmental Limits:

-40 °C to 60 °C (-40 °F to 140 °F)

## Precautions

- intended for use only with Laerdal brand resuscitators.
- PEEP Valves should only be used by qualified persons who have received adequate training.
- A clinician must evaluate the risk/benefit of PEEP for each patient prior to its application.
- Markings of PEEP Valve settings are approximate.
- Patient's condition should be monitored continuously throughout the use of this product

## Symbol Glossary

Unique product reference.

Caution: Rx only. Federal law (USA) restricts this device to sale by or on the order of a physician.

Do not reuse.

Product is not made with Natural Rubber Latex

The product is in compliance with the essential requirements of Council Directive 93/42/EEC as amended by Council Directive 2007/47/EC.

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All rights reserved.
and “helping save lives” are all trademarks
Laerdal Medical Corporation
Laerdal Medical Canada, Ltd.