THE BAG II

Laerdal Disposable PEEP Valve Assembly For use with all models of Laerdal brand resuscitators

Directions for Use

REF Cat. No. 845140 **QTY** 1

REF Cat. No. 845240 **QTY** 10

LOT

Assembly Package Contains:

One – 30mm OD Expiration Diverter (Fig 1, item 1)

One – Disposable PEEP Valve – 30 mm ID (Fig 1, item 2)

PEEP Valve Assembly

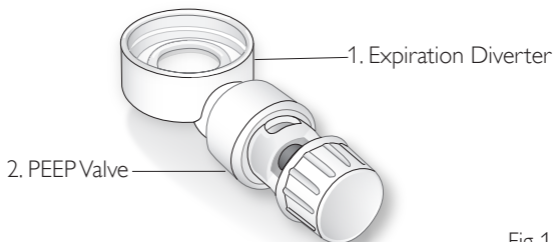
Fig 1

Single Patient Use - Non Sterile - Do Not Sterilize

CONTENTS

English	Directions for Use	3
Français	Mode d'emploi	6
Español	Instrucciones	9
Português	Instruções de Utilização	12
简体中文	使用说明	15
日本語	取扱説明書	18

The PEEP Valve

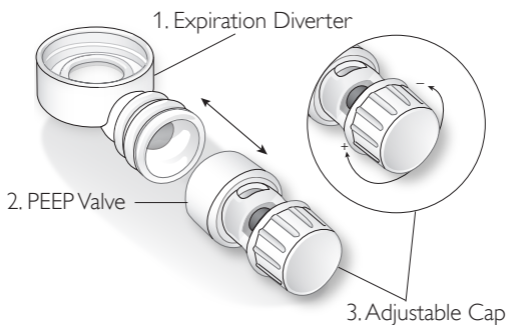
Fig 2

Laerdal Resuscitator Assembly

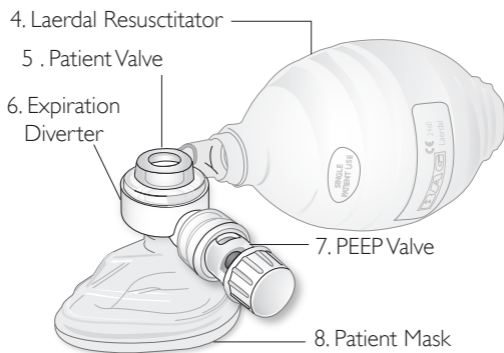
Fig 3

Intended Use

The Laerdal Disposable PEEP Valve Assembly is a single patient use accessory for use with all models of Laerdal brand resuscitators to enable clinicians to apply and adjust the level of PEEP therapy being administered to a patient. The PEEP Valve connector is the standard 30 mm ID size.

Operating Instructions

1. Upon removal from package, inspect to ensure the assembly is complete (items 1 and 2) and undamaged, and that the Adjustable Cap (3) moves easily and will hold its position.
2. Connect PEEP Valve to the resuscitator. Ensure that items 1 and 2 are connected firmly, then press the Expiration Diverter (1) onto the Laerdal resuscitator patient valve.
3. Turn Adjustable Cap (3) so that its bottom edge is set to the desired PEEP level setting.

Operating Environmental Limits:

-10 °C to 40 °C (14 °F to 104 °F)

Storage Environmental Limits:

-40 °C to 60 °C (-40 °F to 140 °F)

Precautions

- intended for use only with Laerdal brand resuscitators.
- PEEP Valves should only be used by qualified persons who have received adequate training.
- A clinician must evaluate the risk/benefit of PEEP for each patient prior to its application.
- Markings of PEEP Valve settings are approximate.
- Patient's condition should be monitored continuously throughout the use of this product

Symbol Glossary

REF

Unique product reference.

R_x Only USA

Caution: Rx only. Federal law (USA) restricts this device to sale by or on the order of a physician.

Do not reuse.

Product is not made with Natural Rubber Latex

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All rights reserved.
and “helping save lives” are all trademarks