# Resusci Anne

for

## First Aid

Important Product Information

# Resusci Baby

for

## First Aid

Important Product Information

**Laerdal**  
helping save lives

English

## Resusci Anne First Aid / Resusci Baby First Aid

### Notes

Important information about the product or its operation.

### Care, Maintenance and Cleaning

For instructions on cleaning Resusci Anne and Resusci Baby faces, refer to the Cleaning Advice for Laerdal manikin faces insert. Keep the manikin clean and in a hygienic condition.

- Replace the airway assembly after every class, especially if mouth to mouth resuscitation is practiced.
- We recommend that you use a separate Laerdal Manikin Face for every student.
- If several students use one manikin face, thoroughly sanitize the manikin face after every use.
- Clean all skin parts regularly. Use warm soapy water or Laerdal manikin wipes.
- Use of Manikin Face Shields provides a clean barrier between your lips and the manikin face.

These can discolor the manikin:

- Pigments from lipstick and pens
- Latex gloves
- Using clothes other than Resusci Anne and Resusci Baby clothing.

### Cautions

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the manikin.

#### Caution Resusci Anne First Aid

- Use of automatic chest compression machines may damage the manikin.

### Certification, Compliance and Labels

	Manufacturer
	Date of Manufacture
	Reference order number
	Warning / Caution symbol

### Specifications

Operation temperature: 32°F to 104°F (0°C to +40°C)

Humidity: < 95% relative humidity

Storage temperature: 5°F to +122°F (-15°C to +50°C)

#### Dimensions Resusci Anne First Aid

Full body: 69.7" x 20.5" x 9.8" (177 cm x 52 cm x 25 cm)

Weight: 79.2 lbs (36 kg)

#### Dimensions Resusci Baby First Aid

Full body: 23.2" x 10.2" x 5.5" (59 cm x 26 cm x 13.5 cm)

Weight: 4.0 lbs (1.85 kg)

### Warranty

The Laerdal Resusci Anne First Aid and Resusci Baby First Aid has a one-year limited Warranty. Refer to the Laerdal Global Warranty for terms and conditions.

Français

## Resusci Anne First Aid/Resusci Baby First Aid

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© 2013 Laerdal Medical AS. All rights reserved.