


SimBaby

Tracheostomy

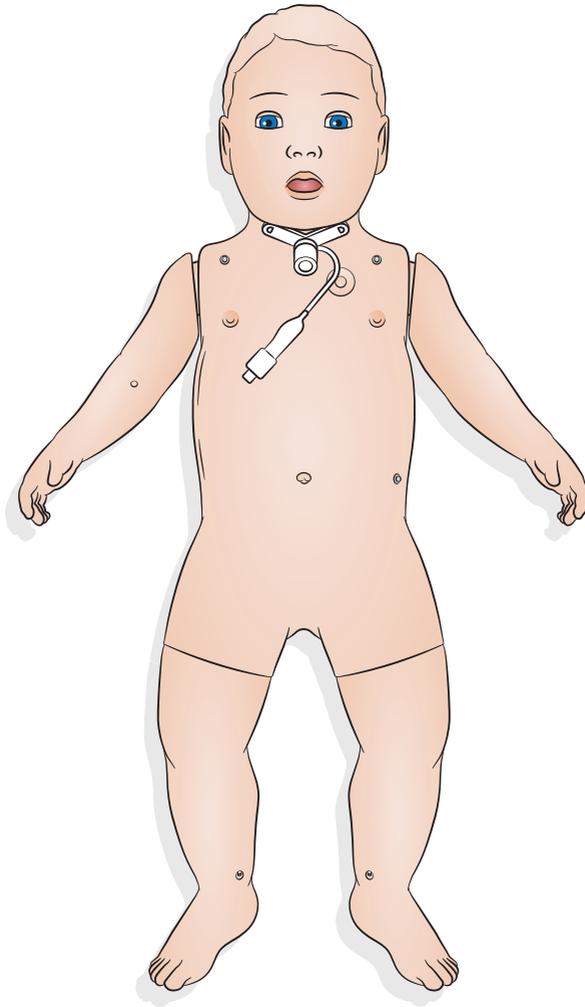

Use - Tracheostomy

Tracheostomy

To perform tracheostomy care and emergency procedures, select preferred setting in SimPad PLUS or LLEAP.

Settings available:


- Open
- Partial
- Closed
- False passage / mucus plug
- Ventilator setting




Notes

- *Pharyngeal swelling functionality is not available with SimBaby with Tracheostomy*
- *Only use tracheostomy tubes up to 4.0 mm uncuffed and up to 3.5 mm cuffed*
- *Remember to reinsert the tracheostoma plug after use*

SimPad PLUS

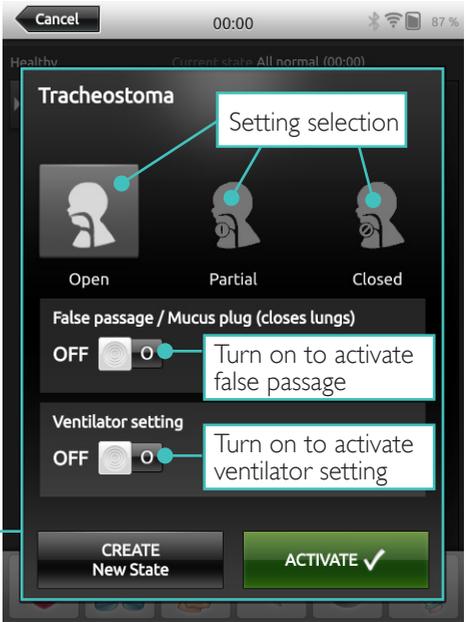

LLEAP

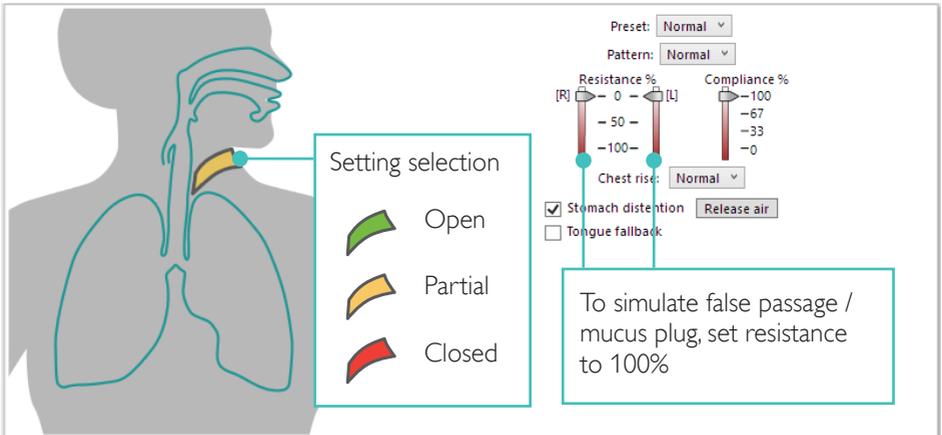

© 2019 Laerdal Medical AS. All rights reserved.

Manufacturer: Laerdal Medical AS
P.O. Box 377
Tanke Svilandsgate 30, 4002 Stavanger, Norway
T: (+47) 51 51 17 00

Printed in Norway

20-16236 Rev A

www.laerdal.com



Laerdal
helping save lives