

First Time WiFi Setup for Simulator Router and SimPad PLUS

Intended Use

The recommended simulator setup for this router is: one simulator; one SimPad Plus and one PC with Patient Monitor (Optional). Please follow this process in the right order to ensure an optimal connection.




How-to-video

For an easy step by step video of this process and language versions, go to:



www.laerdal.com/simulator-router

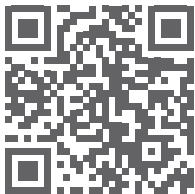

If you need a more advanced setup with more simulators, contact your local IT support. Alternatively, contact Laerdal for further help.

1 Plug in and turn on router

Steady pink light = ready

2 Turn on Simulator

Steady green light = ready

3 Turn on SimPad PLUS

4 Connect SimPad and Simulator with cable

5 Select Simulator

Press and select connected simulator. Follow on-screen instructions.

6 Select WiFi network

Press and select WiFi network.

7 Enter Network Name and password

Look under the router to find the name of the network and the password.

When the network is established, you can disconnect the network cable.

8 Connect to PC with Patient Monitor (Optional)


Turn on PC and open Laerdal Simulation Home, then open "Network Selector". Select same WiFi network and follow on-screen instructions.

If you experience connection problems, repeat this process.
Important: Do not reset the router. This might require customer service from Laerdal.

First Time WiFi Setup for Simulator Router and PC with LLEAP

Intended Use

The recommended simulator setup for this router is: one simulator; one PC with LLEAP and one PC with Patient Monitor (Optional). Please follow this process in the right order to ensure an optimal connection.

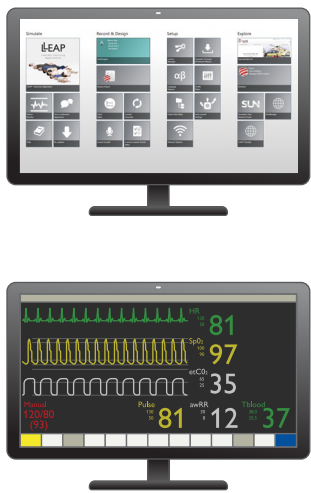


How-to-video

For an easy step by step video of this process and language versions, go to:



www.laerdal.com/simulator-router

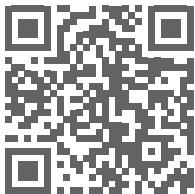

If you need a more advanced setup with more simulators, contact your local IT support. Alternativley, contact Laerdal for further help.

1 Plug in and turn on router

Steady pink light
= ready

2 Turn on Simulator

Steady green light
= ready

3 Turn on PC with LLEAP

4 Connect PC and Simulator with cable

5 Open Simulator Firmware & Network Wizard

Follow on-screen instructions to select your simulator

Simulator Firmware & Network Wizard - 6.6.0.3884

Simulator Firmware & Network Wizard

With Simulator Firmware & Network Wizard you can keep your simulator software up-to-date and configure network options

Make sure you are connected to the Simulator using a network cable

Detecting Simulator and SimPad. Please wait...

Please note that it might take a couple of minutes for a SimPad or Simulator to be detectable by the wizard...

Can't find the simulator?

Help and information

6 Open Network Selector

Select same WiFi network as Simulator, and follow on-screen instructions

Select wireless simulation network

| | |
|-------------------------|---------------|
| DIRECT-7BTAK-2016msPV | Other network |
| LaerdalNet | Other network |
| Laerdal-SIMPCLA-VC3B0J9 | Other network |
| Laerdal-TC-NOMarkus | Other network |
| LLEAP-MOBILE | Other network |
| LMUC | Other network |
| SimNetAP | Other network |
| TestRouter2 | Other network |

☒ Reconnect to selected Wifi on windows login

☐ Show only Simulator created networks

7 Enter Network Name and password

Look under the router to find the name of the network and the password.

NETGEAR Orbi Router RBR20

Network Name:
Laerdalnet1234
(last four digits in MAC address)

Network Key (Password):
XXXXXXXXXXXX

SERIAL
XXXXXXXXXXXX

MAC
XXXXXXXXXXXX

When the network is established, you can disconnect the network cable.

8 Connect to PC with Patient Monitor (Optional)

Turn on PC and open Laerdal Simulation Home, then open "Network Selector". Select same WiFi network and follow on-screen instructions.

If you experience connection problems, repeat this process.
Important: Do not reset the router. This might require customer service from Laerdal.