
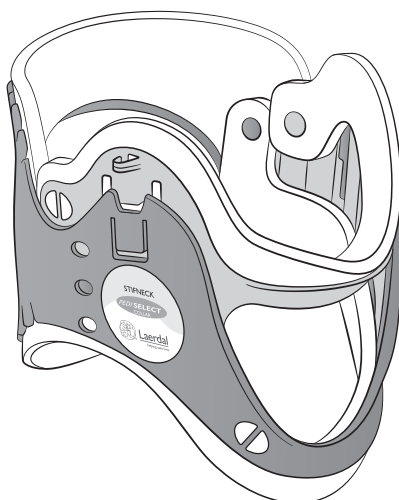
Stifneck Select









Select and Pedi-Select Extrication Collar

User Guide

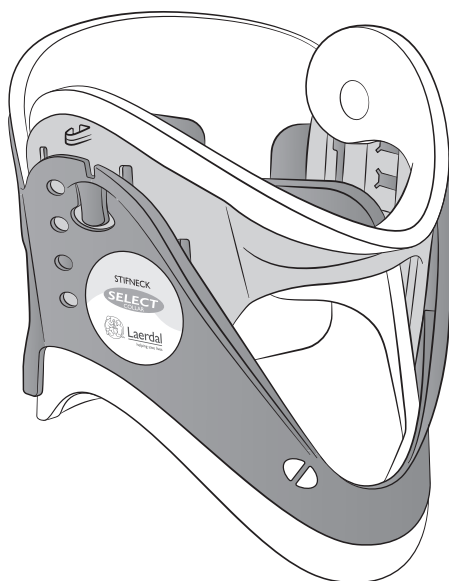- 98001005 Stifneck Select (J) (Qty:1)
- 98001205 Stifneck Select, green (J) (Qty:1)
- 98002005 Stifneck Pedi-Select (J) (Qty:1)
- 98991040 Stifneck Select, Private Label (Qty: 1)
- 98992040 Stifneck Pedi-Select, Private Label (Qty: 1)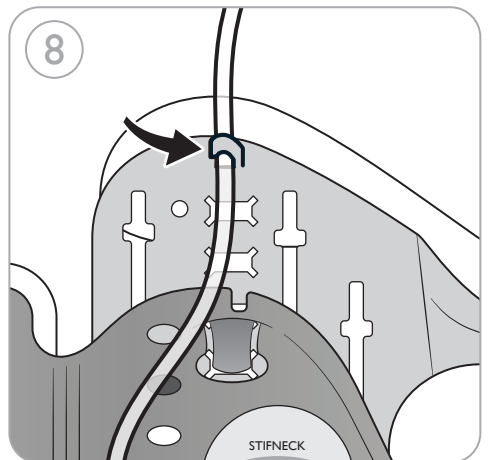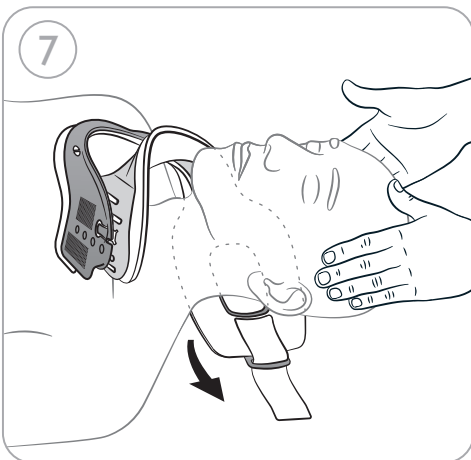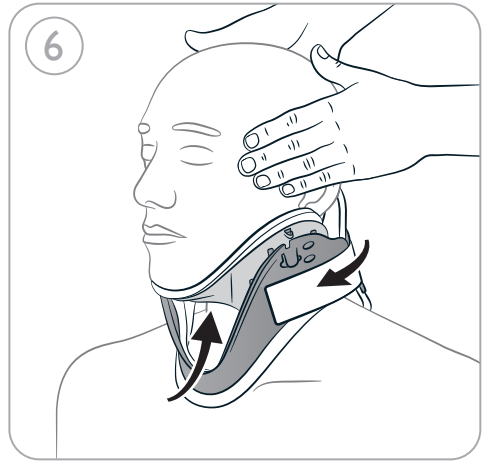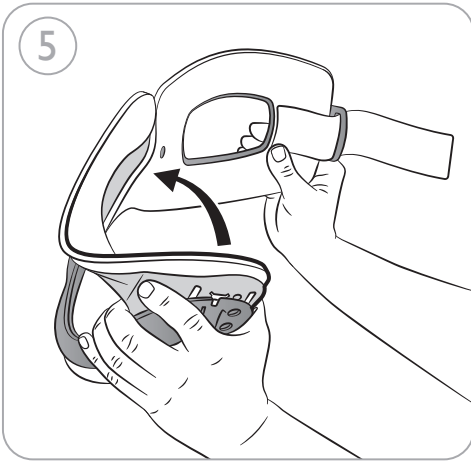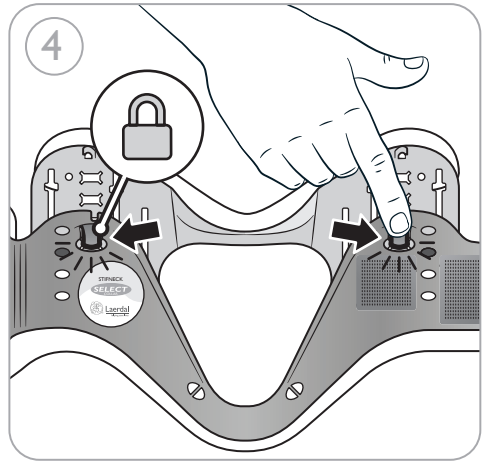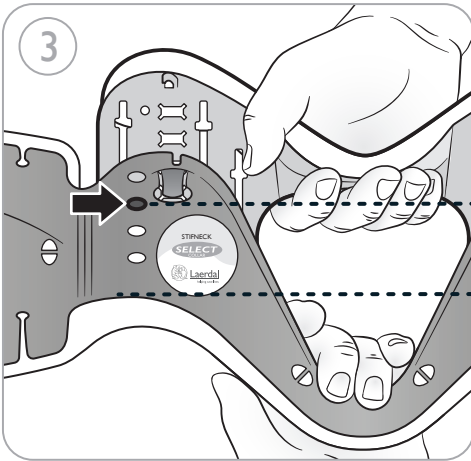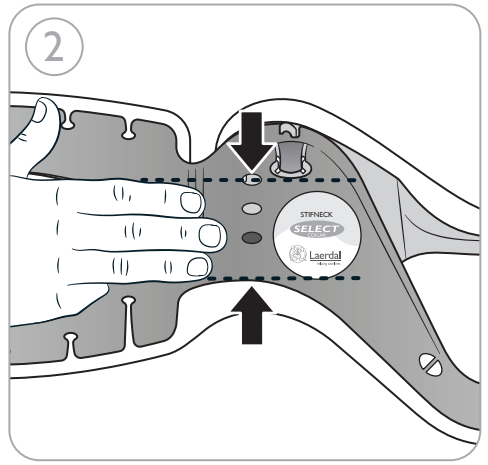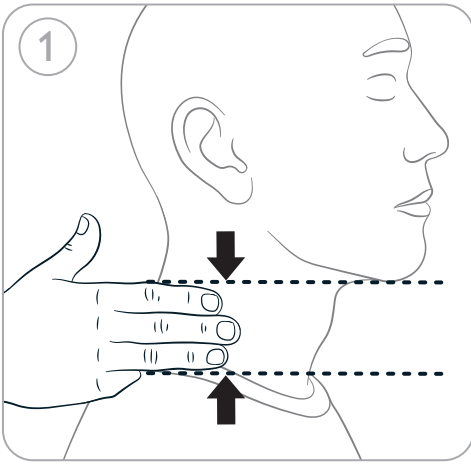

Intended Use

This product is intended to support cervical spine in a neutral position during transportation, in combination with other cervical and full body immobilization devices (CSIDs and FBSIDs).

 Notes

- In cases of a suspected spinal cord injury, proper neck immobilization is only one part of total immobilization.
- It is imperative that the patient is properly immobilized to prevent any movement of the spinal column.
- Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

 Warnings

- Stifneck Select Collars should only be used by healthcare providers.
- Improper use may result in injury or permanent disability. Always follow local spinal immobilization protocol.
- Use correct collar size. An oversized collar may hyperextend a patient's cervical spine. An undersized collar may not provide appropriate stability.
- Do not adjust the Select or Pedi-Select collar on patient.

User Instructions

1. Measure patient in neutral head position from shoulder to chin. Align head to neutral or "eyes forward" position unless contraindicated by local protocol.
2. Match collar size to patient as shown in step 2 (from plastic edge to size window). Select has 4 positions, Pedi-Select has 3 positions.
3. Adjust collar and chin support to size selected in step 2.
4. Lock both sides by pressing the two lock tabs.
5. Preform the collar.
6. Apply collar while manually maintaining neutral head position. Position chin support under chin. If a different size is needed, remove, resize, and re-apply. Pull the collar around the neck ensuring a close fit, while holding the front in place - then fasten.
7. For a supine patient, slide the rear panel behind the neck before positioning the chin support.
8. Cannula hooks are provided for attaching supplemental oxygen lines.

Technical Specifications	
Storage	Do not store collar in folded position. Store in flat position.
Storage Temperature	-34 °C to 52 °C (-30 °F to 125 °F)
Operating Temperature	-25 °C to 43 °C (-13 °F to 110 °F)
Symbol Glossary	
	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.
	Not made with natural rubber latex.
	Do not re-use.
	Medical Device
	Rx Federal law (USA) restricts this device to sale by or on the order of a physician.

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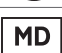
Manufactured in China for:

Laerdal Medical AS
P.O. Box 377
Tanke Svilandsgate 30, 4002 Stavanger, Norway
Tel.: (+47) 51 51 17 00

Distributed by:

Canada and Latin America:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel.: +1 (416) 298-9600,
Toll free 888/LAERDAL (523-7325)
ou en français (800) 567-9987
Fax: +1 (416) 298-8016

USA:

LAERDAL MEDICAL CORPORATION
167 Myers Corners Road, P.O. Box 1840
Wappingers Falls, New York 12590-8840
Tel.: (800) 431-1055, +1 (845) 297-7770
Fax: (800) 227-1143, +1 (845) 298-4545

Australia

Laerdal Pty. Ltd.
8 Stamford Road
Oakleigh VIC 3166.

Laerdal do Brasil

Al.Tocantins 125 – sl 01 – Alphaville
Barueri / SP / Brazil
06543-260
Tel.: +55 11 4193-8007
e-mail: comercial@laerdal.com

© 2020 Laerdal Medical AS. All rights reserved.

Printed in China