# Penguin

## Newborn Suction

Reusable

**REF** Cat. No. 98600060 **EN**  
**QTY** 1  
**DATEX** Not made with natural rubber latex  
 ! Discard if damaged  
 ! Disinfect after use  
**MD** Medical Device

This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.

**Note**  
 Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

**Technical specifications:**  
 Suction strength (typical): 100 mm Hg [136 cm H<sub>2</sub>O]  
 Operating temperature: 0 °C (32 °F) to 50 °C (122 °F)  
 Storage temperature: -20 °C (-4 °F) to 60 °C (140 °F)  
 Material: Silicone  
 Fluid capacity: 75ml

Protected under US Patent 8,696,648 and other international patents; US design patent D634,002 S and other international design registrations. Ownership and all rights reserved.

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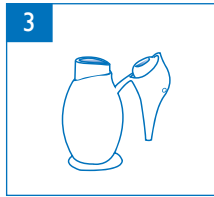
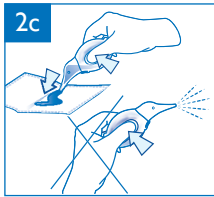
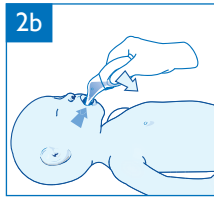
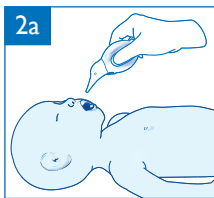
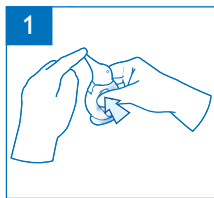
© 2021 Laerdal Medical AS. All rights reserved.

Manufactured in China for:  
 Laerdal Medical AS  
 Tanke Svilandsgate 30, P.O. Box 377,  
 4022 Stavanger, Norway  
 Tel. (+47) 51511700  
 Fax (+47) 51523557  
 E-mail: laerdal.norway@laerdal.no

Distributed in the USA by:  
 LAERDAL MEDICAL CORPORATION  
 167 Myers Corners Road, P.O. Box 1840  
 Wappingers Falls, New York 12590-8840  
 Tel. (800) 431-1055, +1 (845) 297-7770  
 Fax (800) 227-1143, +1 (845) 298-4545  
 Email: customerservice@laerdal.com

**EN**  
 This Device is intended for the removal of fluids from nasal and oral cavities of newborns and infants. Also suitable for more general use as an irrigating syringe.  
 Penguin must be cleaned and high-level disinfected before first time use.

**1 To test before use**  
 Squeeze body and block the suction tip to verify there is no air leakage. Body should not expand until tip is unblocked.

**2 To Use**  
 a Squeeze and insert  
 b Release to suction  
 c Empty contents  
 d Repeat as required.  
 Suction mouth first, then nose.

**3 To reduce the risk of cross-contamination, disinfect after use**  
 Open head. Wash and brush in soapy water. Flush in clean water. Autoclave (136 °C (277 °F) and 2.0 kg/cm<sup>2</sup> for 10-20 min); OR boil for 10 min. Dry completely and keep clean.

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