EN

IT

DE

FR

ES

NL

PT

SV

DA

NO


FI

PL

V-VAC

Manual Suction Unit

User Guide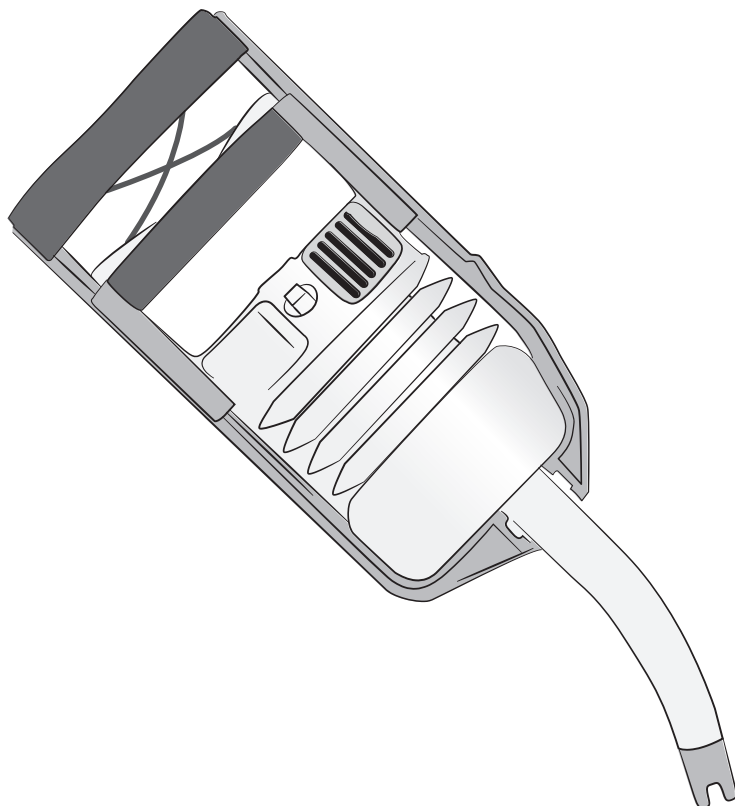


The V-VAC Manual Suction Unit is a manual portable unit designed to provide quick and effective suction. Fully assembled, it weighs less than one pound (0,45 kg) and can be adjusted to accommodate different hand sizes or to provide a range of suction pressures from no more than ~170mmHg up to ~380mmHg. It is possible to continue suction after overflow has occurred.

Intended Use

The V-VAC is a manual suction device intended for pharyngeal suction. V-VAC is a portable unit designed to provide quick and effective emergency suction in situations where conventional powered apparatus cannot be used or is not available.

Components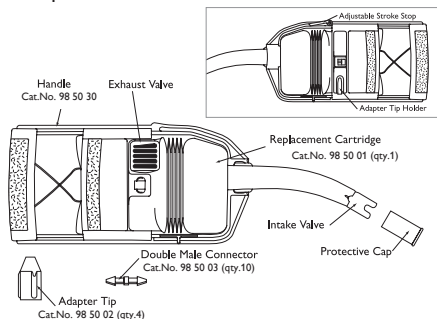

Cautions/ Warnings

⚠ Warnings and Cautions

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the product.

📄 Note

A note states important information about the product or its operation.

⚠ Warnings

- All parts except the handle are designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.
- Training should take place before the device is used on a patient.
- The V-VAC is not intended for use in an MRI setting. It contains magnetic parts.

⚠ Cautions

- Read these Directions for Use carefully and become familiar with the operation and maintenance before you use the unit.
- The use of suction devices may require appropriate infection disease precautions be taken during use, cleaning of the device and disposal of the cartridges.
- Federal law (USA) restricts this device to sale by or on the order of a physician.
- Do not block exhaust outlet during use.
- Replacement cartridges are not intended for reuse.
- Keep cartridge compressed until installed in handle. Remove retaining strap just prior to installation
- Not intended for tracheal suction.
- To avoid clogging the exhaust filter when the device is partially filled, do not point the cartridge tip up

The intake valve (cartridge tip) is designed to prevent the backward flow of suctioned matter back through the tip when cartridge tip is pointed downward.


📄 Note


Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

Cartridge Installation

1. Hold the handle as shown in your left hand. Holding the cartridge at a right angle, insert the oval-shaped hole fitting firmly into the fitting on the handle (see inset).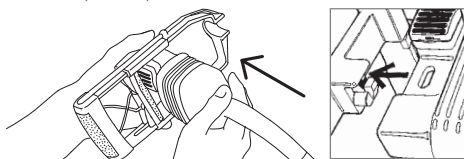

2. Keeping the cartridge in the fitting, pivot it towards the handle and then snap it into place with your thumbs. The intake valve is protected by a plastic cap.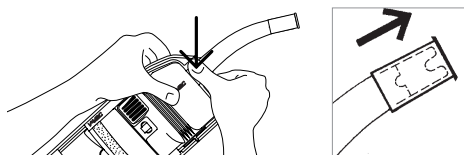

⚠ Warning


Be sure to remove this protective cap before using the unit (see inset)

Function Test

1. Squeeze and release the handle a few times to ensure that the cartridge is correctly installed and that the handle functions properly.
2. Observe that the cartridge expands and releases.
3. Use your hand to observe that air is exiting the suction tip when releasing the handle


Stroke Adjustment

To adjust the stroke of the handle (and thereby adjust the suction pressure that can be generated), squeeze the handle and flip the stroke stop over (see inset). The front position provides maximum suction pressure of ~380mmHg. The rear position offers a lower suction pressure with a maximum of ~170mmHg. To secure the stroke stop, push down until it snaps into place.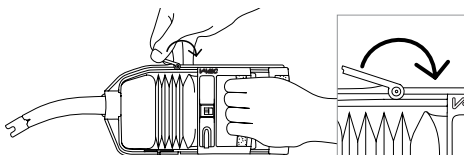

Adapter Tip


An adapter tip is provided for additional applications. This can be stored on the adapter tip holder (see inset). Most standard suction catheters without suction activation ports can be used with the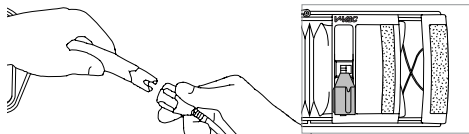

adapter tip. Catheters should be firmly seated into the adapter tip before use. The adapter tip should then be pushed over the intake valve. A double male connector is provided to use the adapter tip with female-ended catheters.

Cartridge Removal


1. Hold the handle as shown with the tip pointed away from you. With the heel of your left palm, push out on the cartridge reservoir while gripping the handle frame beneath the nozzle with the fingers of your left hand.
2. As the cartridge pops out, catch it with your right hand, pivot it to a right angle position and remove. As an additional safe-guard against leakage, the protective cap can be replaced over the intake valve.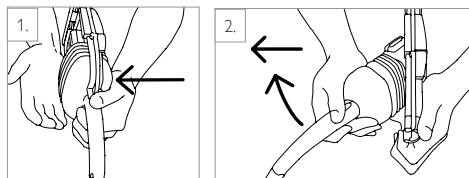

Troubleshooting


Clogging-Intake

Clogging of the intake valve is minimized due to its large (1/2") bore. If a very large particle clogs the tip, increased resistance will be felt when squeezing the handle. By keeping the handle squeezed, the particle can be held on the tip and lifted out of the patient's mouth. To unclog the intake valve, keep the tip pointed down and push through or remove the stuck particle. Suctioning can then be continued as before.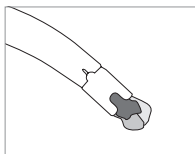

Clogging-Exhaust

The exhaust valve has been positioned to limit the possibility of it becoming clogged during use. If the 425 ml cartridge becomes full, the exhaust filter could clog. This can also happen if the cartridge is partially full and is held tip up. If the exhaust filter clogs, the handle will not return to the ready (compressed cartridge) position. In this situation, flip open the exhaust valve cap that holds the filter. In most cases, the exhaust valve will clear and the unit will again operate.

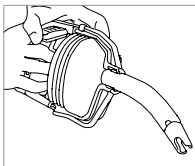 Note

With the exhaust filter removed, some emesis may escape through the exhaust valve.

Cleaning

Handle can be cleaned and disinfected after patient use.

1. Wash and scrub in warm soapy water.
2. Rinse in clean water for min 1 min.
3. Submerge for 10 minutes in a 0.5% Sodium Hypochlorite solution.
4. Rinse under running tap water for 3 min.
5. Allow to dry completely.

Check the handle's condition and function before returning it to service.

Specifications

Spesification Operation	
Peak Airflow:	70 liters/minute
Vacuum, Approximately	380 mmHg plus (51 kPa plus)
Vacuum Range, Approximately	170 mmHg – 380 mmHg plus (23 kPa – 51 kPa plus)
Physical Characteristics	
Size (fully assembled):	14.5" x 2.7" x 4.8" (37cm x 7cm x 12,2cm)
Weight Approximately:	10.3 oz. (292g)
Cartridge Capacity	425ml
Environmental	
Operating Temperature	-18°C to 45°C (0°F to 100°F)
Storage Temperature	-40°C to 63°C (-40°F to 150°F)
Materials	
Handle	PA, PC, Stainless steel, POM, PP, Foam
Cartridge	HDPE, LDPE
Catheter Adapter	TPE
Connector Double Male	PA
Dimensions	
Cartridge Spout	Approximately 15 cm
Symbol Glossary	
	Single Patient Use. Do not re-use.
	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.
	Medical Device
	Caution: Federal law restricts this device to sale by or on the order of a physician.

Limited Warranty

Please refer to the GlobalWarranty statement for additional terms and conditions (www.laerdal.com)

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© 2021 Laerdal Medical AS. All rights reserved.