EN

Intended Use

Laerdal Silicone Masks are intended to be used for ventilation of patients.

The Multi-Function Mask Cover is intended to be used to add additional support, when needed, to Laerdal Silicone Masks (size 3-4, 4-5+) and to allow for strapping of Laerdal Silicone Masks to the patient.

Decontamination

Thorough decontamination of Laerdal Silicone Masks and Multi-Function Mask Covers is necessary after each use. Decontamination of the mask and mask cover prior to first time use is also recommended.

- Pre-rinse Disassemble the mask and cover (if applicable) and rinse in lukewarm (<43°C) water before exposure to detergent.
- Washing Wash mask and cover in hot water (60°-70°C) with a detergent compatible with the mask and cover material (see materials list).
- Rinse Rinse the mask and cover in clean water until all traces of detergent are removed. Allow to air dry.

- Disinfection/Sterilisation. One of the following methods is recommended:

Method	Parameters/Concentration	Exposure time	Post- treatment
STERILISATION - The sterilisation methods apply to all parts except head-straps.			
Steam Autoclaving	Autoclave at 132-137°C	15 minutes	Allow parts to cool
HIGH-LEVEL DISINFECTION - High-level disinfection methods apply to all parts.			
Cidex OPA (orthophthalaldehyde)	Conc.:0.55% Ambient temperature	60 minutes	Rinse all parts thoroughly in sterile water. Allow to dry
Sodium Hypochlorite (Not cleared for use in the US)	Conc.:0.5% Ambient temperature	20 minutes	Rinse all parts thoroughly in sterile water. Allow to dry

Technical Information

Specifications		
Storage temperature	- 40°C to 70°C	40% to 95% relative humidity
Operating temperature	-18°C to 60°C	15% to 95% relative humidity
Patent port on mask fits standard 15/22 mm connector		

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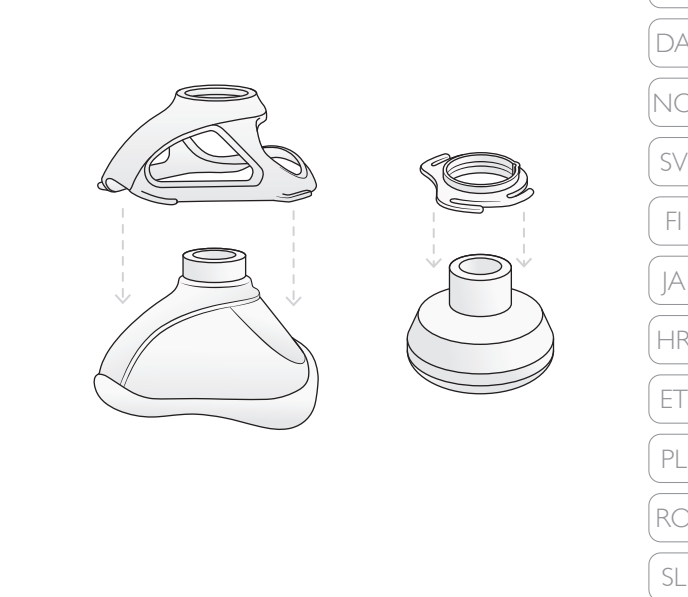
User Guide

www.laerdal.com

Materials	
Mask	Silicone (SI)
Multi-Function Mask Cover	Polysulfone (PSU)
Head Strap	PolyVinylChloride (PVC)
Attachment Ring	Polysulfone (PSU)

- Illustration of assembly with Laerdal Silicone Resuscitator
- Illustration of assembly with Laerdal Head Strap

Models

851500 (05) Silicone Mask No.00 (JA)
851600 (05) Silicone Mask No.01 (JA)
851700 (05) Silicone Mask No.2 (JA)
860220 (05) Silicone Child Mask 3-4 w/Multi-Function Mask Cover (JA)
860221 Silicone Child Mask 3-4 w/o Multi-Function Mask Cover
870220 (05) Silicone Adult Mask 4-5 w/Multi-Function Mask Cover (JA)
870221 Silicone Adult Mask 4-5 w/o Multi-Function Mask Cover
872220 (05) Adult & Child Silicone Mask (JA)

Accessories

871000 (05) Silicone Extension Tube (28 cm) (JA)
850900 (05) Manometer Connector (JA)
850500 (05) Expiration Diverter (Ø 30 mm) (JA)
531907 Intake Valve Adapter (Ø 23 mm)
865200 (05) Multi-Function Mask Cover 3-4 (JA)
875200 (05) Multi-Function Mask Cover 4-5 (JA)
870400 (05) Head Strap w/ Attachment Ring (JA)
870120 (05) Hanging Loop (JA)
870702 Disposable O₂ Reservoir

-  This product is in compliance with the essential requirements of EU Council directive 93/42/EEC as amended by EU Council directive 2007/47/EC.

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