

Connect the PC with AED Trainer 2

Serial Port



Recommended cable

If the PC has Serial Port DB-9 connector, it is recommended to use below cable.

Serial Port to Serial Port - RS 232

DB-9S female to DB-9P male connector



If the PC only has USB port, use below cable.

USB to Serial Port Adapter

DB-9P - 9 poles male connector

Item 400-20150 - USB-Serial Adapter (5 meter)

UT232R-500 from FTDI



Use of a USB cable might cause software update challenges.

Please notice

- If the PC only has USB port, please make sure to use a high-quality cable
- Cable UT232R-500 from FTDI is recommended - it includes embedded electronics that functions well with AED Trainer 2
- USB cable with low voltage might cause software update problems.
- USB cable can cause timeout problems in serial communication.