
Laerdal Disposable Mask with Inflation Port

User Guide

- EN
- FR
- DE
- ES
- IT
- NL
- PL
- DA
- NO
- SV
- FI
- JA

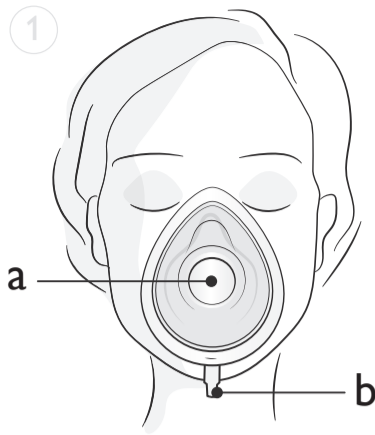REF Cat.no. 845155, 845156, 845150, 845160, 845170

© 2019 Laerdal Medical AS. All rights reserved.

Manufactured in China for:
Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30,
4002 Stavanger, Norway
T: (+47) 51511700


Printed in China


www.laerdal.com

20-13738 Rev.B

2 Function Test - Leakage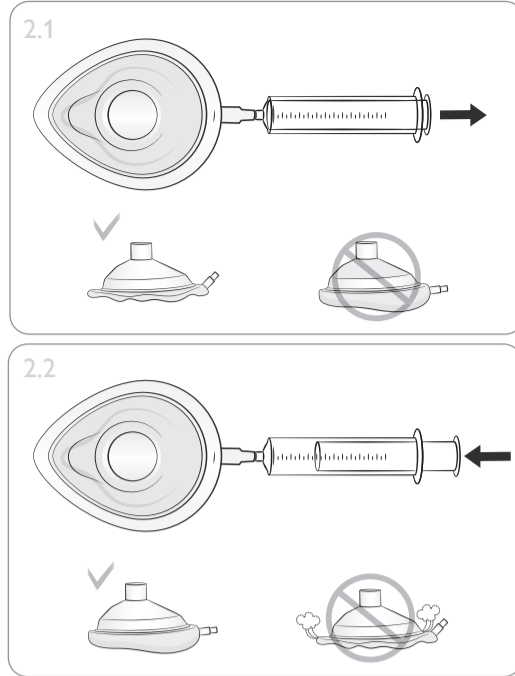

3 Use

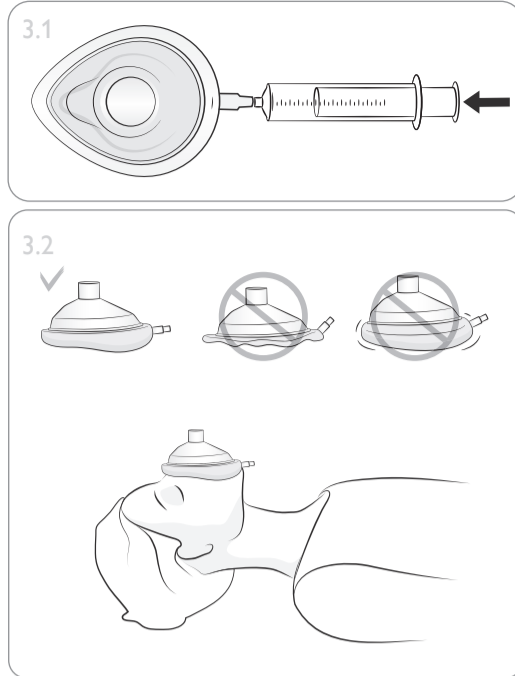Maximum Inflation Volume (Guidance)		
Size	Size Guidance	Maximum Inflation Volume
1	Infant	20 ml
2	Small Child	60 ml
3	Child	70 ml
4	Small Adult	80 ml
5	Adult	95 ml

Specifications	
Storage temperature	-40 °C to +60 °C (-40 °F to 140 °F)
Operating temperature	-10 °C to +50 °C (14 °F to 122 °F)
Patient port on mask fit standard 15/22 mm connector	
Materials	
Mask dome and cushion	PVC
Check valve	PC
Heat shrink tape	PVC
The materials have not been tested for flammability.	

Symbol Glossary	
	The product is in compliance with the essential requirements of Council Directive 93/42/EEC as amended by Council Directive 2007/47/EC.
	Single Patient Use. Do not re-use.
	Reference order and number
	LOT Number (Production Date)
	Not made with natural rubber latex
	Dispose of according to local protocol.

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com.

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EN

Intended Use

The Laerdal Disposable Mask with Inflation Port is intended to be used for ventilation of human patients.

Warnings

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

Warnings

- This device should only be used by persons who have received adequate training.
- Do not attempt to clean or re-use this device.
- Do not expose the valve to any cleaning solution as it may cause premature valve failure.

1. Illustration Overview

- a Ventilation Connector
- b Valve Port

2. Function Test - Leakage

2.1 Deflate

- Fully deflate mask using a syringe. Remove syringe.
- Examine cushion to ensure it remains fully deflated.
- If the cushion inflates, this indicates a leak.

2.2 Inflate

- Fully inflate mask using a syringe. Remove syringe.
- Examine cushion to ensure it remains fully inflated.
- If the cushion deflates, this indicates a leak.

3. Use

- 3.1 Inflate mask with a volume of air suitable to give an effective seal.
- 3.2 Press the mask on the patients face to obtain a seal.

Note

There should be sufficient air in the cuff so that the rigid plastic cannot be felt through the mask when it is pressed onto the face. The mask must not be over-inflated. There should be some give left in the mask to allow an effective seal to be formed.

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