


Coaxial Vacuum Connector Kit for Laerdal Suction Unit (Cat # 780422)

Directions for Use

Important

Read these directions completely before using this kit. Refer to the Laerdal Suction Unit (LSU) Directions for Use for further information.

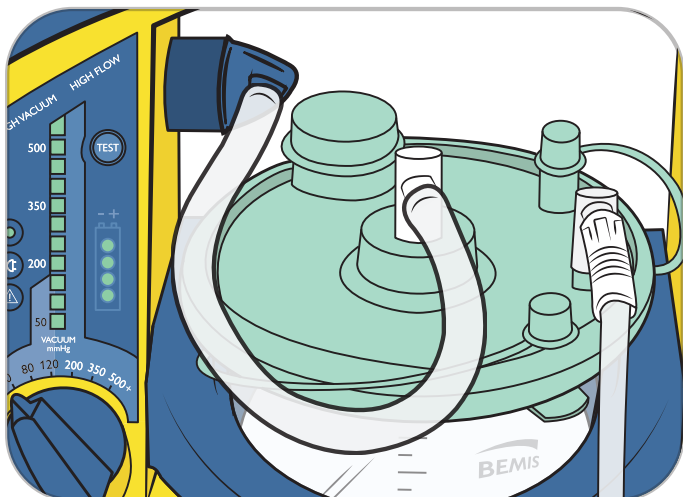

Installation:

The Coaxial Vacuum Connector Kit must be inserted between the Canister and the LSU Cabinet. To install the Coaxial Vacuum Connector:

- 1) Attach the blue angled connector of the Coaxial Vacuum Connector to the LSU Vacuum Connector Inlet.
- 2) Attach the Vacuum Connector Tubing to the elbow connected to the Vacuum Port on the Canister Lid to form a closed circuit between the canister and the pump unit.
- 3) Run the LSU Device Test according to the instructions found in the LSU Directions for Use. Paragraph, 3.2.3 Test.

⚠ Cautions

- *Install and use this Kit only as described in this Directions for Use. Non-adherence can lead to negative consequences such as the LSU being unable to perform its intended function, possibly resulting in user inability to provide intended patient care.*
- *The connector and tubing are single patient use items and should be disposed of according to local maintenance procedures.*

Reorder Information


780422 Coaxial Vacuum Connector - LSU (Bemis)
(Includes blue angled connector, connection tube and Directions for Use).



Single Use. Do not re-use.

Coaxial Vacuum Connector
Blue Angled Connector

Vacuum
Connector
Tubing

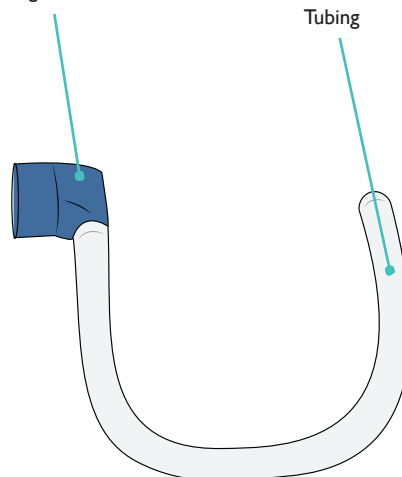

Warranty

Please see the Global Warranty statement
for terms and conditions.

© 2017 Laerdal Medical AS. All rights reserved.

Made in China for: Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30
4002 Stavanger, Norway
Tel +47 51 51 17 00
Fax +47 51 52 35 57

Distributed in USA by:
Laerdal Medical Corporation
167 Myers Corners Road
P.O. Box 1840
Wappingers Falls, New York 12590-8840
Tel +1 (800) 431-1055, (845) 297-7770
Fax +1 (800) 227-1143, (845) 298-4545

Distributed in Canada by:
Laerdal Medical Canada, Ltd.
151 Nashdene Road, Unit # 45
Toronto, ON, Canada M1V 4C3
Tel +1 (888) LAERDAL (523-7325)
Tel +1 (416) 298-9600
Ou en francais (800) 567-9987
Fax +1(416) 298-8016

Distributed in Australia by
Laerdal Pty. Ltd.
8 Stamford Road
Oakleigh VIC 3166.
Australia

Printed in China

20-11443 Rev B