Stifneck

User Guide

980100 Stifneck Baby No-Neck (Qty. 1)
980200 Stifneck Pediatric (Qty. 1)
980300 Stifneck No-Neck (Qty. 1)
980400 Stifneck Short (Qty. 1)
980500 Stifneck Regular (Qty. 1)
980600 Stifneck Tall (Qty. 1)

ENGLISH
FRANÇAIS
ESPAÑOL
PORTUGUÊS

Manufactured in China for:
LAERDAL MEDICAL AS, Norway
P.O. Box 377, N-4002 Stavanger, Norway
Tel. +47 51 51 17 00,
Fax +47 51 52 35 57

Intended Use

This product is intended to support cervical spine in a neutral position during transportation, in combination with other cervical and full body immobilization devices (CSIDs and FBSIDs).

Important


- In cases of a suspected spinal cord injury, proper neck immobilization is only one part of total immobilization.
- It is imperative that the patient be properly immobilized to prevent any movement of the spinal column.
- Should any serious malfunction, undesirable incident with, or deterioration in the functionality or performance of the device occur, contact Laerdal promptly. The competent authority where the incident took place and/or the device was used should also be notified.

Warnings

- Stifneck Collars should only be used by qualified healthcare providers.
- Improper use may result in injury or permanent disability. Always follow local spinal immobilization protocol.
- Use the correct size collar. An oversized collar may hyperextend a patient's cervical spine. An undersized collar may not provide appropriate stability.

User Instructions


1. Measure the Patient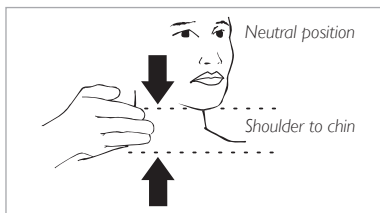


Align the head to neutral or “eyes forward” position unless contraindicated by local protocol.

2. Match the collar size to the patient:


Plastic edge to hole or to rivet if assembled.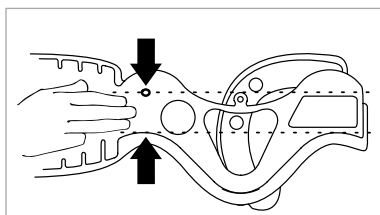

3. Insert the rivet into the hole.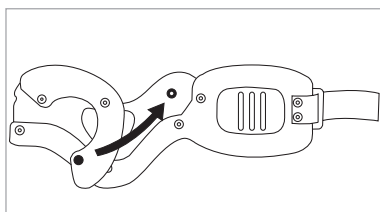


4. Preform the collar

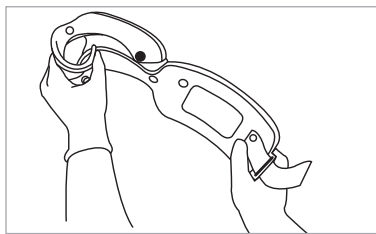
5. Apply the collar while manually maintaining neutral head position

Place the chin support under the chin.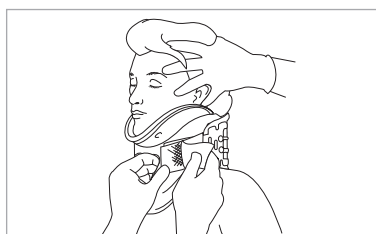

Pull the back of the collar snug while holding the front in place, and then fasten.

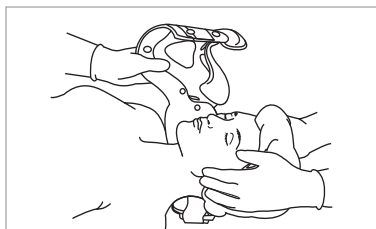For a supine patient, slide the rear panel behind the neck before placing the chin support.

Technical Specifications

Storage:

Do not store collar in folded position.
Store in flat position.

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EN

This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.

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EN

Rx Federal law (USA) restricts this device to sale by or on the order of a physician.

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EN

Single Use Warning: Designed for single use only. Do not re-use. Re-use will lead to increased risk of cross contamination, degradation of performance and/or device malfunction. Laerdal is not responsible for any consequences of re-use.

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EN

Global Warranty:
See the Laerdal Global Warranty statement for terms and conditions. Visit www.laerdal.com.

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