© 2019 Laerdal Medical AS.
All rights reserved.

Manufactured in China for:
Laerdal Medical AS
P.O. Box 377, Tanke Svilandsgate 30,
4002 Stavanger, Norway
T: (+47) 51511700

Printed in China


20-13738 Rev.B

www.laerdal.com

Laerdal Disposable Mask with Inflation Port

User Guide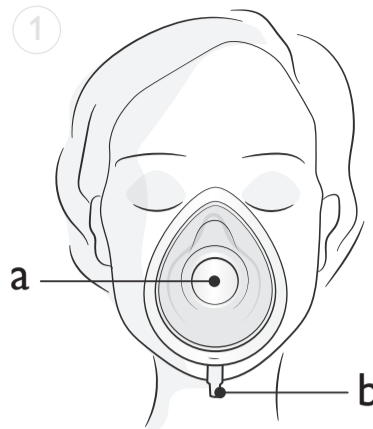


REF Cat.no. 845155, 845156, 845150, 845160, 845170


- EN
- FR
- DE
- ES
- IT
- NL
- PL
- DA
- NO
- SV
- FI
- JA

2 Function Test - Leakage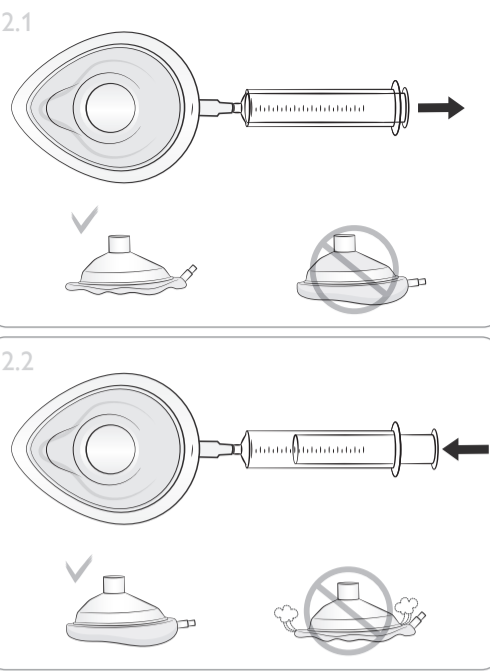

3 Use

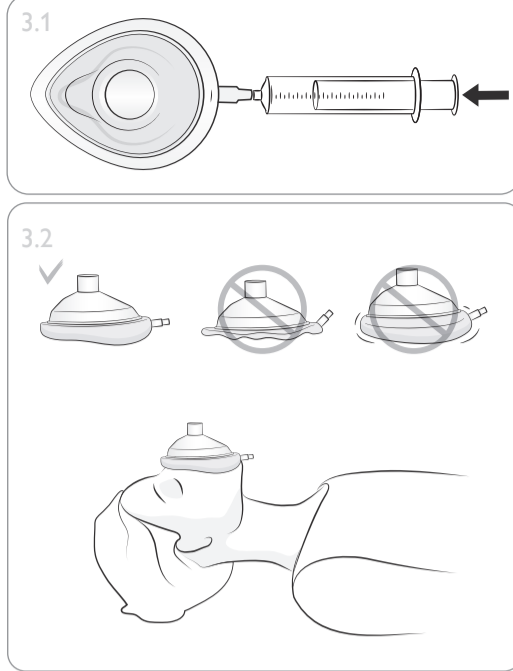EN

Intended Use

The Laerdal Disposable Mask with Inflation Port is intended to be used for ventilation of human patients.

Warnings
A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

Warnings
• This device should only be used by persons who have received adequate training.
• Do not attempt to clean or re-use this device.
• Do not expose the valve to any cleaning solution as it may cause premature valve failure.

1. Illustration Overview

- a Ventilation Connector
- b Valve Port

2. Function Test - Leakage

- 2.1 Deflate**
- Fully deflate mask using a syringe. Remove syringe.
 - Examine cushion to ensure it remains fully deflated.
 - If the cushion inflates, this indicates a leak.
- 2.2 Inflate**
- Fully inflate mask using a syringe. Remove syringe.
 - Examine cushion to ensure it remains fully inflated.
 - If the cushion deflates, this indicates a leak.

- 3. Use**
- 3.1 Inflate mask with a volume of air suitable to give an effective seal.
- 3.2 Press the mask on the patients face to obtain a seal.

Note
There should be sufficient air in the cuff so that the rigid plastic cannot be felt through the mask when it is pressed onto the face. The mask must not be over-inflated. There should be some give left in the mask to allow an effective seal to be formed.

| Maximum Inflation Volume (Guidance) | | |
|-------------------------------------|---------------|--------------------------|
| Size | Size Guidance | Maximum Inflation Volume |
| 1 | Infant | 20 ml |
| 2 | Small Child | 60 ml |
| 3 | Child | 70 ml |
| 4 | Small Adult | 80 ml |
| 5 | Adult | 95 ml |

| Specifications | |
|--|-------------------------------------|
| Storage temperature | -40 °C to +60 °C (-40 °F to 140 °F) |
| Operating temperature | -10 °C to +50 °C (14 °F to 122 °F) |
| Patient port on mask fit standard 15/22 mm connector | |

| Materials | |
|-----------------------|-----|
| Mask dome and cushion | PVC |
| Check valve | PC |
| Heat shrink tape | PVC |

The materials have not been tested for flammability.

Symbol Glossary

| | |
|--|---|
| | The product is in compliance with the essential requirements of Council Directive 93/42/EEC as amended by Council Directive 2007/47/EC. |
| | Single Patient Use. Do not re-use. |
| | Reference order and number |
| | LOT Number (Production Date) |
| | Not made with natural rubber latex |
| | Dispose of according to local protocol. |

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com.

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