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[The Laerdal Learning Solution Team](#)

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#### HeartCode 2025 BLS Online

##### How Does This Program Work?

HeartCode® 2025 BLS Online is a self-directed, comprehensive eLearning program that uses adaptive learning technology to allow learners to acquire and demonstrate Basic Life Support skills using a personalized learning path that adapts in real time to a learner's performance. The program is designed for healthcare professionals who need Basic Life Support training for their clinical duties. Utilizing a variety of eLearning assets such as dramatizations, Cognitive Assessment Activities, illustrations, knowledge checks and interactive activities, this program teaches BLS knowledge and skills. This method of learning provides training consistency and adaptability to different learning styles. Students can work at their own pace applying their knowledge to real-time decision making. Debriefings and coaching are provided immediately after each knowledge check and each Cognitive Assessment Activity. Topics covered in this course are:

- The steps of the Chains of Survival
- How to perform high-quality CPR for an adult, a child and an infant
- How to use an AED
- Effective team dynamics in a multirescuer resuscitation
- How to treat special considerations in basic life support
- How to relieve foreign-body airway obstruction for an adult, a child and an infant

Students who successfully complete the cognitive portion and the hands-on session (skills) will receive an AHA BLS Provider course completion card (eCard), valid for two years.

##### Who Should Take This Program?

Healthcare professionals and other personnel who need to know how to perform CPR and other basic

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Welcome to HeartCode® 2025 BLS Online

Steps to Complete the Program:

1. Access and complete the cognitive portion.
2. Complete the program survey.
3. Download your completion certificate by clicking the 'Certificate' button.
4. Select "Claim CME/CE" to claim continuing education credit.
5. Complete a hands-on session (skills) at an RQI Simulation Station or with an AHA Instructor to earn an eCard.

Additional Information:

\*The program survey must be submitted after reviewing all the program content.

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The screenshot shows the RHAPSODE learner interface. On the left is a 'LEARNER' dashboard with a gear icon (2) for settings. In the center is a user profile card with a globe icon (3) for language selection. Below the profile is a language selection form with a search box containing '日本語' (4) and a radio button selected for '日本語'. On the right is a course list with a vertical menu icon (3) next to the 'HeartCode BLS 2025' course, which is highlighted with a red box. A red arrow points from this menu icon to a blue callout box that says 'コース名 ここをクリックしコース開始'. Another red arrow points from the course name 'HeartCode BLS 2025' to the same callout box.

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The screenshot displays the 'HeartCode 2025 BLS Online' interface. On the left, a sidebar contains a hamburger menu icon (three horizontal lines) highlighted with a red box. Below it, the 'PROVIDER MANUAL' option is highlighted with a red box and a red arrow pointing to a thumbnail image of the manual cover. The cover features the American Heart Association logo and the text 'BLS 次救命処置'. On the right side of the interface, a 'Table of Contents' panel lists sections P1 through P5. A red box highlights the 'Read this section' button at the top right of this panel, with a red arrow pointing to the 'PROVIDER MANUAL' thumbnail.

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Site powered, owned and operated by the Laerdal Learning Platform for AHA Training Centers

Educational materials and credentials provided by the American Heart Association

© Copyright 2021 Laerdal Medical All right reserved. Unauthorized use prohibited.

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Certificate Language \*

Japanese

Note: make sure that you have made right selection, certificate will be shown in the same language.

Continue

English (Default) ▾

 **Laerdal** Certificate Site  
helping save lives

English (Default) ▾

## You have these certificates

Showing 1 to 4 of 4 entries Rows per page 10 ▾

< 1 >

Certific

RQI Pro

RQI Pro

RQI Provider eCredential (Japanese)

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© Copyright 2020 Laerdal Medical