EN

FR

DE

ES

IT

BR

NL

PL

JA

ZH

KO

Labia Repair Model

Important Product Information

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www.laerdal.com

Laerdal

helping save lives

Read these instructions thoroughly. Observe all warnings, cautions and instructions in the User Guide and in this Important Product Information booklet. Retain this booklet for future reference.

Warnings and Cautions

A Warning states a condition, hazard, or unsafe practice that can result in serious personal injury or death.

A Caution states a condition, hazard, or unsafe practice that can result in minor personal injury or damage to the product.

Notes

A note states important information about the product or its operation.

General handling

The Labia Repair Model is designed to be used by learners under instructor guidance, beginner, or experienced health professionals to practice skills and develop competence in assessing and suturing labial and first-degree lacerations.

Instructors observe and provide guidance to users as they develop their technique.

Warning

Needles and sharps should be handled with care to avoid injury.

Note

- Users should be either experienced in the safe operation of the clinical equipment used with suture training or supervised.
- *To practice respectful care, the Labia Repair Model can be placed in front of a standardized patient, or the Vulva Trainer can be attached to MamaBirthie or another object.*
- Use proper and safe surgical instruments and technique.
- Users are responsible for the disposal of sharps and needles. Dispose of and recycle in accordance with the local legislation.

Care, Maintenance and Cleaning

Keep the Labia Repair Model clean and in a hygienic condition. Do not wash or tumble dry, as it could damage product.

Wipe the device with a clean cloth dampened with clean water, or Laerdal manikin wipes, then leave to dry.

Cautions

- *Do not submerge in any liquid*
- *Do not operate in rain or near water*

Symbol Glossary

Symbol	Definition
	Manufacturer
	Date of Manufacture
	Reference Number

Specifications

Size and Weight	
Dimensions	L 16 x W 11 x H 14 cm
Weight	95 g (\pm 5 g)
Environment	
Operating temperatures	10 °C to 40 °C
Storage temperatures	-15 °C to 50 °C
Relative humidity	20 % - 90 % RH non-condensing

Warranty

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com

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© 2023 Laerdal Medical AS. All rights reserved.