- EN
- ES
- IT
- FR
- DE
- NL
- NO
- SV
- FI
- ZH
- JA
- KO

# Nursing Anne

Directions for Use

# DIRECTIONS FOR USE

## Table of Contents

Items Included	1
Skills Taught	1
Teeth	2
Carotid Pulse	2
Tracheostomy Plug & Care	2
NG Tube Placement	2
Intubation	2
Lungs	2
Stomach Reservoir	2
Belly Plates	2
Injection Pads	3
Waist, Knee and Ankle Joints	3
Arms	3
Legs	3
IV Arm	3
Replacing Skin & Vein System	3
SimPad Connection	4
Blood Pressure Arm	4
Auscultation of Heart, Breath, Bowel Sounds and Fetal Tones	4
Genitalia	4
Colon Reservoir	5
Urinary Catheterization and Enema Simulation	5
Troubleshooting	5
Care and Maintenance	5
Replacement Parts	5

### Caution: Latex

This product contains **Natural Rubber** latex which may cause allergic reactions when in contact with humans.

The product, when carrying the CE-mark, is in compliance with essential CE requirements and other relevant provisions of council directive 1999/5/EC.

UKCA (U.K. Conformity Assessed): This product complies with the requirements of U.K. legislation of Electromagnetic Compatibility Regulations (S.I. 2016/1101).

## Items Included:

- (1) Full-Body Female Manikin
- (1) Articulating Blood Pressure Training Arm (Nursing Anne SimPad)
- (1) Articulating Female Multi-Venous IV Training Arm
- (1) Hospital Gown
- (1) Male Genitalia
- (1) Female Genitalia
- (3) Urinary Valves
- (3) Anal Valves
- (1) Simulated Blood Concentrate
- (1) 100 cc Syringe
- (1) Manikin Lubricant
- (1) Assembly Tool Kit

## Optional Accessories

### Nursing Anne Modules:

- (1) Breast Exam Module, with interchangeable abnormalities
- (1) Mastectomy Module with staples and drain
- (1) Fundus Module with interchangeable uteri

### Wound Care & Assessment Set:

- (1) Abdominal Incision Module with Painted Sutures
- (1) Abdominal Incision Module with Staples and Penrose Drain
- (1) Abdominal Incision Module with Nylon Sutures and Penrose Drain
- (1) Abdominal Subcutaneous Heparin and Insulin Injection Module
- (1) Abdominal Packing Module
- (1) Infected Colostomy Stoma
- (1) Vento-Gluteal and Gluteal Decubitus Ulcer Modules
- (1) Below Knee Amputation Stump
- (1) Thigh Packing and Irrigation Module
- (1) Thigh Suture Module
- (1) Thigh Debridement Module
- (1) Varicose Vein Leg with Stasis Ulcer
- (1) Diabetic Foot Module

### Skills Taught:

- Basic patient handling
- Denture care
- Oral hygiene
- Oral and nasal Intubation
- Eye and ear irrigation (simulated)
- NG Tube insertion, care medication administration and removal
- Lavage/Gavage
- Tracheostomy care and suctioning
- Blood pressure skills (When used with SimPad)
- IV care and management
- Subcutaneous and intramuscular injection
- Oxygen delivery procedures
- Ostomy irrigation and care
- Catheterization skills
- Enema simulation
- Colonic irrigation
- Wound assessment and care
- Bandaging and dressing
- Post-surgical mastectomy-care procedures
- Recognition of breast disorders, their sizes and relative locations
- Practice of fundus massage skills
- Auscultation and recognition of normal and abnormal heart, breath and bowel sounds and fetal tones (When used with SimPad)

Nursing Anne Advanced is a full-body, lifelike vinyl manikin designed to teach all skills from basic patient handling to advanced nursing, including the measurement of noninvasive blood pressure and the auscultation and recognition of normal and abnormal heart, lung and bowel sounds and fetal tones.

The manufacturing quality of this simulator should provide many sessions of training when reasonable care and maintenance are practiced.

*Laerdal Recommends:*

*IV Injections – 22 gauge needle, or smaller  
Tracheostomy Tube – Size 6  
NG Tube – Size 14 French  
Endotracheal Tube – 7.5 or smaller  
Laryngoscope Blade - Size #2 or #3, straight or curved blade  
Urethral Catheter – 16 French  
Enema Simulation – 7 mm*

**Teeth –**

**To remove:** Grasp firmly and lift.

*Insert and remove bottom teeth first when performing denture care.*

**To reinsert both sets:**

- Begin on one side and work around, aligning ridges with corresponding grooves in jaws.
- Press firmly into place.

**Carotid Pulse –**

To generate a carotid pulse, use the hand held red bulb located on the right axillary side of manikin. Firmly squeeze bulb while palpating for pulse at carotid site.

**Tracheostomy Plug –**

The tracheostomy plug may be removed by grasping firmly, then lifting up and out. To replace, press into hole. (Photo 1)

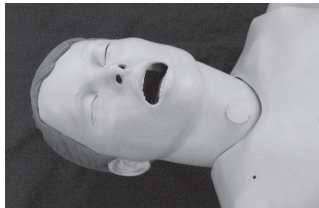
Photo 1

**Tracheostomy Care –**

- Mix solution of 1/2 cup mild liquid detergent and 1/2 cup water.
- Remove lungs.
- Pour mixture into simulated lungs.
- Fill lungs to a level where suction catheter will pick up mixture.
- Reattach lungs at connector.

The consistency of the solution approximates the mucus normally suctioned in a tracheostomy patient.

*We recommend a size 6 tracheostomy tube. Opening will accommodate other sizes, but a size 6 is usually most suitable.*

**Drain and air-dry lungs immediately after each use.**

**NG Tube Placement –**

This manikin features an esophagus and stomach reservoir for practice of NG tube insertion and skills such as lavage and gavage.

*For best results, lubricate the tube with either manikin lubricant or liquid detergent prior to NG or OG tube insertion.*

**Intubation –**

The following equipment is recommended:

- Endotracheal tube, size 7.5 ID or smaller
- Manikin Lubricant or liquid soap
- 10 ml syringe
- Laryngoscope blade, size #2 or #3 straight or curved.
- Laryngoscope handle
- NG Tube, size 14 French

Remove trach plug prior to insertion of intubation tubes.

*Intubation tubes and airway passages should be sprayed with manikin lubricant or lubricated with liquid soap prior to intubation.*

**Lungs –**

Lungs may be removed from bronchial tubes by unscrewing them at the connector site. To replace, reverse procedure.

**Stomach Reservoir –**

Stomach reservoir attaches to esophagus with a connector. To fill reservoir with water, use 100 cc syringe.

*Dusting with talcum powder is helpful in replacing belly plates.*

**Belly Plate with Multi-Sounds –**

**To remove:**

- Apply pressure to center of plate.
- Work your fingers under edge of plate while lifting.
- Disconnect bowel sound speaker connector to release plate.

**To insert:**

- Position plate over cavity.
- Reconnect bowel sound speaker.
- Work plate edges into groove on pelvis cavity.

**Belly Plate with Colostomy Stoma –**

**To remove:**

- Apply pressure to center of plate.
- Work fingers under edge while lifting. (Photo 2)

Photo 2

**To insert:**

- Attach valve and genitalia.
- Work plate edges into pelvis cavity.

The belly plate with colostomy also contains the urinary reservoir and colostomy stoma.

**To fill reservoir:**

- Remove colostomy stoma.
- Fill with fluid.
- Gently reinsert stoma.

*Make sure genitalia and urinary connectors are in place prior to filling.*

**Injection Pads –**

- To remove, squeeze pad with fingers and pull.
- To replace, squeeze pad and insert in opening.

*Dusting with talcum powder is helpful in replacing belly plates.*

Injection pads may be injected with water. Foam should be removed from pads immediately following training. Squeeze out fluid and air dry.

**Waist, Knee and Ankle Joints –**

These joints are connected with a bolt and lock nut. To remove, unscrew lock nut and pull bolt from joint. Segments will now easily disconnect.


*Attachment hardware is located in envelope inside carton.*

**Arms –**

Arms are attached with a 4 1/2" long bolt, three washers, a spring and a wing nut.

**To attach:**

- Unfasten chest skin from shoulders.
- Pull skin back to expose chest cavity.
- Locate hole in shoulder.
- Remove deltoid injection pad from arm by squeezing skin in center of pad while pulling.
- Slide small washer over bolt.
- Insert bolt through hole in arm and hole in manikin's shoulder.
- Add large washer, spring, another large washer and wing nut to bolt inside manikin's chest. (Figure 1)
- Use a screwdriver to hold bolt steady while tightening wing nut.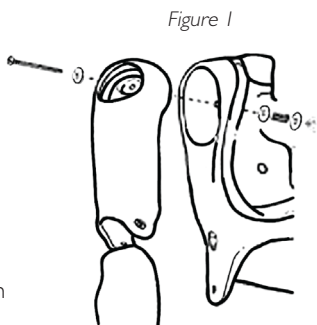

To remove arms, reverse procedure.

**Legs –**

Legs are attached with a 6" long bolt, a spring, a large washer, a small washer and a wing nut.

*Attachment hardware is located in envelope inside carton.*

**To attach:**

- Remove belly plate and colon reservoir from manikin to locate hole in hip.
- Remove thigh injection pad from leg by squeezing skin in center of pad while pulling.
- Slide spring and small washer over bolt.
- Insert bolt through hole in thigh.
- Insert bolt through hole in pelvis.

- Slide large washer and wing nut over bolt.
- Use screwdriver to hold bolt while tightening wing nut inside pelvis. (Figure 2)

To remove legs, reverse procedure.

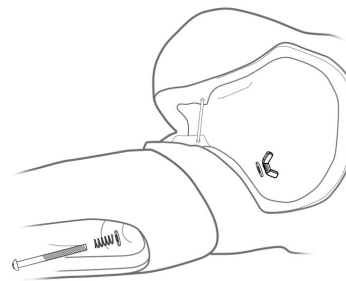
Figure 2

**IV Arm – Multiple Venipuncture sites:**

- Dorsal Veins of Hand (3)
- Antecubital
  - Cephalic Vein
  - Median Vein
  - Basilic Vein

*A 22 gauge needle or smaller is recommended to extend the life of the IV Arm.*

*When using an IV catheter, lubricate with manikin lubricant for easier insertion.*



**Directions for Use:**

- Attach IV bag to IV tubing.
- Attach IV tubing to either vein.
- Allow fluid to flow through arm and out other vein.
- Clamp off flow of water from open vein.

The arm is now ready to practice venipuncture.

**Replacing Skin and Veins:****I. Remove skin:**

This can be done with a sharp knife, scalpel or scissors. (Figure 3) When finished, discard skin. (Figure 4)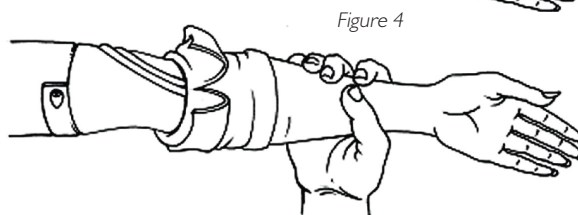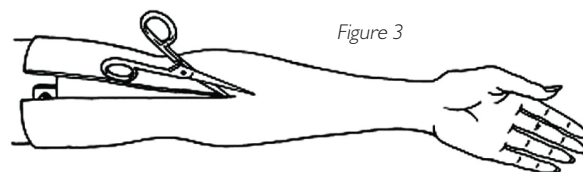

**Replacing veins:**

1. Remove tubing from track in mandrel. Glue may need to be scraped away to allow removal.
2. Rinse and dry vein grooves well and swab with alcohol. Be sure to remove any excess glue.
3. Place new veins along grooves, (Figure 5) spot gluing as needed. (We recommend a fast-drying glue.)

**Replace skin:**

1. Generously lather arm mandrel with liquid soap.
2. Slide hand into skin.
3. Work skin over fingers, as with a glove.
4. Work arm skin up, over mandrel.

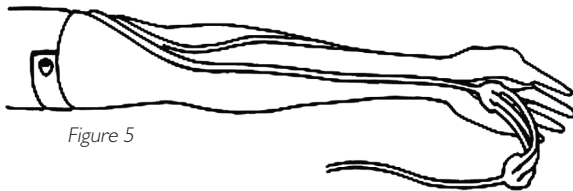
Figure 5

**SimPad Connection**

1. Connect Nursing Anne to Link Box, via cable located on back of manikin. (Photo 3).
2. Connect clear tubing exiting manikin together with the cable.

Photo 3

**Blood Pressure Arm –**

1. Place Blood Pressure Cuff on arm.
2. Attach tubing on cuff to clear tube, located underneath Nursing Anne's arm. (Photo 4)

Photo 4

See SimPad DFU for complete Blood Pressure operating procedures.

*Do not insert needles into blood pressure arm.*

Auscultation of Heart, Breath and Bowel Sounds and Fetal Tones – See SimPad DFU for complete operating procedures.

**Genitalia:**

Both male and female genitalia have been provided for urinary catheterization and enema training procedures.

Pelvic support pin must be removed and replaced immediately when assembling and disassembling genitalia. (Figure 6)

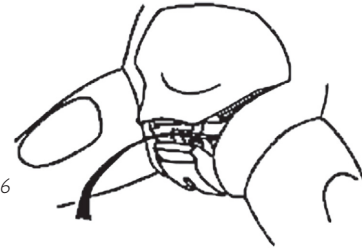
Figure 6

**Pelvic support pin must be replaced.**  
 Failure to replace pin will result in pelvic spread.  
 If this occurs, the genitalia module will no longer fit properly in the manikin.

**To disassemble:**

1. Lift up on belly plate.
2. Pull upper portion of genitalia back to remove pelvic support pin.
3. Belly plate, genitalia with connectors, and colon reservoir may now be removed simultaneously.

**To assemble: (Photos 5, 6 & 7)**

1. Screw both valves onto genitalia. When attaching valves, be sure alignment of genitalia, urinary reservoir (belly plate) and colon reservoir are as illustrated in Figure 7.
2. Lift up on edge of belly plate.
3. Pull down upper portion of genital.
4. Insert into pelvic cavity.
5. Replace the pelvic support pin.
6. Press belly plate into position.

Figure 7

Photo 5  
Slide tube to meet ridge on connector:

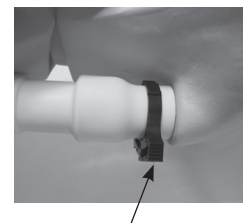
Photo 6  
Area where clamp is placed.

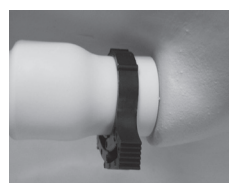
Photo 7  
Apply clamp and squeeze to tighten.

**Colon Reservoir –**

Colon reservoir inserts into pelvic cavity with narrow end downward and connector pointing outward. This connector attaches reservoir to anal valve on genitalia. (Photo 8)

Photo 8

Reservoir should be removed from manikin, inverted, drained and completely air-dried before storing.

**Urinary Catheterization and Enema Simulation –**

*Catheters should be well lubricated with manikin lubricant or full strength liquid soap prior to using.*

After several uses, catheter may “hang” when water has washed away lubricant, causing friction and binding. If this occurs, generously lubricate a catheter and insert into urethra several times.

If catheter hangs during removal, work it with an “in and out” motion. Use warm water in reservoirs when possible to keep soap from clogging the catheters. Make sure reservoirs are well drained of all water before disconnecting valves.

**Check List for Valves That May Leak:**

- a. Do valve connections have an O-ring?
- b. Does the white clamp fit firmly against base of valve?
- c. Are valves attached correctly?
- d. If valves are attached correctly, are they properly connected to reservoirs and to genitalia?

*A size 16 French catheter is suggested for urethral catheterization.  
Size 7mm catheter is suggested for enema simulation.*

**Troubleshooting –****What can I do if my manikin develops a pelvic spread?**

Cinch pelvis with a belt and heat with a hair dryer to soften, cinching as you heat. Insert pin, leaving belt in place until cool.

**What do I do if my connectors leak?**

Make sure C-clamps are squeezed tightly into place and connectors are screwed on tightly.

**What can I do if I can't remove my catheter?**

Make sure the cuff is deflated.

**Care and Maintenance –**

1. Clean with mild soap and water; do not submerge manikin or parts in cleaning fluids or water. Apply a light coat of talcum powder to the face and chest skin to achieve skin-like feeling.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, each manikin should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed. After use of injection pads (use water only), accumulated water should be squeezed out. Do not store wet foam pads in the skin. To prevent mildew or mold, pads can be soaked in a mild solution of disinfectant and water or bleach and water. Squeeze excess solution from pads, allow them to dry, then store or reinsert in manikin.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.

**Replacement Parts:**

For latest version of Spare Parts and Accessories, visit [www.laerdal.com](http://www.laerdal.com)

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© 2022 Laerdal Medical AS. All rights reserved.